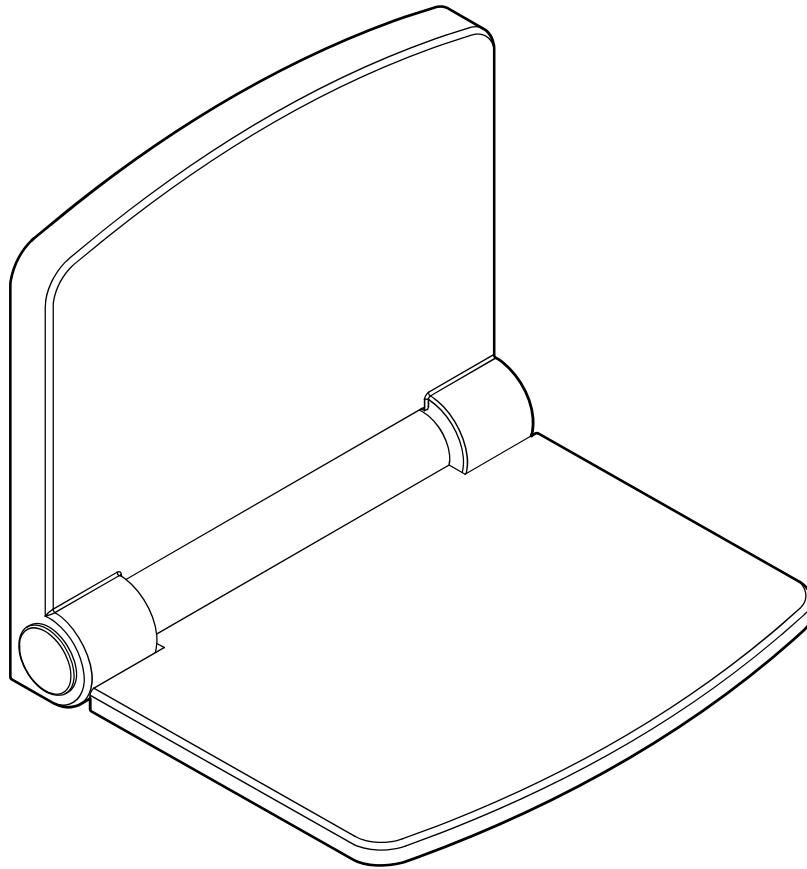
# INVISIA

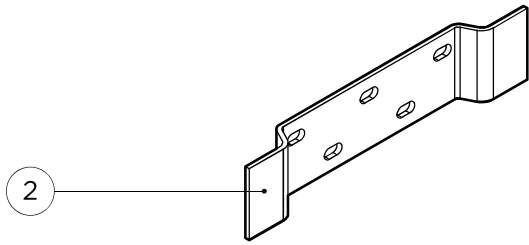
## **Fold Down Seat**


FDS-S  
(DC190 - RevB)

[invisiacollection.com](http://invisiacollection.com)


# A

 FE  
0.75lbs  
0.34kg

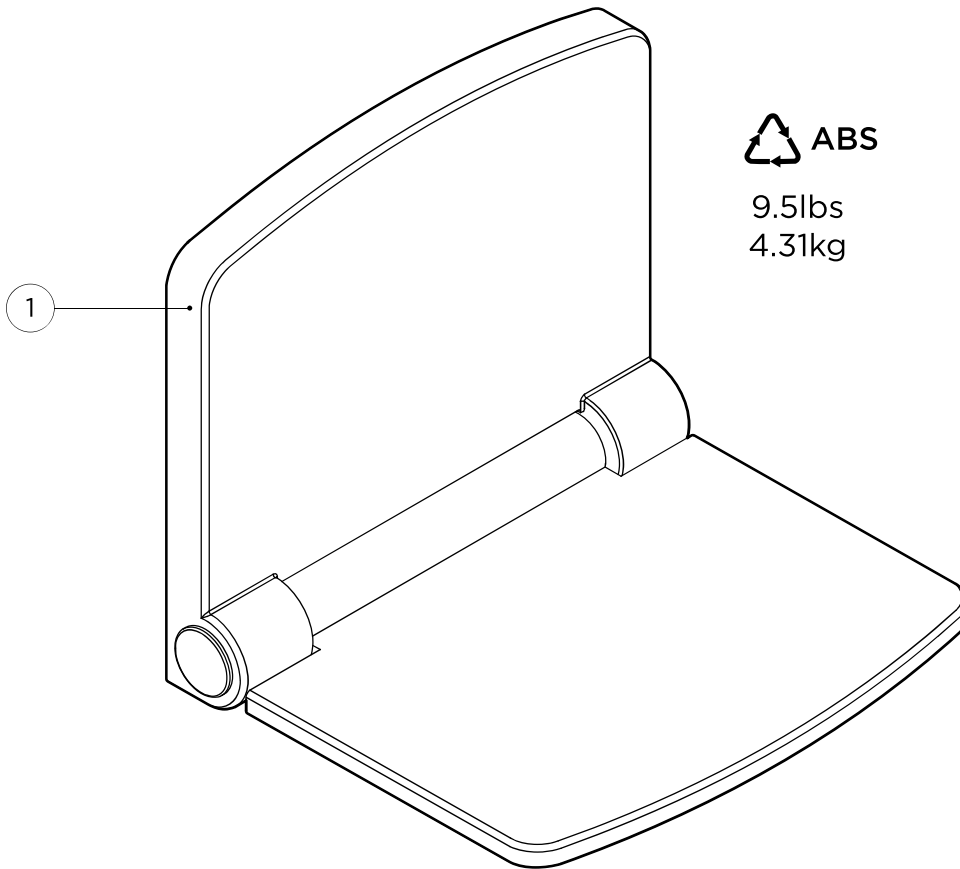
 ABS  
9.5lbs  
4.31kg

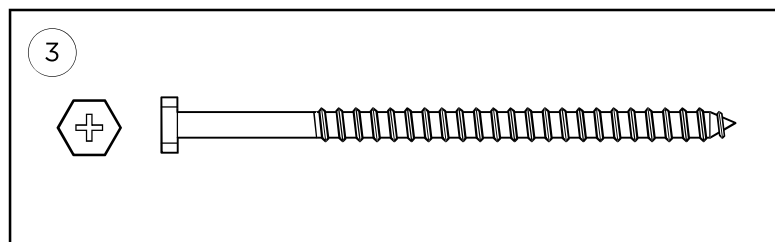
**MAX**

440lbs  
200kg

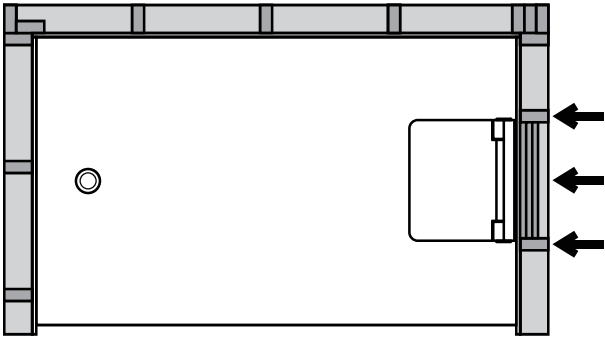
1


2

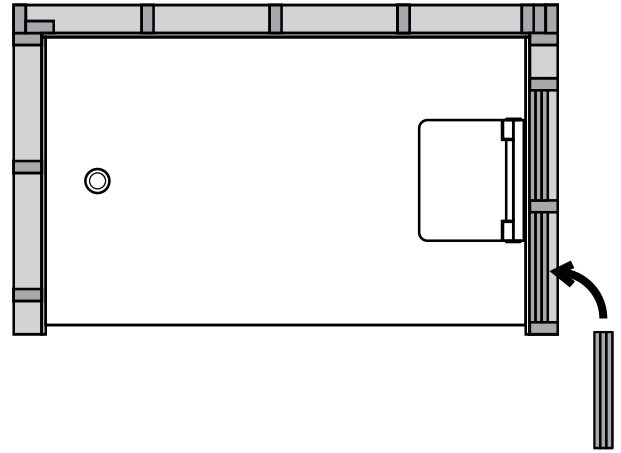
3

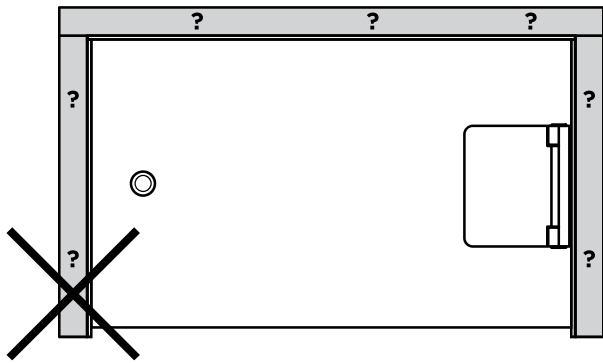
4

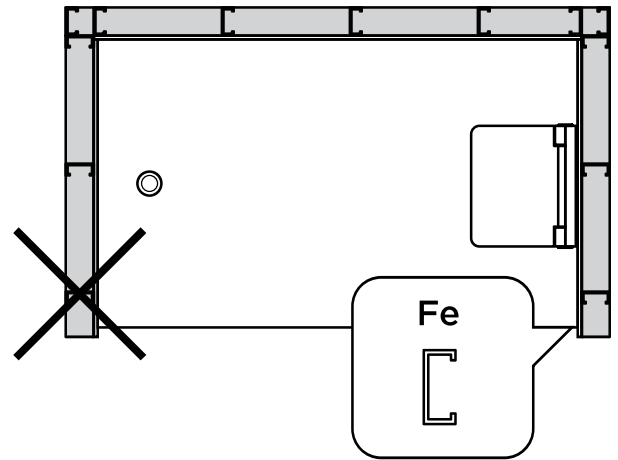
5a

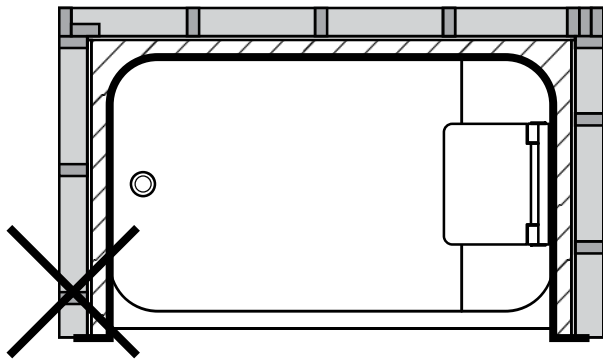
5b

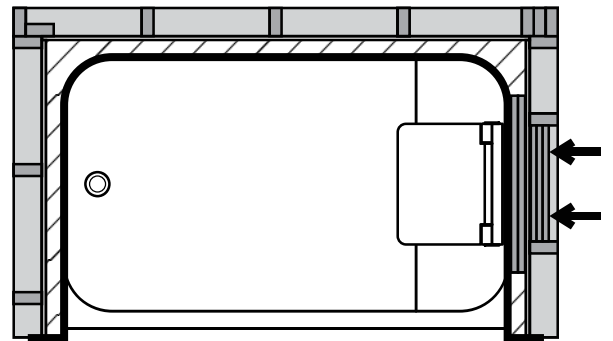
6

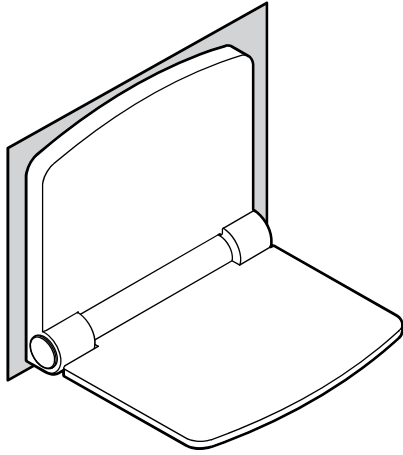
7

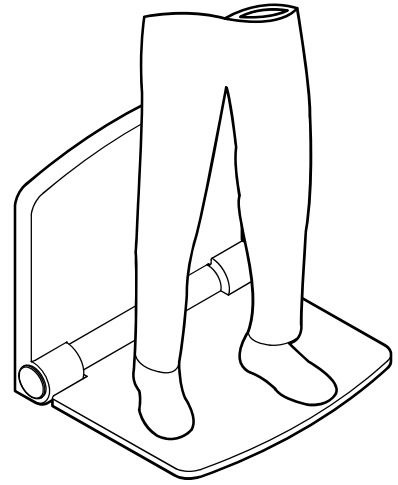
8

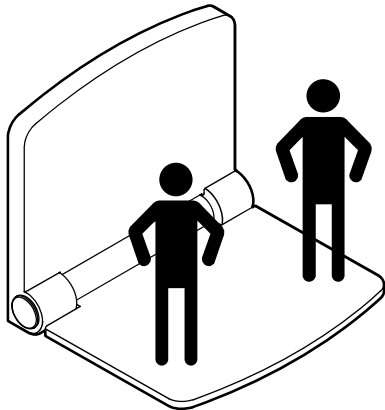9


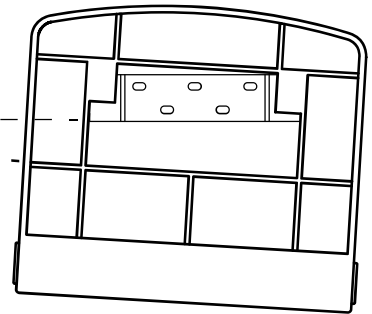
10

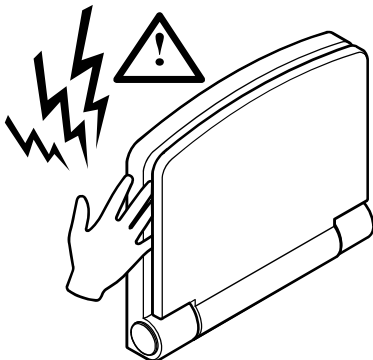
11

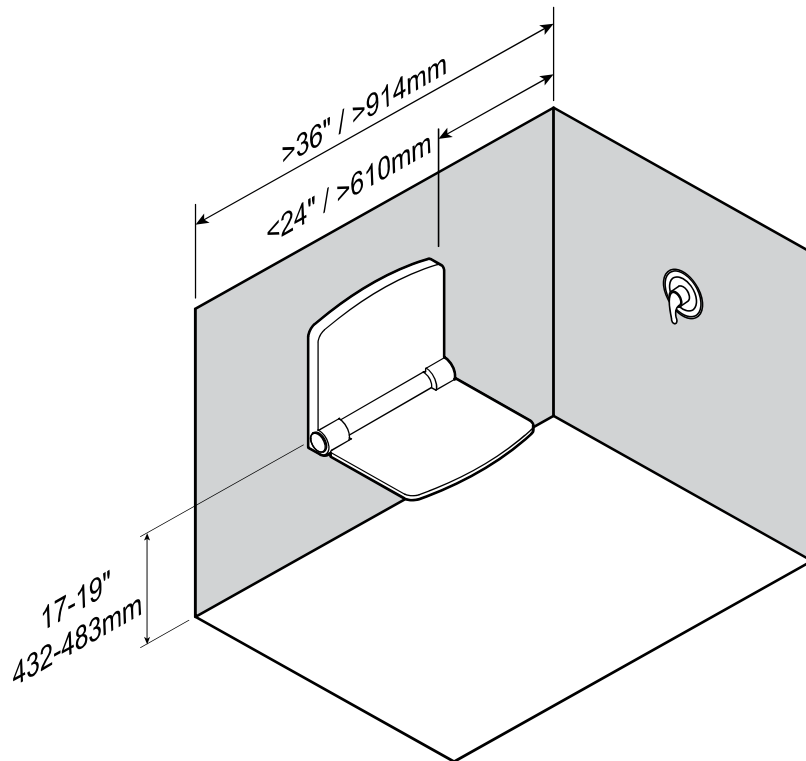
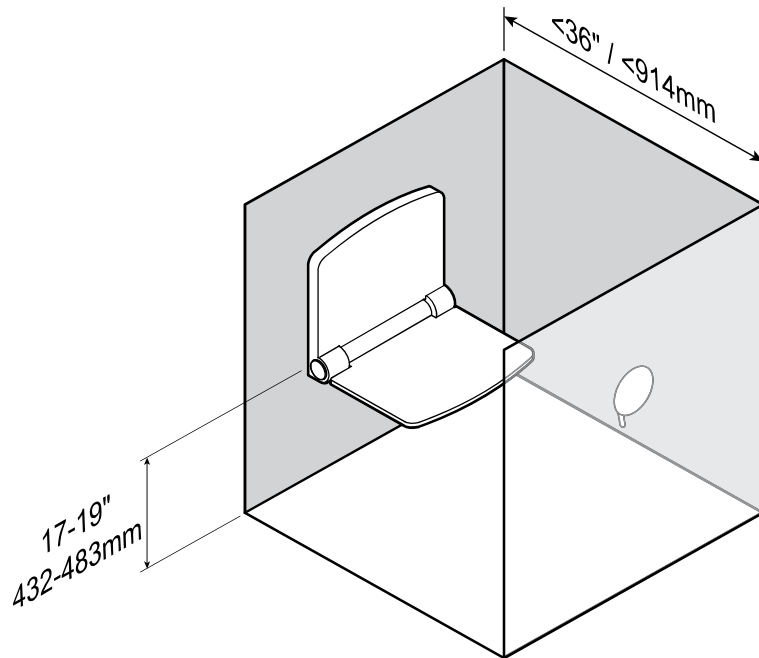
1

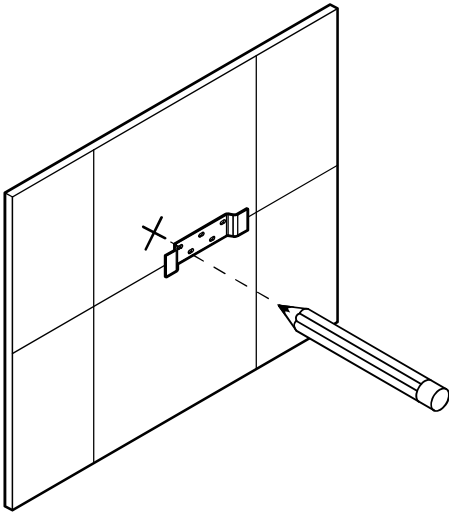
2

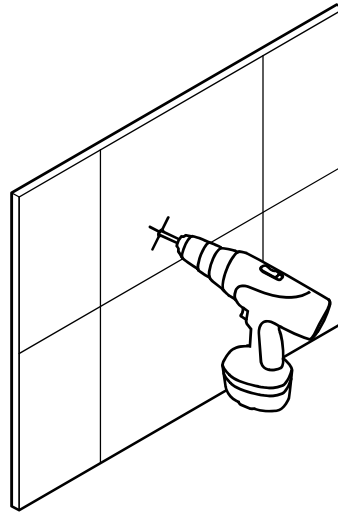
3

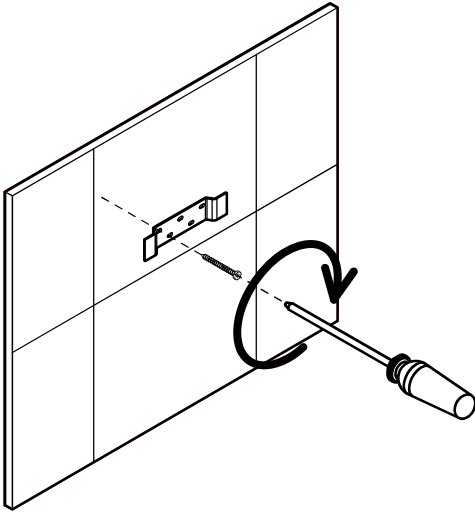
4

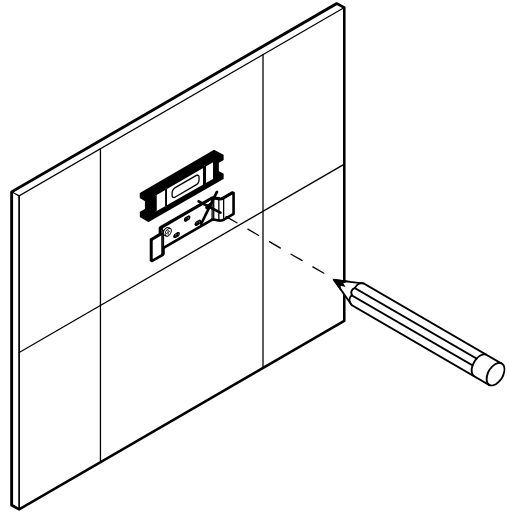
5

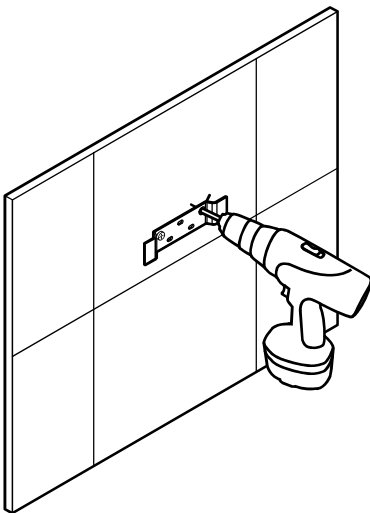
6

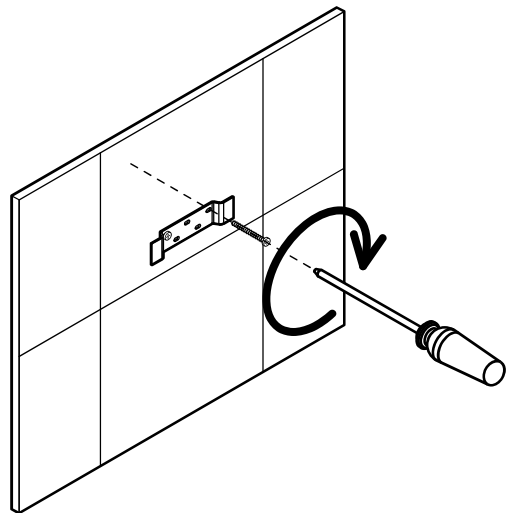
7







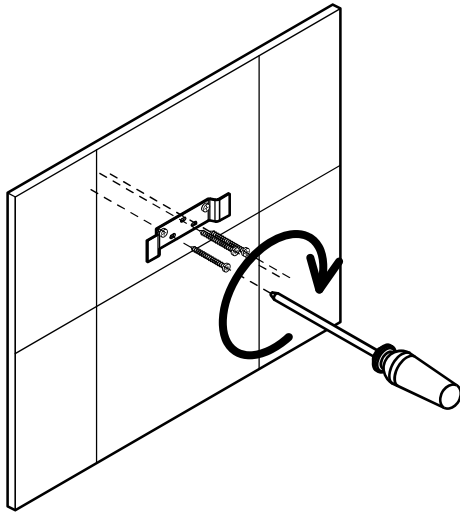
8

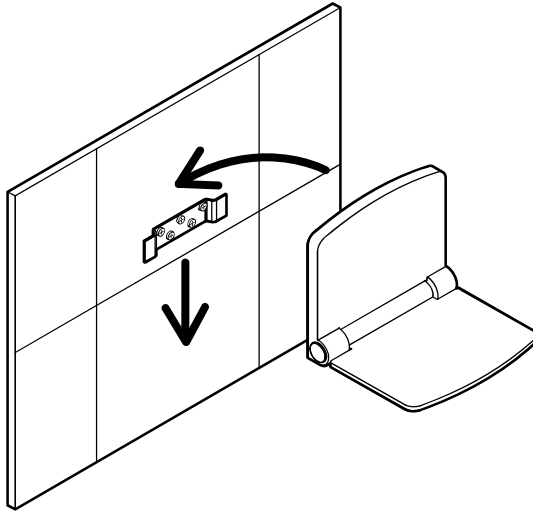9

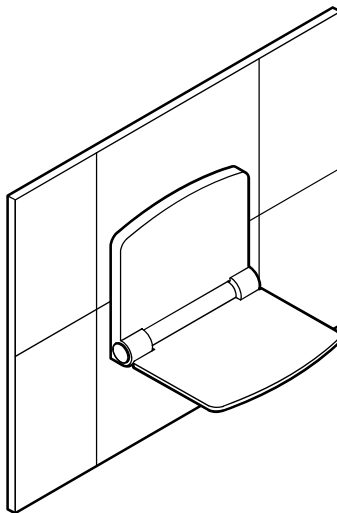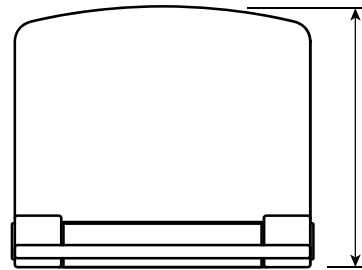
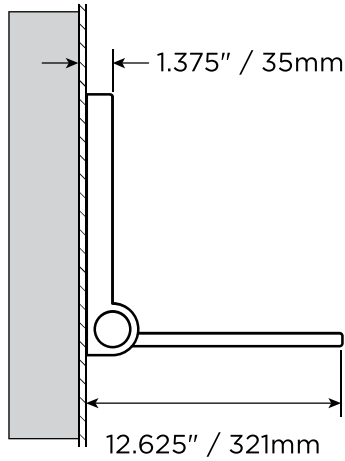
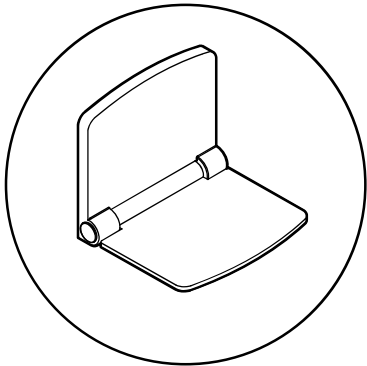
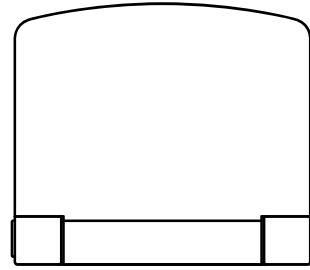
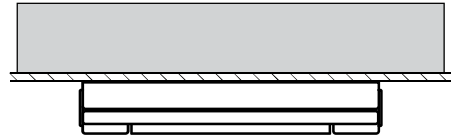
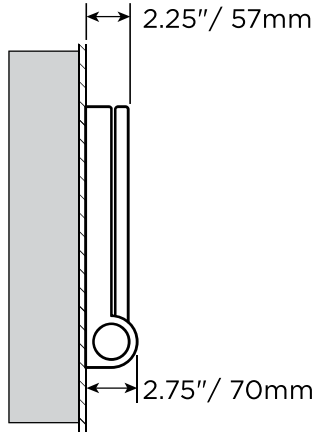
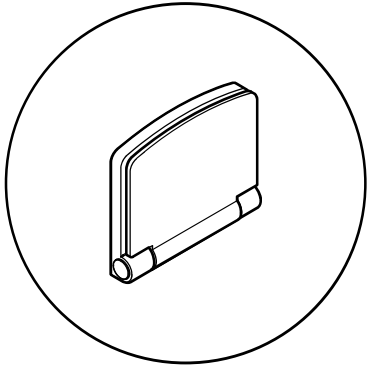
9.5" / 241mm

12.625" / 321mm

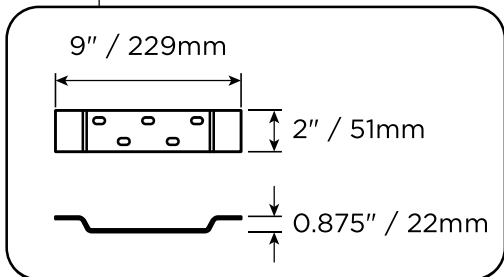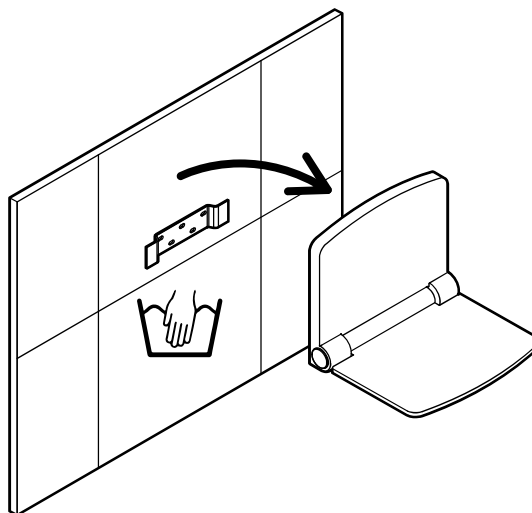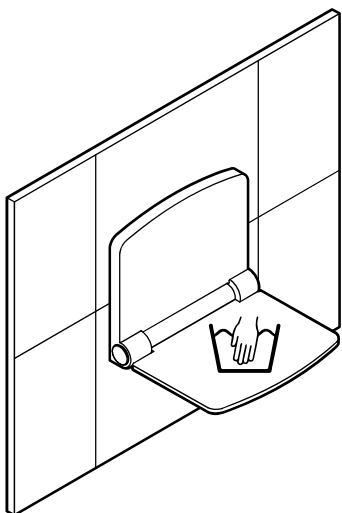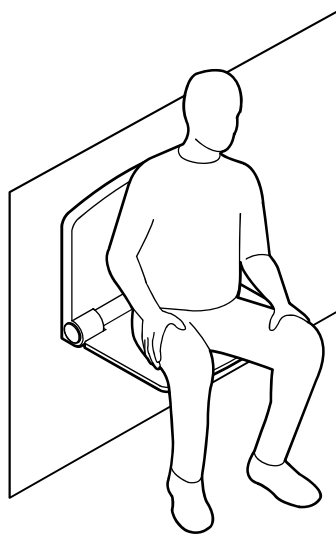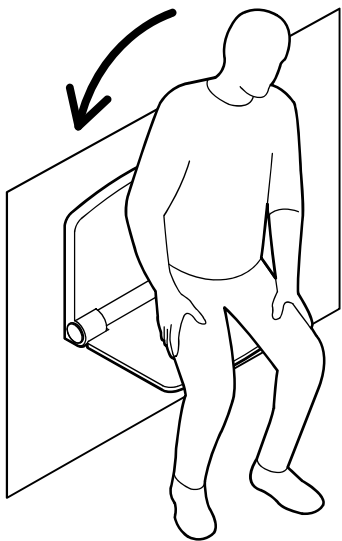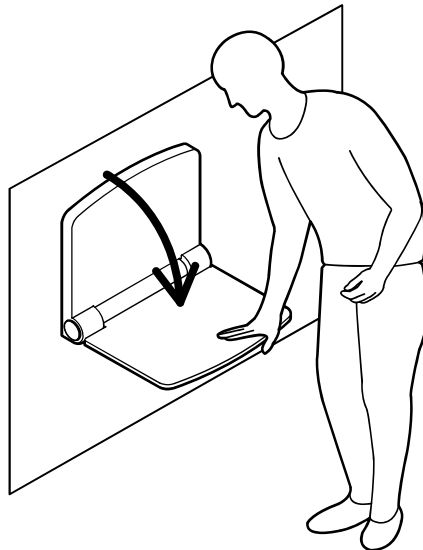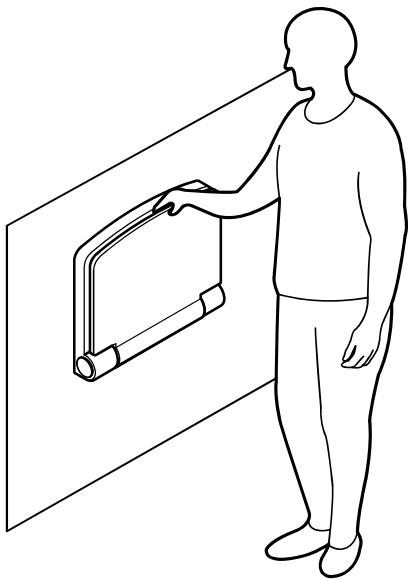14.5" / 368mm

# English

## OVERVIEW

Thank you for choosing the Fold Down Seat from Invisia. Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your Fold Down Seat is properly assembled, installed, and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your Fold Down Seat installed by a qualified contractor.

The Fold Down Seat is a high quality, commercial grade, fold down shower seat. The Fold Down Seat is intended to provide shower seating support for people with reduced mobility. The Fold Down Seat is intended to bear weight ONLY when the seat is in the down position; the seat is not intended to bear weight in any other position. The Fold Down Seat is intended to be folded up when not in use. The hinge allows the seat to open slowly, and lock against the wall in the folded position. The Fold Down Seat is installed with appropriate fasteners to a structural wall. The maximum user weight for this product is 440lbs / 200kgs. Note: Product is not intended to support full body weight (i.e. do not stand on the seat). The Fold Down Seat is not to be used in any other way than described above.

## ONE YEAR WARRANTY

Products are covered by a one year warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, or the use of corrosive or abrasive cleaning products.

Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described hereon; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon.

Specifications subject to change without notice.

HealthCraft Products Inc.  
2790 Fenton Road  
Ottawa, Canada  
K1T 3T7

Gate 88  
Kanalgatan 45B  
Skellefteå, Sweden  
93123

## DELIVERY CONTENTS / PARTS

## FIGURE A.

1. Seat
2. Bracket
3. #14 x 2.5" Wood Screw, S/S (x5)

## INSTALLATION WARNINGS

## FIGURE B.

1. Seat must be installed to walls that are structurally sound.
2. If desired installation location does not have adequate structure, add wood blocking.
3. CAUTION - Do not install if structure behind walls is unknown.
4. CAUTION - Do not install into metal studs.
- 5a. CAUTION - Do not install in fiberglass / acrylic shower enclosures with insufficient stiffness and strength. 5b. Add wood blocking if required.
6. CAUTION - Ensure wood blocking behind wall supports entire footprint of seat.
7. CAUTION - Do not use seat for purposes other than sitting.
8. CAUTION - Do not seat more than 1 person at a time.
9. CAUTION - Ensure seat is fully supported by bracket prior to sitting.
10. CAUTION - Pinch points present when folding seat.
11. CAUTION - Seat and bathtub are slippery when wet.

## SUGGESTED FOLD DOWN SEAT LOCATIONS

## FIGURE C.

To determine the optimal mounting location, we recommend consulting with an appropriate healthcare professional. If side wall (to faucet) is less than 36" / 914mm, seat can be mounted on the back wall. If side wall (to faucet) is greater than 36" / 914mm, seat can be mounted on side wall (less than 24" / 610mm from faucet wall). ADA recommend seat height from finished floor 17-19" / 432-483mm.

## INSTALLATION

## FIGURE D.

1. Mark the location on the wall of the top hole of the bracket.
2. Drill pilot hole for fastener.
3. Fasten the bracket to the wall using appropriate fastener and waterproofing.
4. Use a level to ensure the bracket is level and mark location on the wall of other hole.
5. Drill pilot hole for fastener.
6. Fasten the bracket to the wall using appropriate fastener and waterproofing.
7. Repeat step 5 and 6 for remaining holes (x3).
8. Slide seat onto bracket completely.
9. Inspect hardware regularly, and tighten if required.

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