
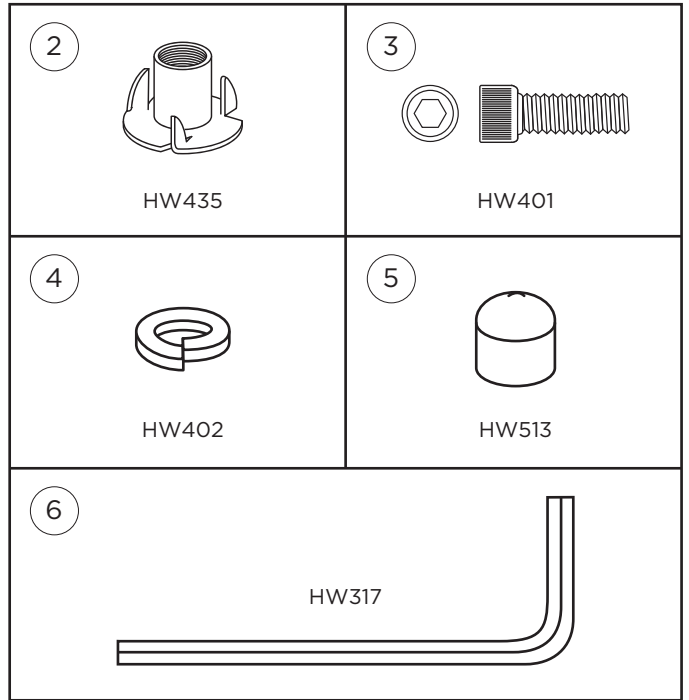
## **P.T.Rail™ Wood Wall Plate**

PT-WP-WD  
(DC138 - RevC)


[healthcraftproducts.com](http://healthcraftproducts.com)

# A

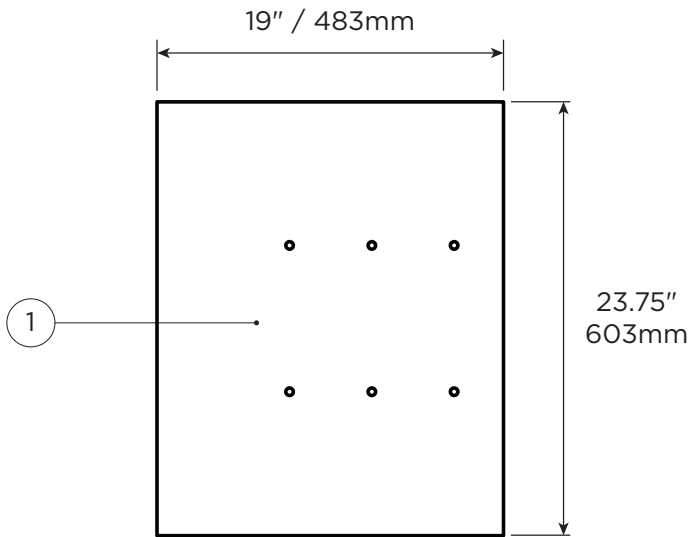
# B


## 1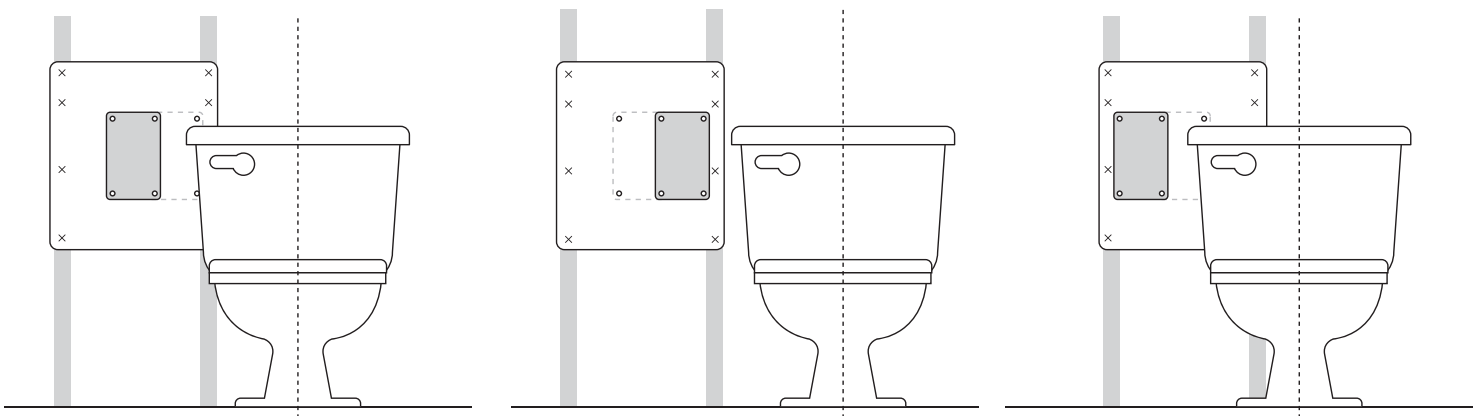


2

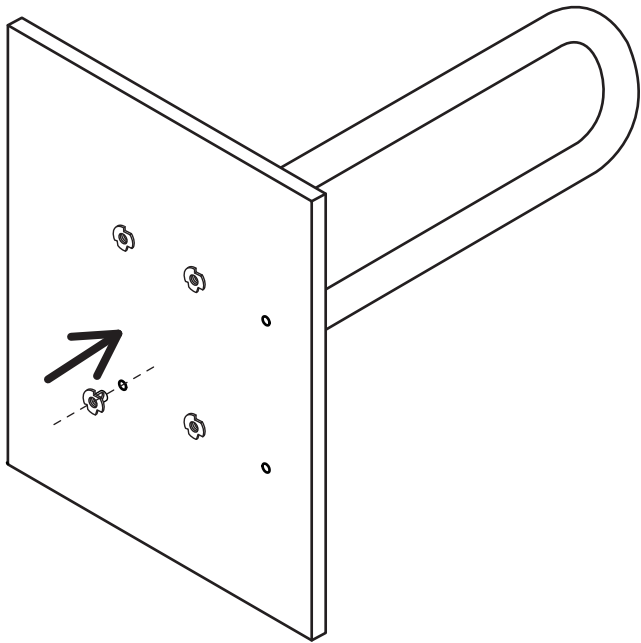
3

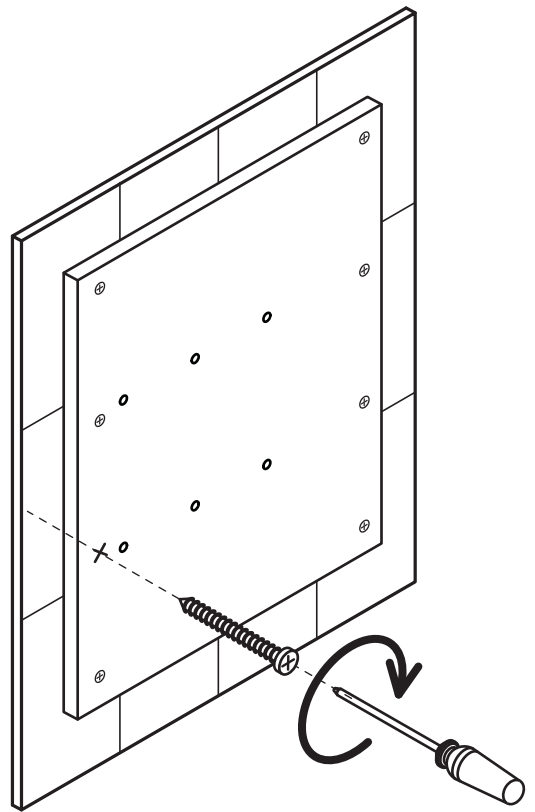4

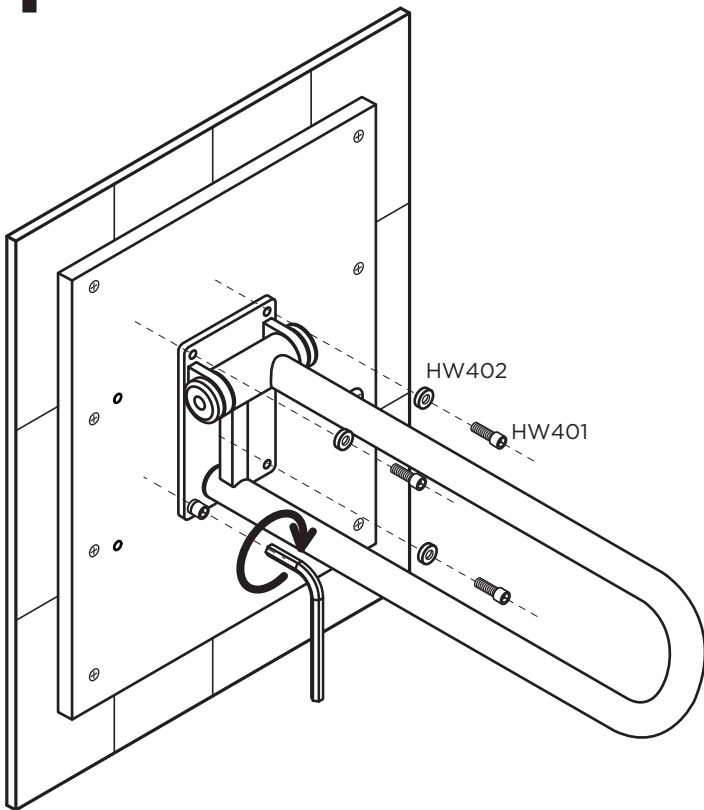5
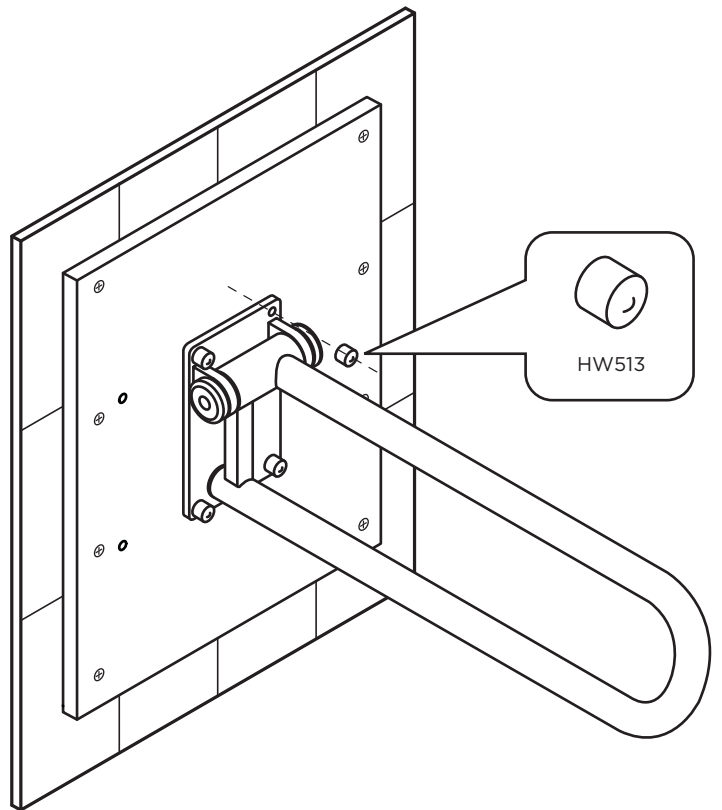
## OVERVIEW

Thank you for choosing the P.T.Rail Wood Wall Plate from HealthCraft. Please read and understand the instructions in this manual (and the P.T.Rail Manual DC110); keep manuals for future reference.

The Wood Wall Plate is a high quality installation accessory to be used with the P.T.Rail. The Wood Wall Plate is intended to secure to the wall and provide a convenient structure to mount the P.T.Rail. The Wood Wall Plate is only to be used with HealthCraft's P.T.Rail. The Wood Wall Plate is not to be used in any other way than described above. The Wood Wall Plate is covered under P.T.Rail warranty (DC110). Specifications subject to change without notice.

## DELIVERY CONTENTS - WOOD WALL PLATE FIGURE A.

1. Wood Wall Plate
2. T-Nut, 3/8-16, 3 Prong (x4)
3. Bolt, 3/8-16 x 1", S/S (x4)
4. Lock Washer, 3/8", S/S (x4)
5. Vinyl Cap (x4)
6. Hex Key, 5/16"

## INSTALLATION INSTRUCTIONS

## FIGURE B.

1. Determine desired location of P.T.Rail on wall. The Wood Wall Plate must extend over and secure to structural wood studs (inside the wall) for proper installation. There are 2 sets of pre-drilled holes for mounting convenience of the Wood Wall Plate, depending on location of toilet in relation to wood studs. Plate can be reversed to accommodate installations on opposite side of toilet. Extra holes can be added to mount the Wood Wall Plate to wood studs. If extra holes are required, use P.T.Rail as drilling guide. NOTE: All extra holes must be 1.5" / 38mm from all edges.
2. Install T-Nuts (x4) into back of Wood Wall Plate. Use 3/8" bolts and P.T.Rail on front side to pull T-Nuts tight into back of Wood Wall Plate. Once T-Nuts are secured, remove P.T.Rail from Wood Wall Plate.
3. Secure Wood Wall Plate to wall using appropriate fasteners (not included).
4. Tighten bolts to secure P.T.Rail in place.
5. Install vinyl caps to cover bolt heads.

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