
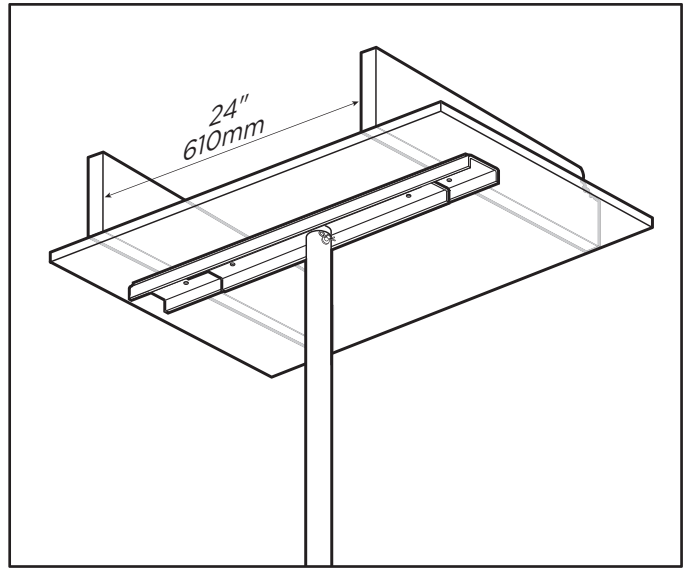
## Ceiling Plate Extender™


SP-CPE  
(DC128 - RevB)

[healthcraftproducts.com](http://healthcraftproducts.com)

# A

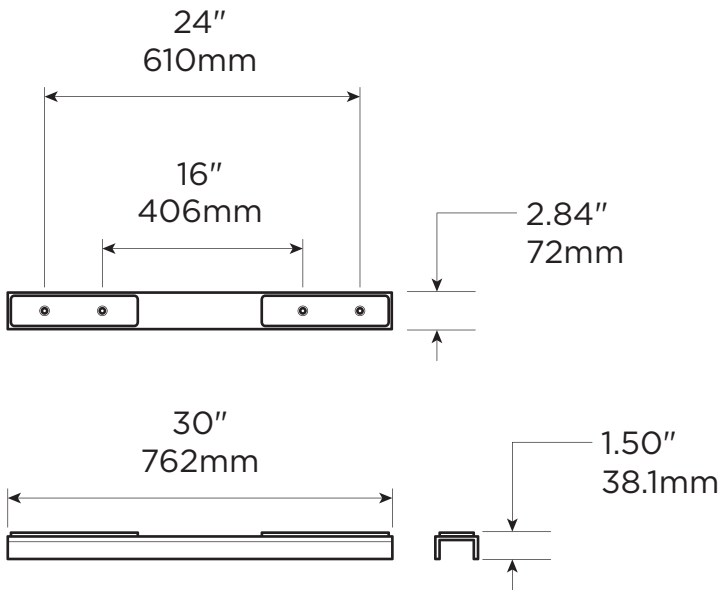
# B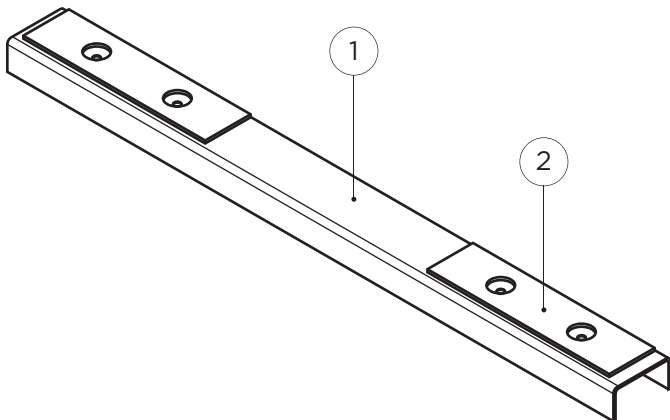


FE / NBR


5.20lbs  
2.36kg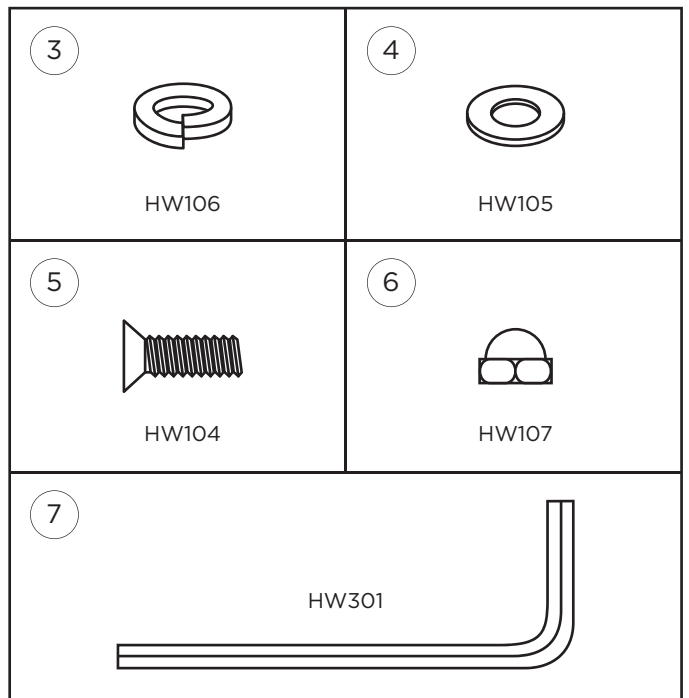


1

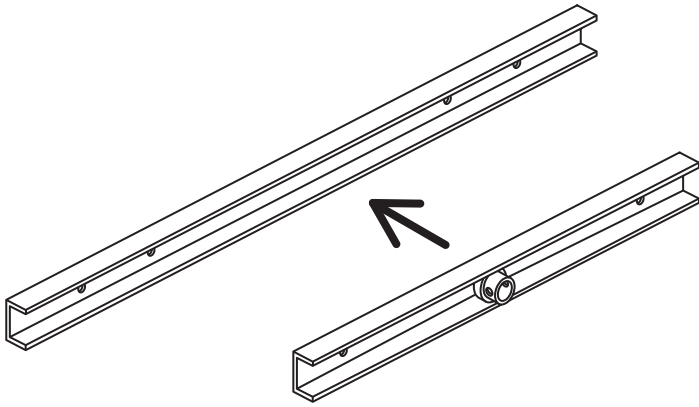
2

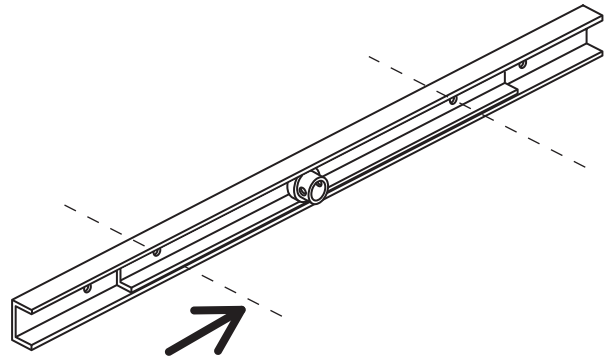3

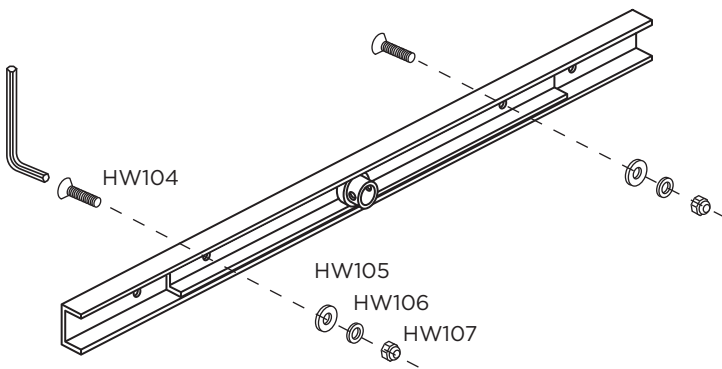4
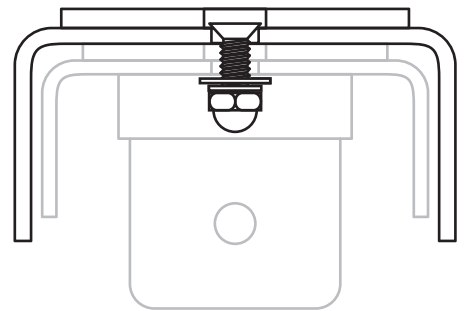
## OVERVIEW

Thank you for choosing the Ceiling Plate Extender from HealthCraft. Please read and understand the instructions in this manual (and the SuperPole Manual DC100); keep manuals for future reference.

The Ceiling Plate Extender (CPE) is a high quality, commercial grade installation accessory to be used with the SuperPole. The CPE is intended to increase the top plate span of the SuperPole from 20-30" / 508-762mm. It can be used when the SuperPole is expected to be subjected to more aggressive use, or where structural ceiling joists have increase spacing (such as 24"/610mm between centers). The CPE is only to be used with HealthCraft's SuperPole System. The CPE is not to be used with the Angled Ceiling Plate. The CPE is not to be used in any other way than described above. The CPE is covered under SuperPole warranty (DC100). Specifications subject to change without notice.

## TECHNICAL DATA

See Figure A.

## DELIVERY CONTENTS - CPE

## FIGURE B.

- |                           |                     |
|---------------------------|---------------------|
| 1. Ceiling Plate Extender | 5. Lock Washer (x2) |
| 2. Top Plate Rubber       | 6. Acorn Nut (x2)   |
| 3. 1/4-20 x .75" Screw    | 7. 1/8" Hex Key     |
| 4. Flat Washer (x2)       |                     |

## INSTALLATION INSTRUCTIONS

## FIGURE C.

1. Place SuperPole top plate inside the CPE.
2. Align holes of the top plate with the middle holes on the CPE.
3. Insert flat head screw thru holes on CPE, and install flat washer, lock washer and acorn nut.
4. Tighten the screw until flush with metal plate of CPE.
5. Continue with SuperPole instructions (DC100).

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