
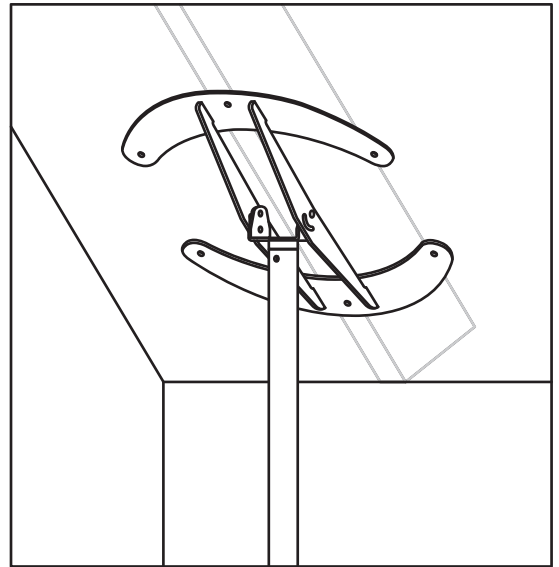
## Angled Ceiling Plate™


SP-ACP  
(DC102 - RevB)

[healthcraftproducts.com](http://healthcraftproducts.com)

# A

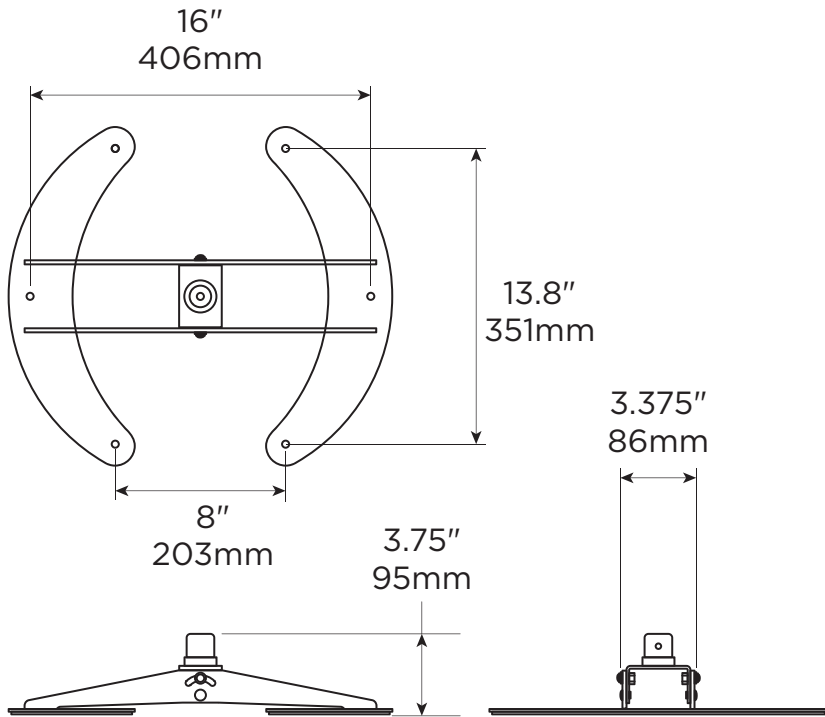




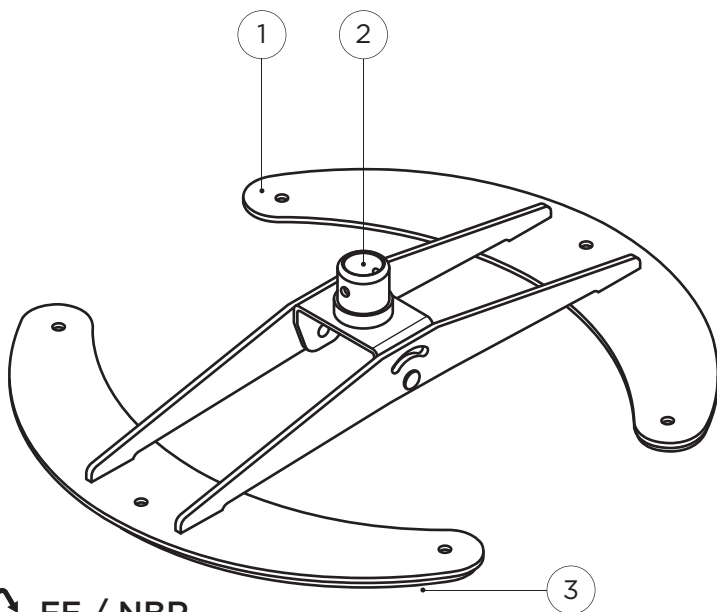

# B

FE / NBR

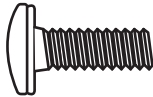5.30lbs  
2.40kg

<p>4</p>  <p>HW155</p>	<p>5</p>  <p>HW154</p>
<p>6</p>  <p>HW151</p>	<p>7</p>  <p>HW161</p>
<p>8</p>  <p>HW322</p>	
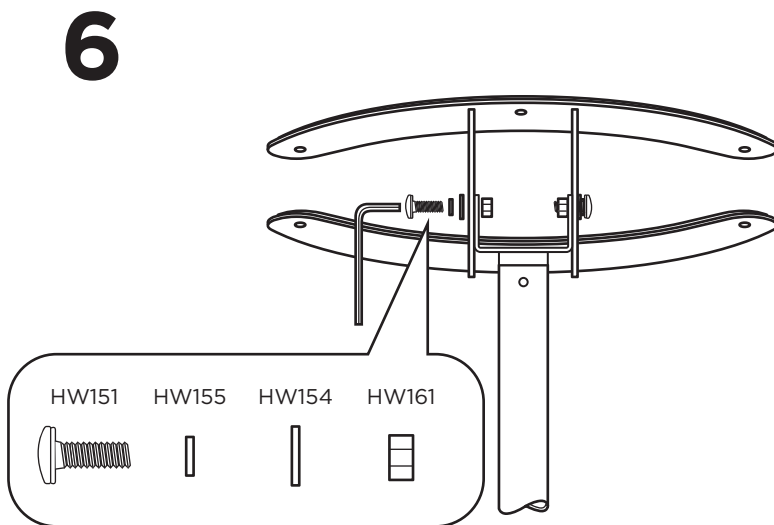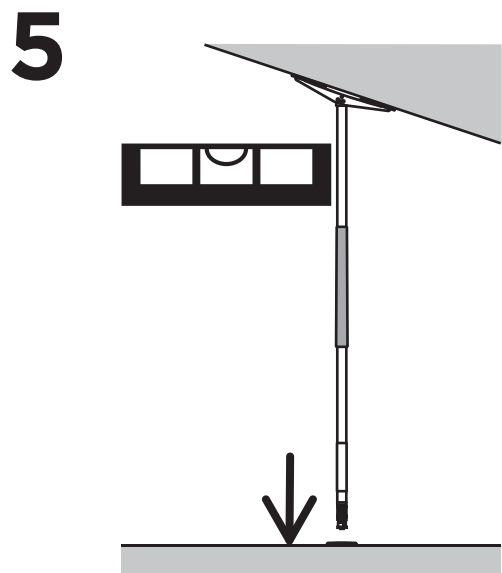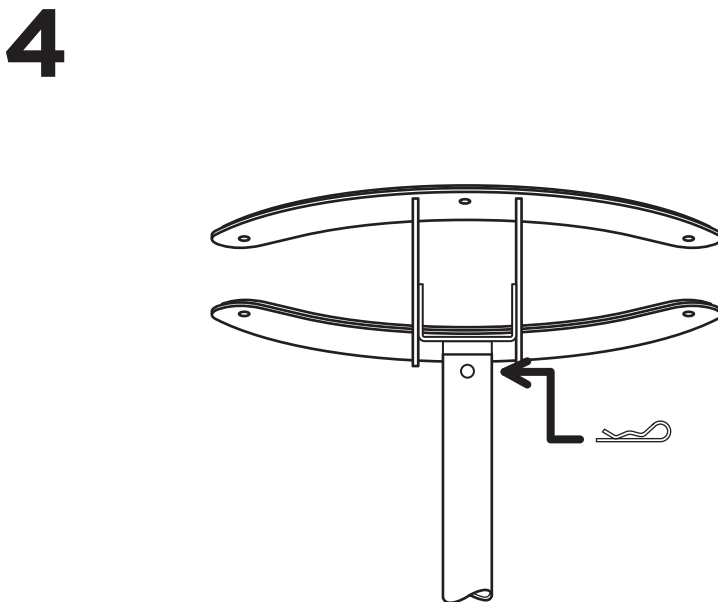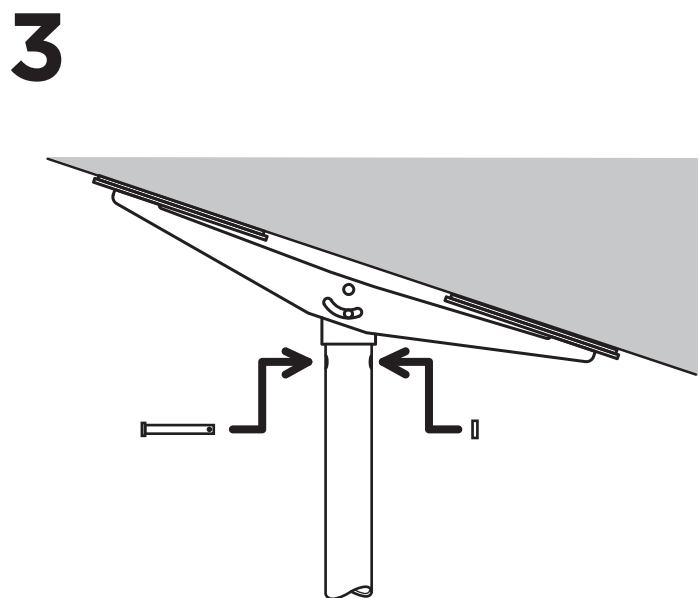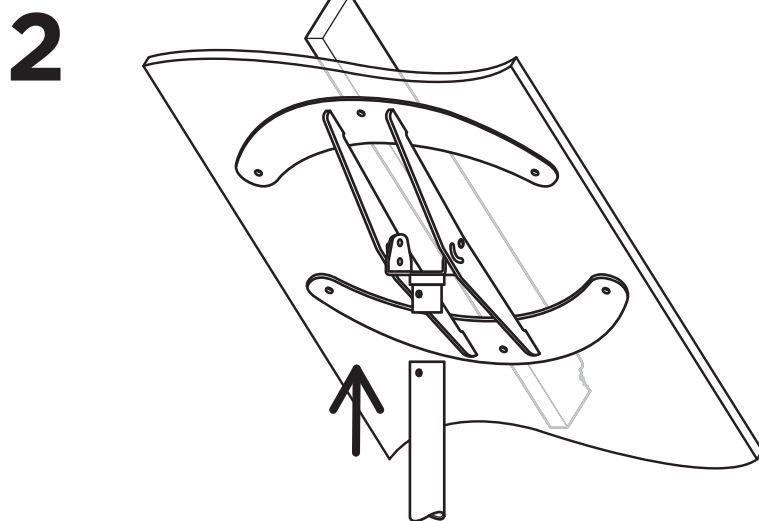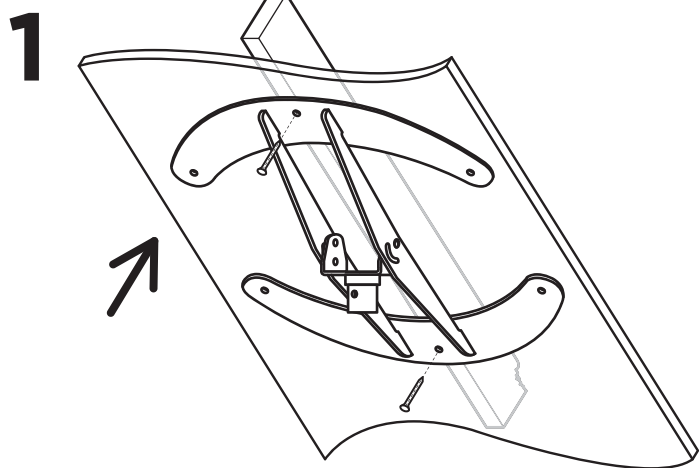
## OVERVIEW

Thank you for choosing the SuperPole System from HealthCraft. Please read and understand the instructions in this manual (and the SuperPole Manual DC100); keep manuals for future reference.

The Angled Ceiling Plate (ACP) is a high quality, commercial grade installation accessory to be used with the SuperPole. The ACP is intended to permit the SuperPole to be installed in areas where the ceiling is vaulted or angled. The ACP attaches to the top of the pole and is adjustable in angle from 0-45°. The ACP must be secured to ceiling joists with appropriate fasteners. The ACP is only to be used with HealthCraft's SuperPole System. The ACP is not to be used with the SuperPole Bariatric, SuperPole Ultra and Ceiling Plate Extender. The ACP is not to be used in any other way than described above. The ACP is covered under SuperPole warranty (DC100). Specifications subject to change without notice.

## TECHNICAL DATA

See Figure A.

## DELIVERY CONTENTS - ACP

- |                          |                             |
|--------------------------|-----------------------------|
| 1. Angled Ceiling Plate  | 5. Flat Washer (x2)         |
| 2. Pivot Bracket         | 6. 1/4-20 x .50" Screw (x2) |
| 3. Top Plate Rubber (x2) | 7. Nylock Nut (x2)          |
| 4. Lock Washer (x2)      | 8. 5/32" Hex Key            |

## INSTALLATION INSTRUCTIONS

## FIGURE C.

1. Secure Angled Ceiling Plate to ceiling with appropriate wood screws (min #10 x 2.5" / 64mm).
2. Slide top end of pole onto pivoting bracket.
3. Align locking pin and secure with nylon washer and hitch pin.
4. Place base plate at the intended location.
5. With SuperPole vertically level, expand extension screw until it goes into hole of base plate.
6. Lock the angle of the ACP pivot bracket with supplied hardware (screw, lock washer, flat washer and nut) on both sides.
7. Continue with SuperPole instructions (DC100).

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