
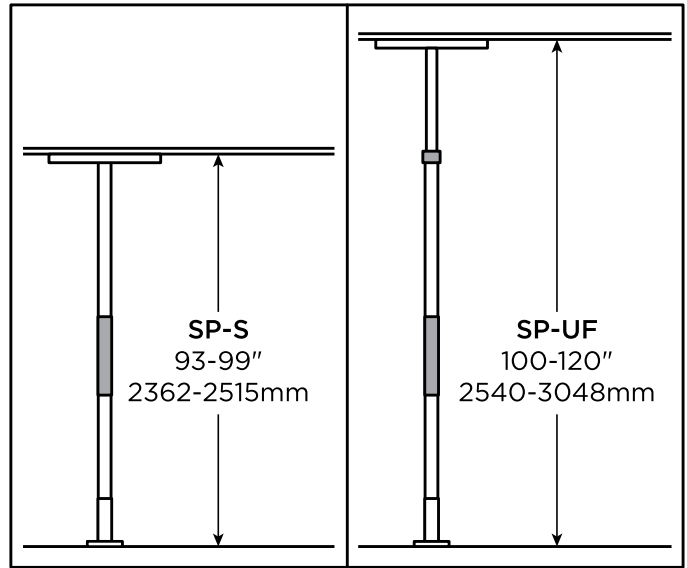
## Uni-Fit Extender™


SP-UF  
(DC105 - RevD)

[healthcraftproducts.com](http://healthcraftproducts.com)


# A







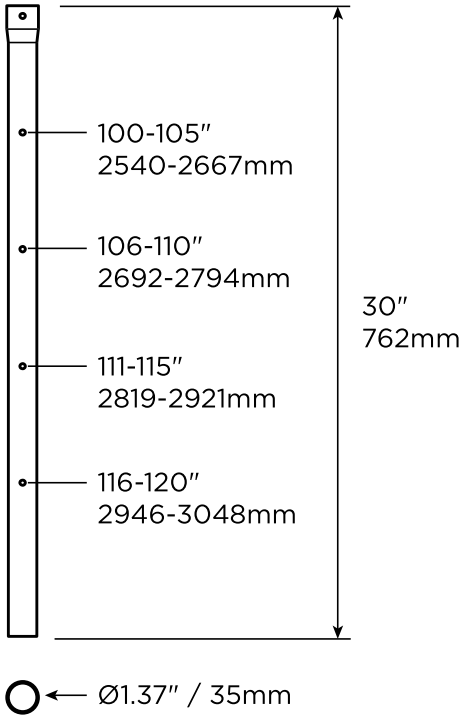
# B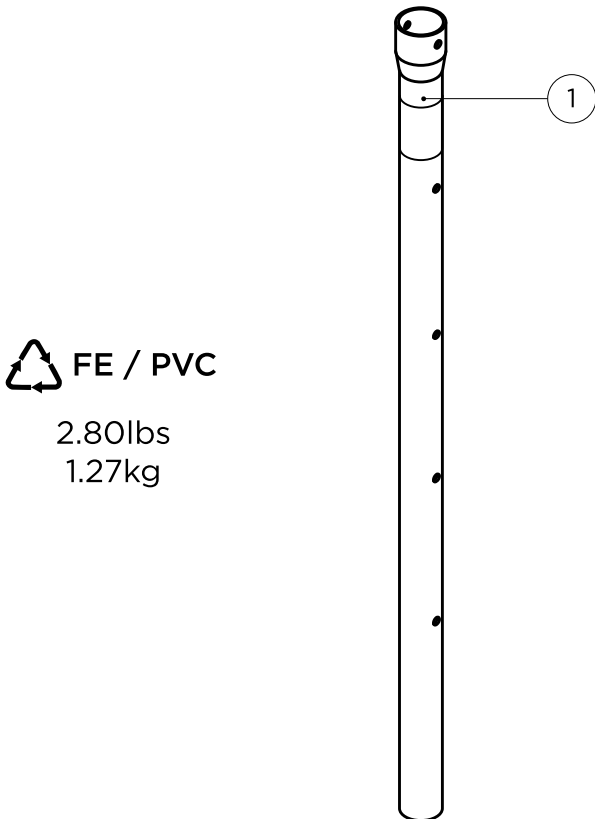

FE / PVC  
2.80lbs  
1.27kg
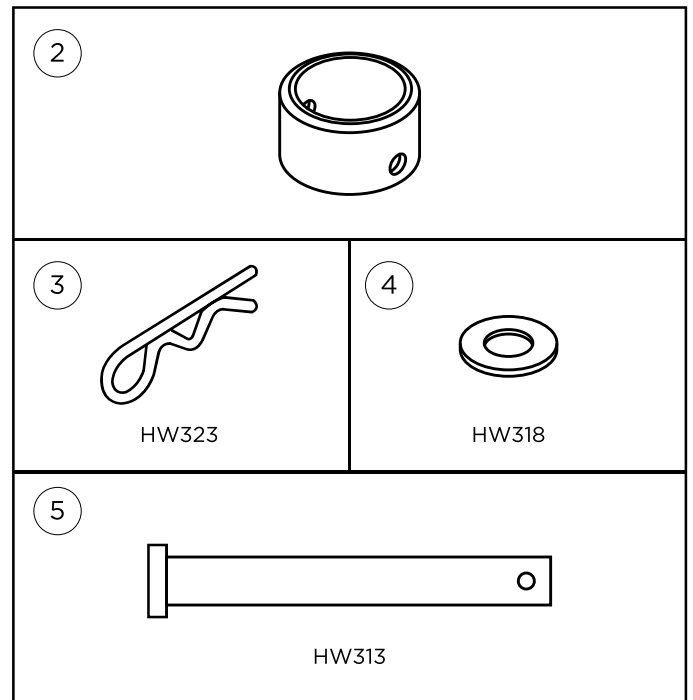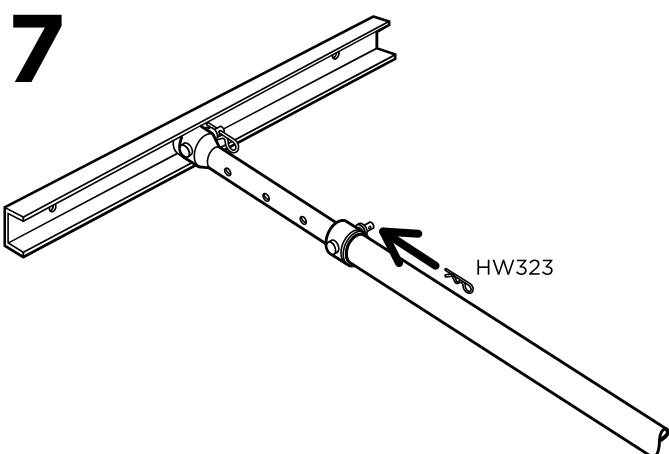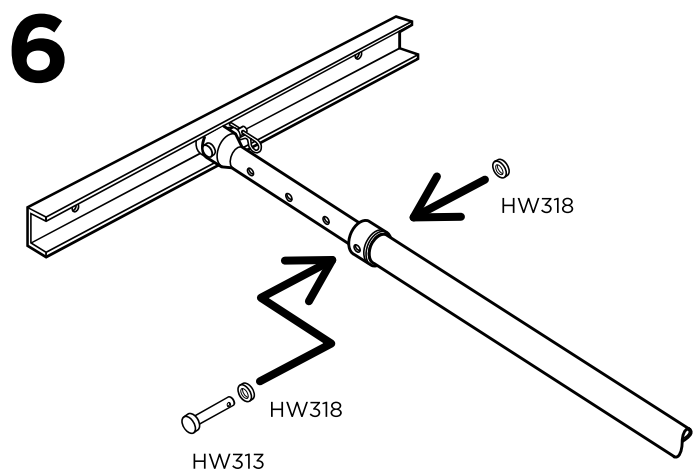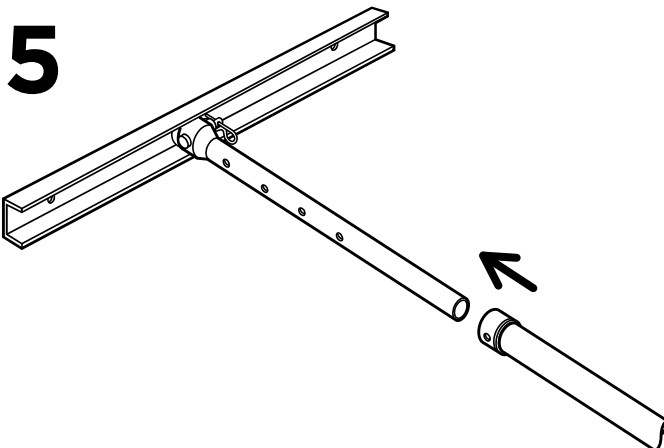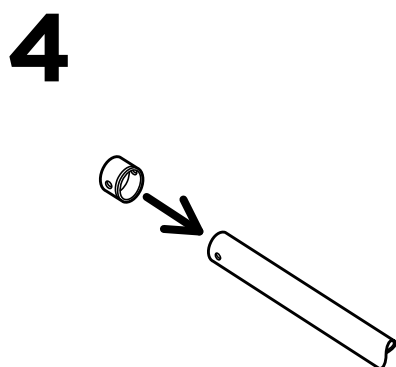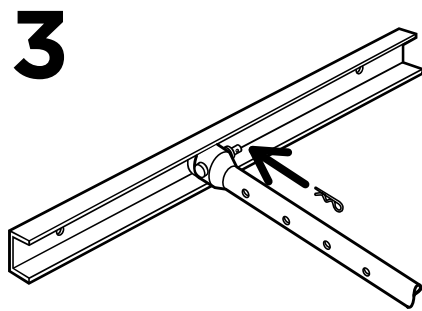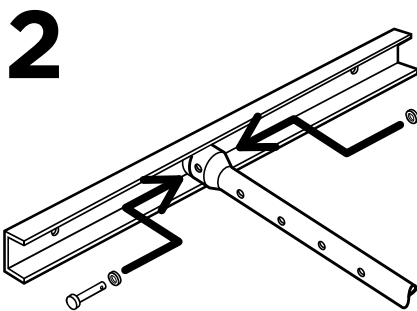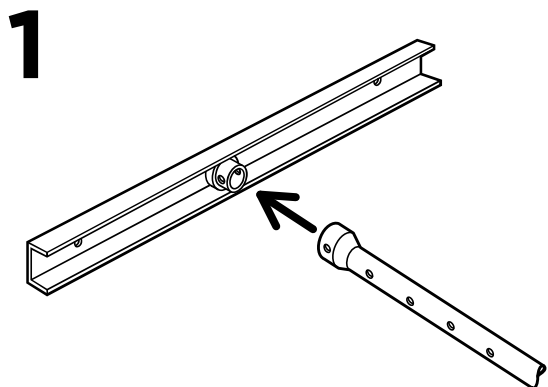
## OVERVIEW

Thank you for choosing the SuperPole System from HealthCraft. Please read and understand the instructions in this manual (and the SuperPole Manual DC100); keep manuals for future reference.

The Uni-Fit Extender is a high quality, commercial grade installation accessory to be used with the SuperPole. The Uni-Fit Extender is intended to increase the floor to ceiling range of the SuperPole from 93-99" to 100-120" (236-251cm to 254-305cm). The Uni-Fit Extender is a metal tube that attaches between the top plate and the pole assembly of the SuperPole. The Uni-Fit Extender features a series of holes that can be selected depending on the desired floor to ceiling height range. The Uni-Fit Extender is only to be used with HealthCraft's SuperPole System. The Uni-Fit Extender is not to be used with the SuperTrapeze, SuperPole Bariatric or SuperPole Ultra. The Uni-Fit Extender is not to be used in any other way than described above. Uni-Fit Extender covered under SuperPole warranty (DC100).

Specifications subject to change without notice.

## TECHNICAL DATA

See Figure A.

## DELIVERY CONTENTS - UNI-FIT

FIGURE B.

UNI-FIT ASSEMBLY	3. Cotter Pin (x1)
1. Uni-Fit Mast with caution label	4. Nylon Washer (x1)
2. Plastic Bushing	5. 2" Clevis Pin (x1)

## INSTALLATION INSTRUCTIONS

FIGURE C.

1. Slide Uni-Fit Extender onto collar of top plate.
2. Align holes and install 1.5" clevis pin with washer on both sides (hardware from SuperPole top plate).
3. Install cotter pin into clevis pin.
4. Slide plastic bushing onto top of pole and align holes.
5. Slide Uni-Fit Extender into top end of SuperPole.
6. Align holes and install 2" clevis pin with washer on both sides.
7. Install cotter pin into clevis pin.
8. Continue with SuperPole instructions (DC100).

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