
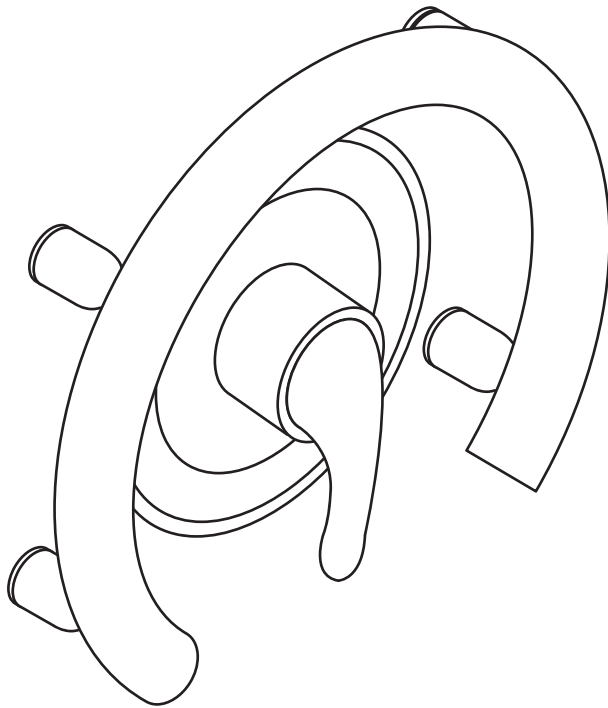
INVISIA


Accent Ring


INV-ACR
(DC151 - RevE)

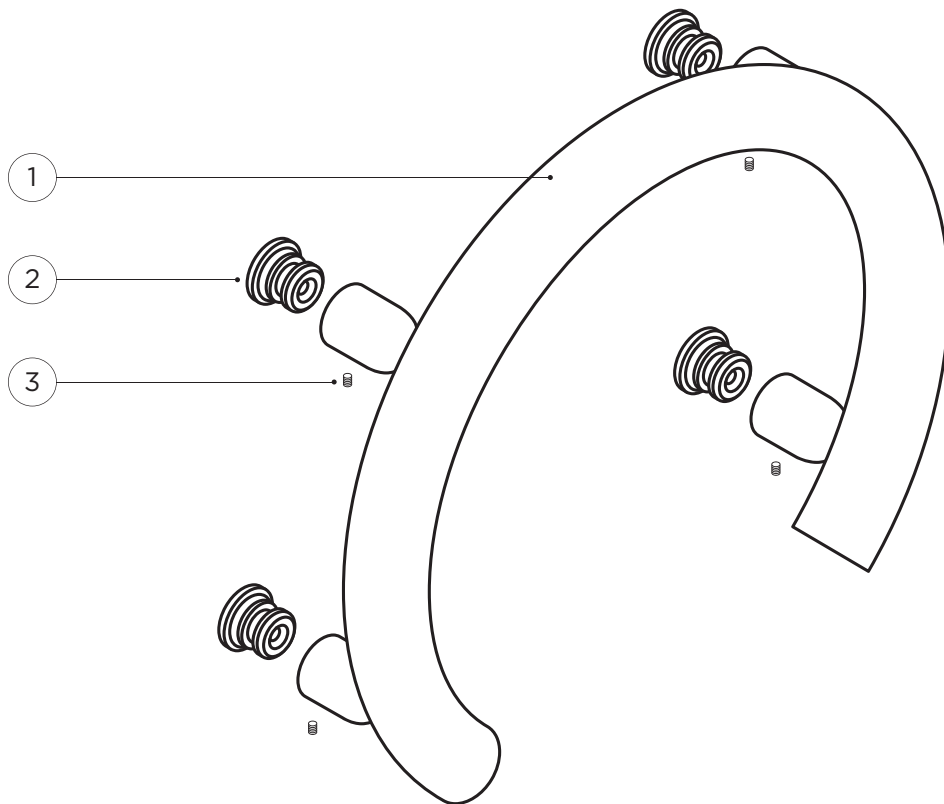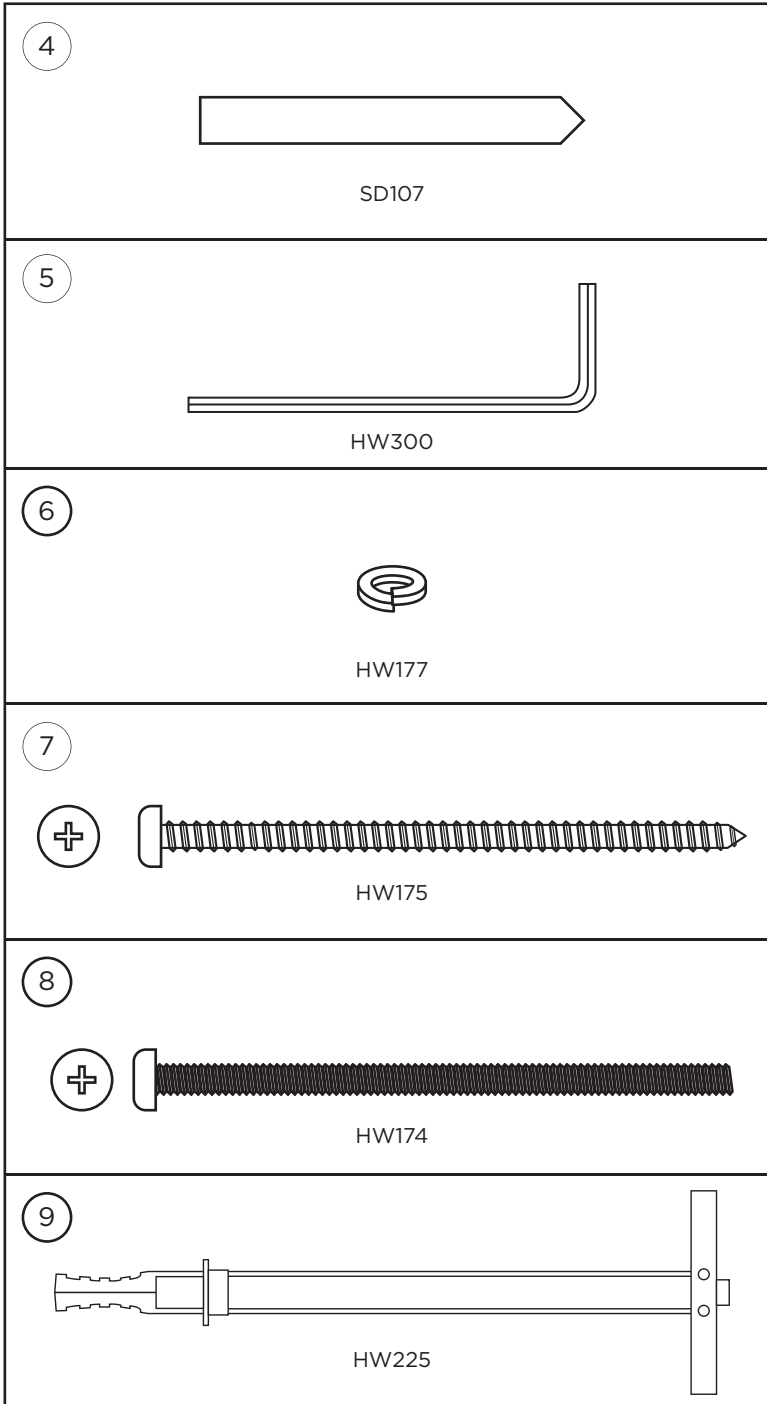invisiacollection.com


1

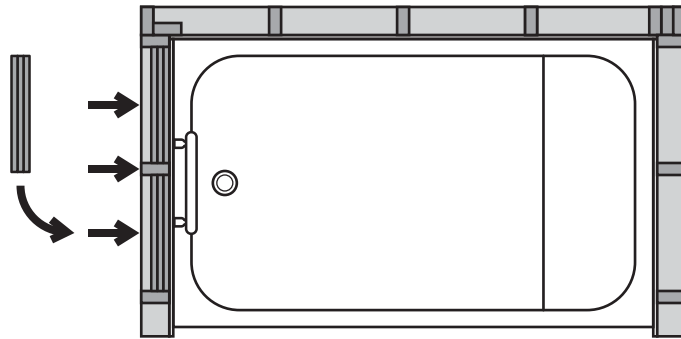2

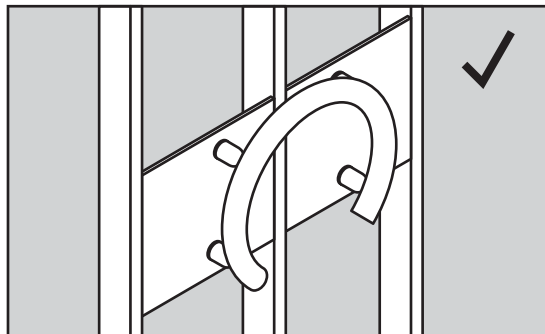MAX
500lbs / 227kg


6 x4

7 x4

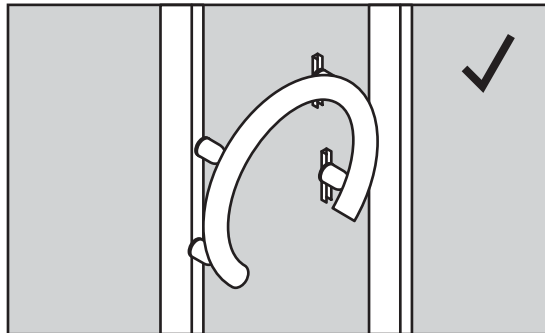MAX
350lbs / 159kg

6 x4

7 x2

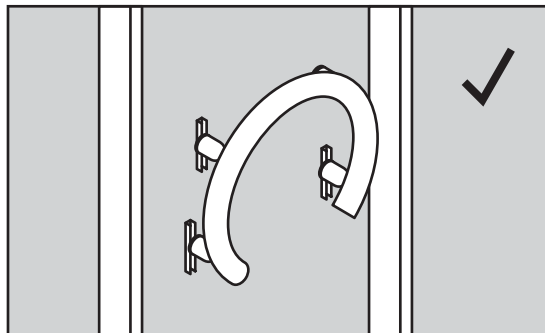
6 x1

8 x2

9 x2

MAX
200lbs / 91kg




6 x4


8 x4

9 x4

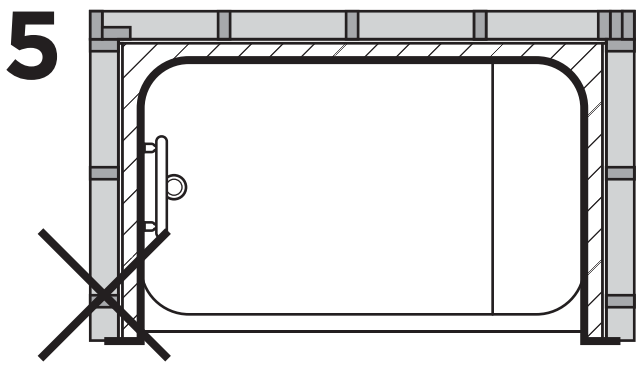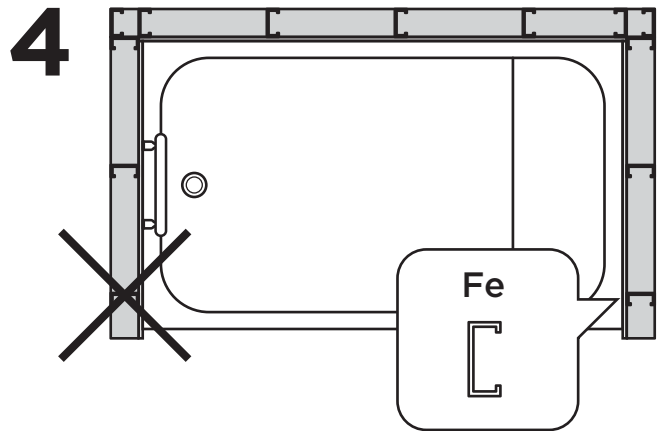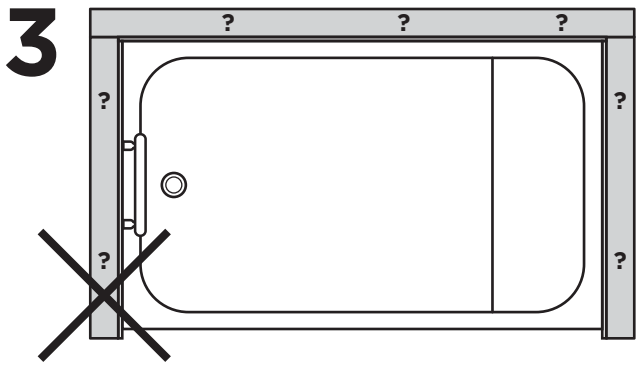
B


C

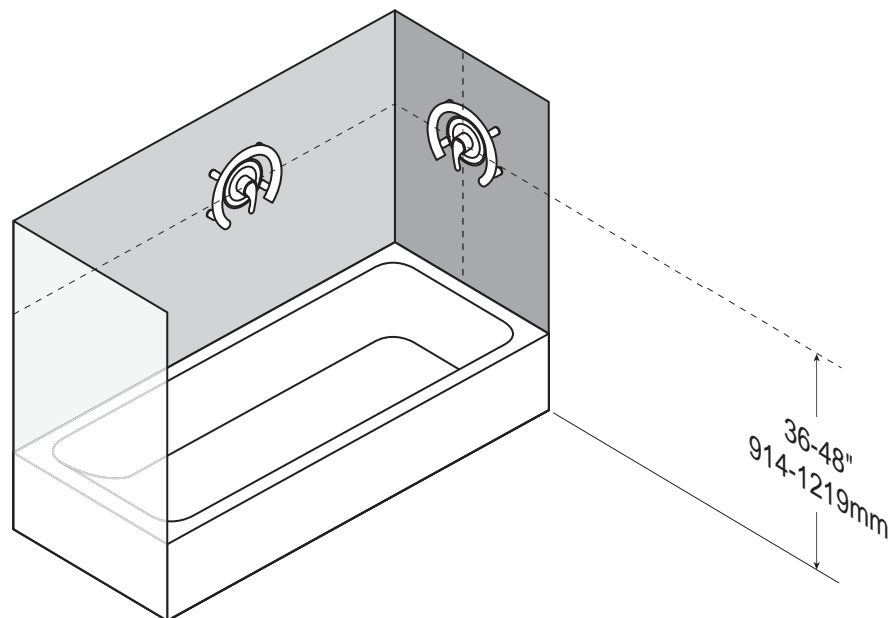
1

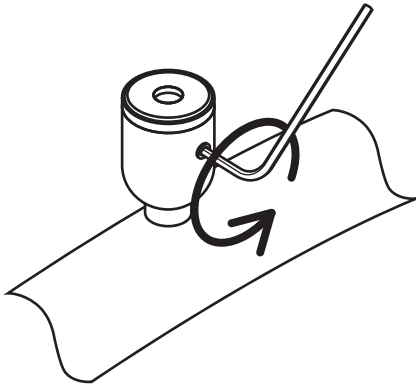
2

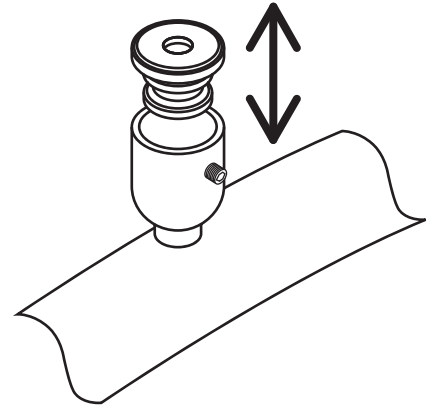
3

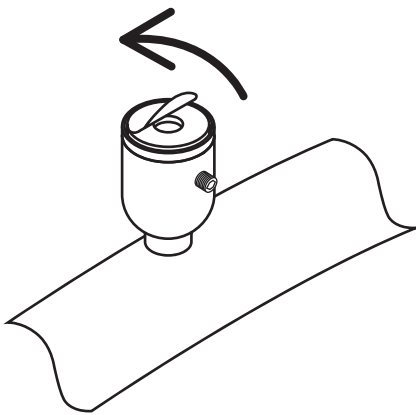
4

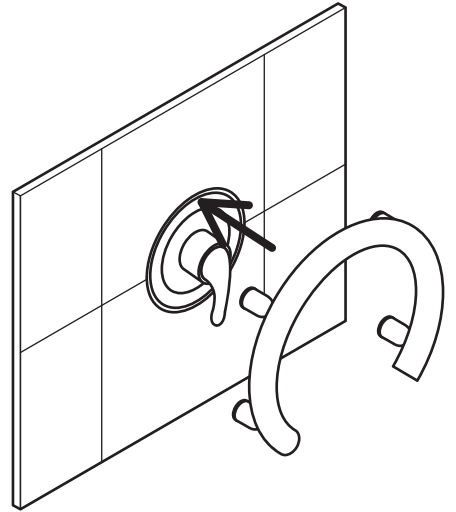
5

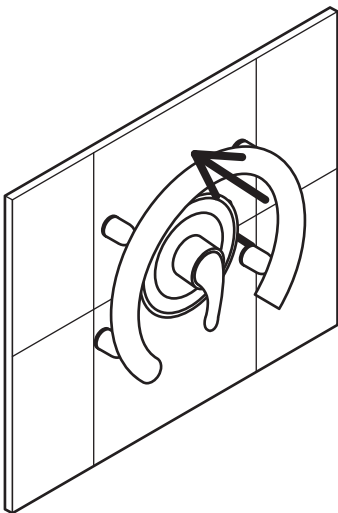
6

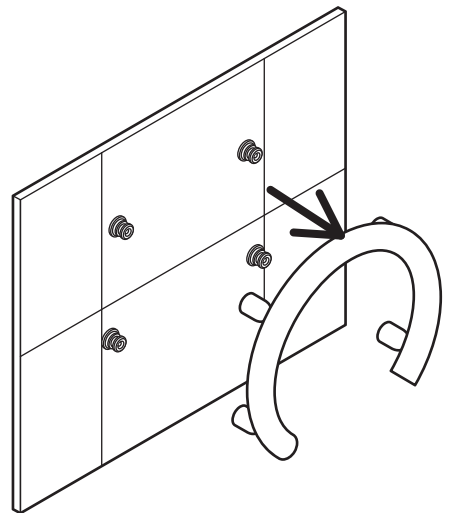
7

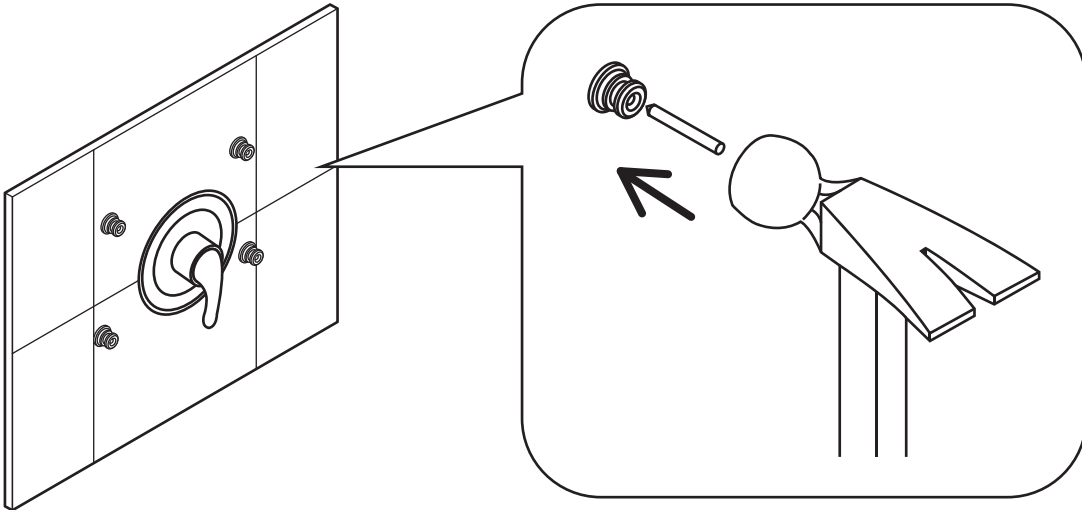
8

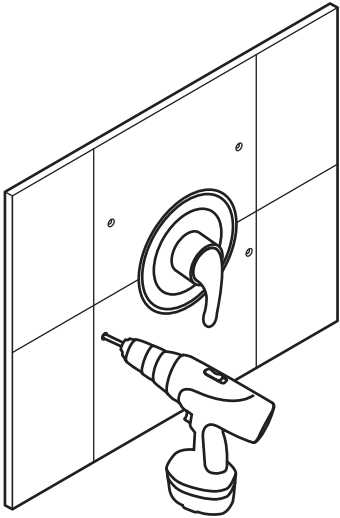
9




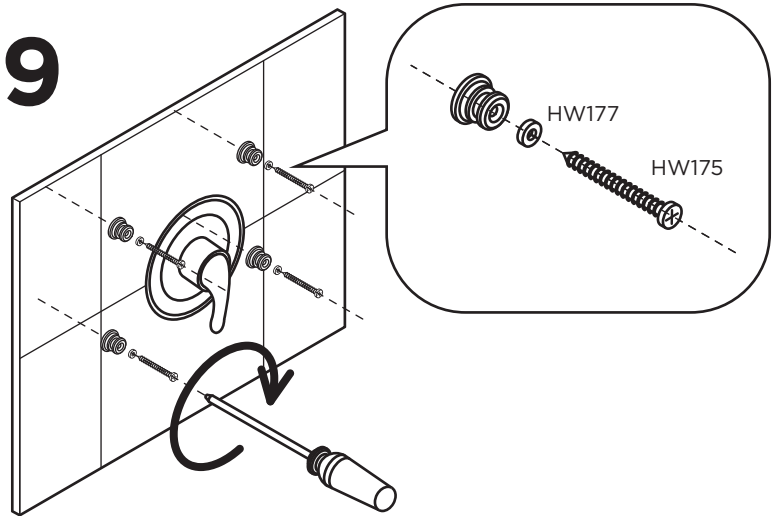
10

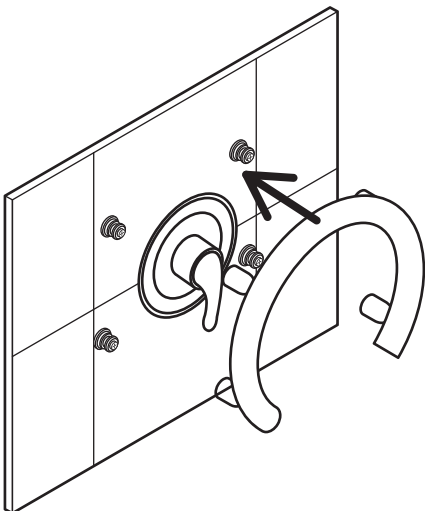11

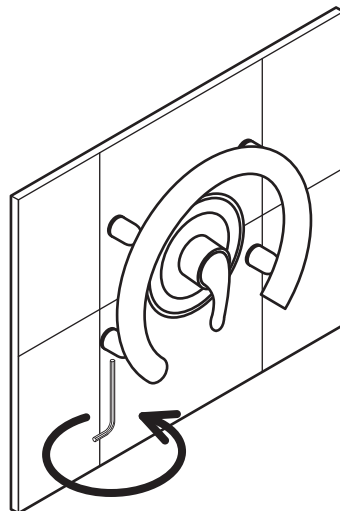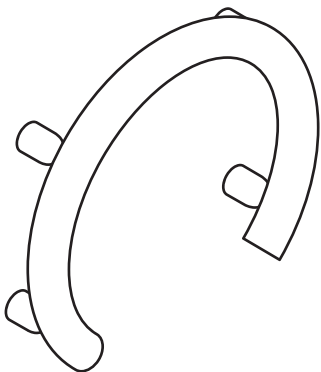
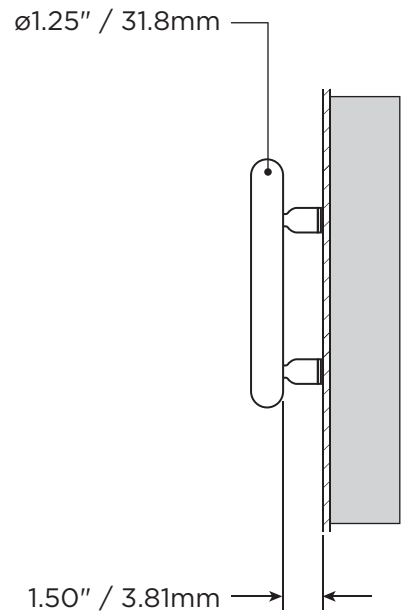
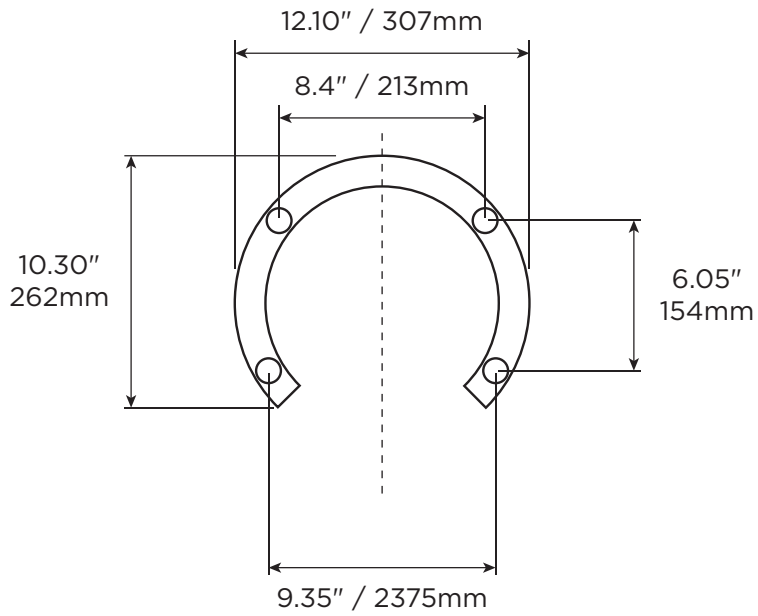
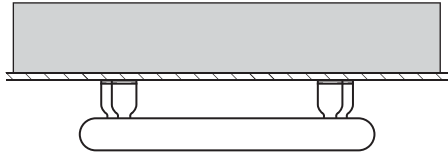
 SS
 2.05lbs
 0.93kg

 SS
 0.05lbs
 0.02kg

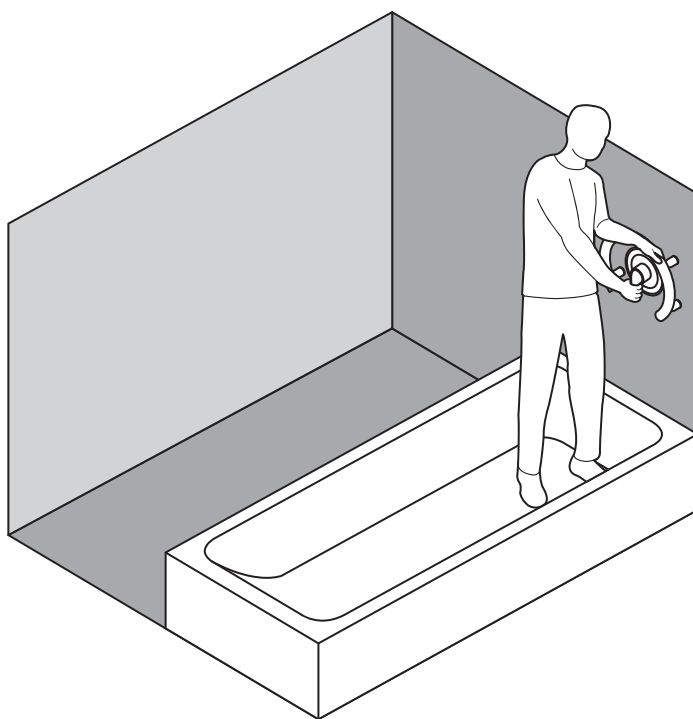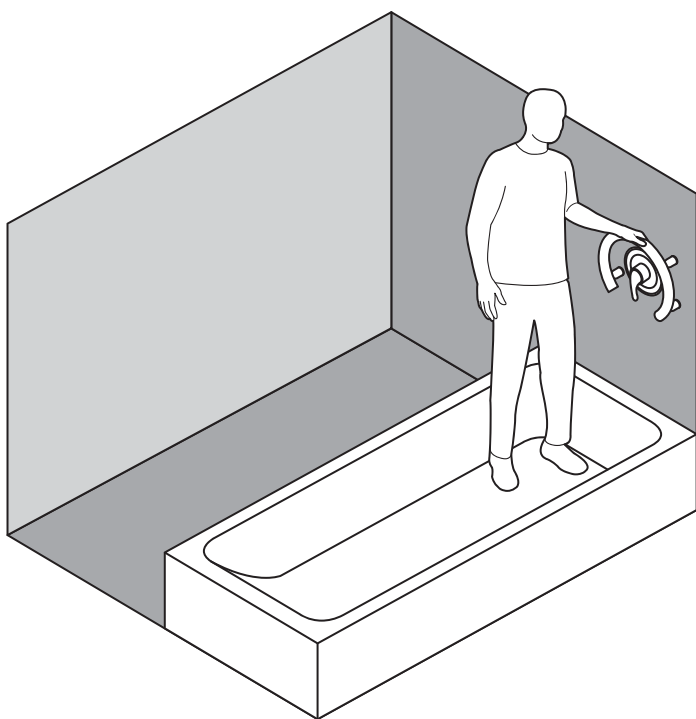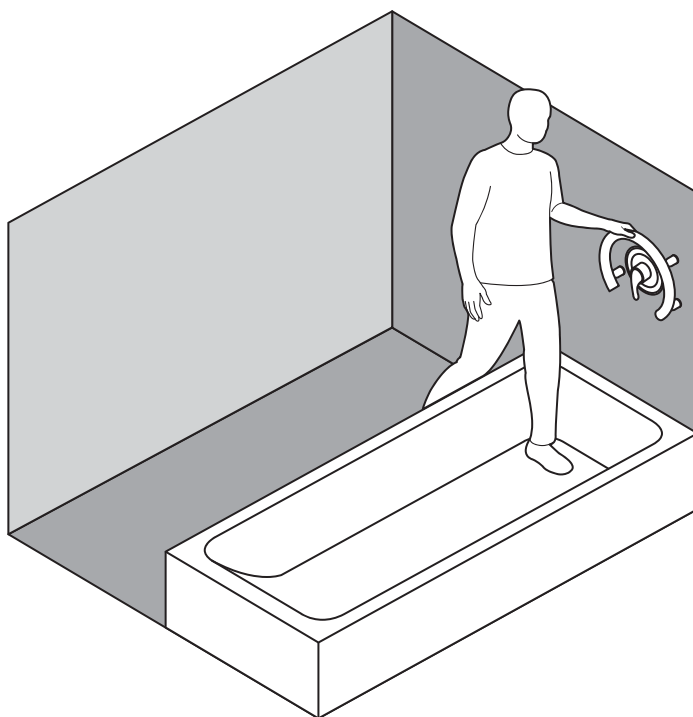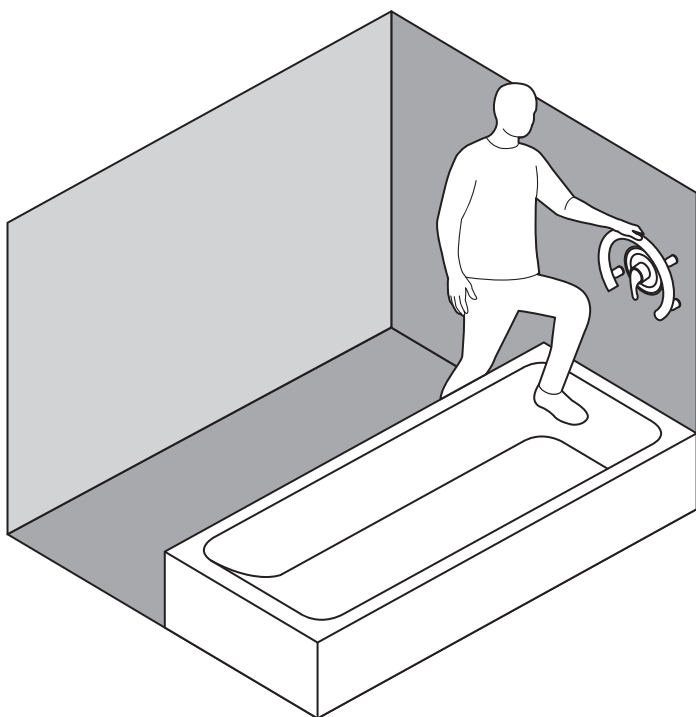MAX
 500lbs
 227kg

English

OVERVIEW

Thank you for choosing the Invisia Accent Ring. Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your Invisia Accent Ring is properly assembled, installed, and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your Invisia Accent Ring installed by a qualified contractor.

The Invisia Accent Ring is a circular grab bar designed to disguise safety in a bathroom accessory. The Invisia Accent Ring is intended for moderate vertical and horizontal load bearing to provide sitting and standing support for people with reduced mobility. The Accent Ring is to be installed with appropriate fasteners to a structural wall. The maximum user weight for this product is 500lbs / 227kgs. Note: Product is not intended to support full body weight. The Invisia Accent Ring is not to be used in any other way than described above.

LIMITED LIFETIME WARRANTY

Products are covered by a limited lifetime warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, or the use of corrosive or abrasive cleaning products.

Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described hereon; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon. Specifications subject to change without notice.

HealthCraft Products Inc.
2790 Fenton Road
Ottawa, Canada
K1T 3T7

Gate 88
Kanalgatan 45B
Skellefteå, Sweden
93123

DELIVERY CONTENTS / ACCENT RING

FIGURE A.

- | | |
|-------------------------------|----------------------------------|
| 1. Accent Ring | 6. Lock Washer, S/S (x4) |
| 2. Wall Stem Assembly (x4) | 7. #10 x 3" Wood Screw, S/S (x4) |
| 3. #10-32 Set Screw, S/S (x4) | 8. 3/16" Bolt, S/S (x4) |
| 4. Center Punch | 9. Hollow Wall Anchor (x4) |
| 5. 3/32" Hex Key | |

INSTALLATION WARNINGS

FIGURE B.

1. Rail must be installed to walls that are structurally sound. If desired installation location does not have adequate structure, add wood blocking.
2. For different wall constructions, use appropriate fastener combination (Note: reduced weight capacity when not full structure).
3. CAUTION - Do not install if structure behind walls is unknown.
4. CAUTION - Do not install into metal studs.
5. CAUTION - Do not install in fiberglass / acrylic shower enclosures with insufficient stiffness and strength.
6. CAUTION - Rail and bathtub are slippery when wet.

SUGGESTED ACCENT RING LOCATIONS

FIGURE C.

To determine the optimal mounting location, we recommend holding the rail in place and walking through the motions of a transfer (sitting, standing, reaching, etc.). Recommended rail height 36-48" / 914-1219mm, or centered around shower control valve.

INSTALLATION

FIGURE D.

1. Back off set screws from rail sockets.
2. Allow wall stems to freely move in and out of rail sockets.
3. Peel off label on back of wall stems to expose sticker.
4. With wall stems inside rail sockets, push the rail against the wall.
5. Firmly press the rail so stickers on back of wall stems contact wall.
6. Remove rail carefully so wall stems stay on the wall.
7. Using wall stems as locating guides, mark the tile with center punch.
8. Drill pilot holes for fasteners.
9. Fasten the wall stems to the wall using appropriate fasteners and waterproofing.
10. Place rail back onto the fastened wall stems.
11. Secure rail to wall stems by tightening set screws. Inspect hardware regularly, and tighten if required.

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