
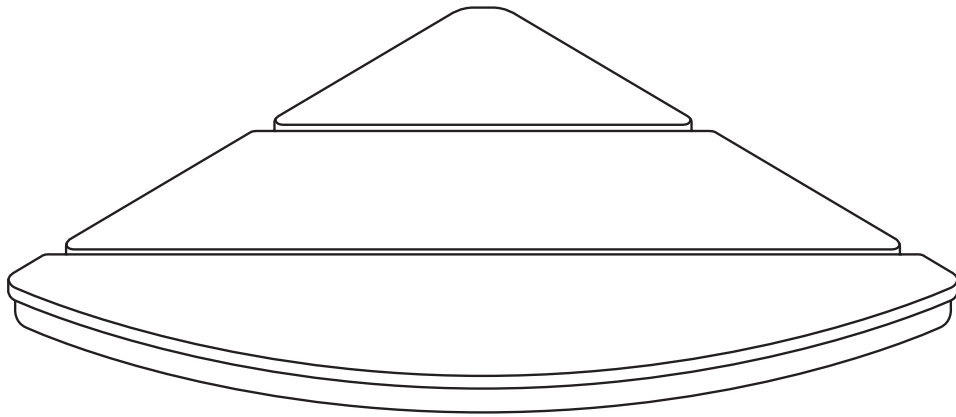
# INVISIA






## **Corner Seat**

INV-WSCS  
(DC143)

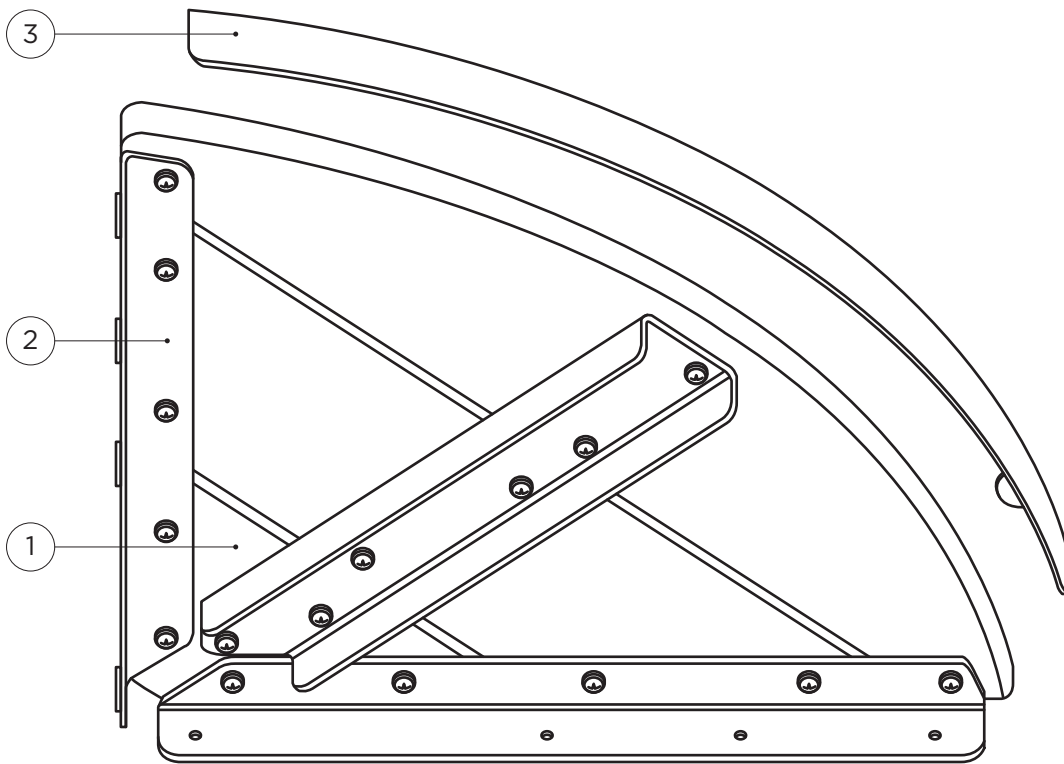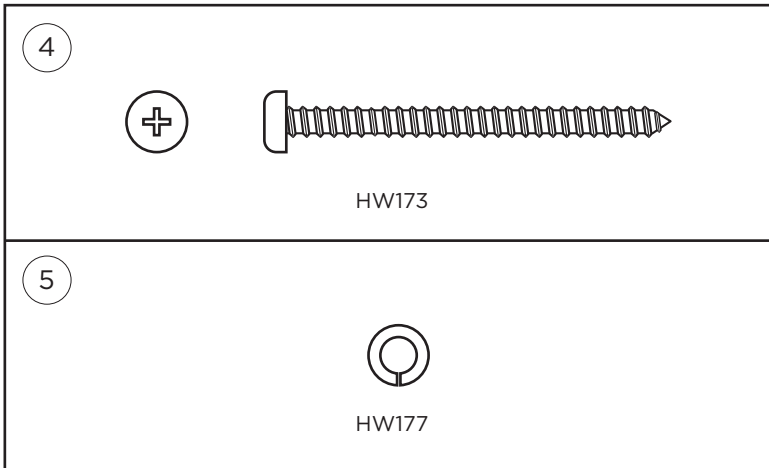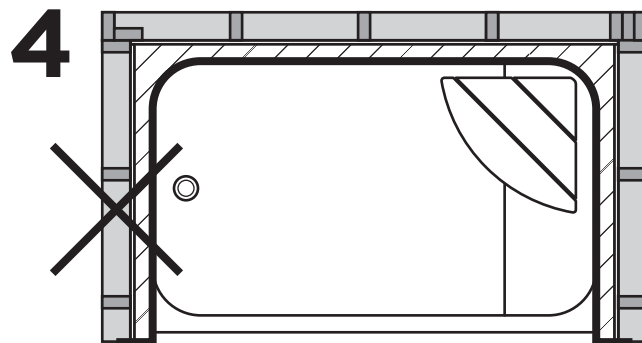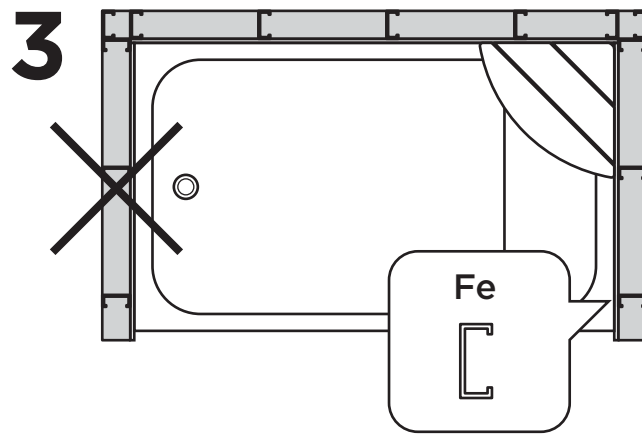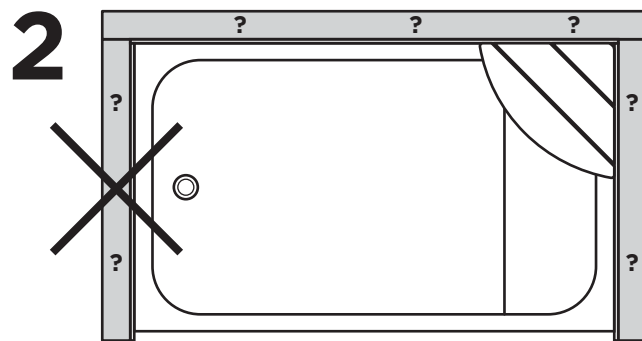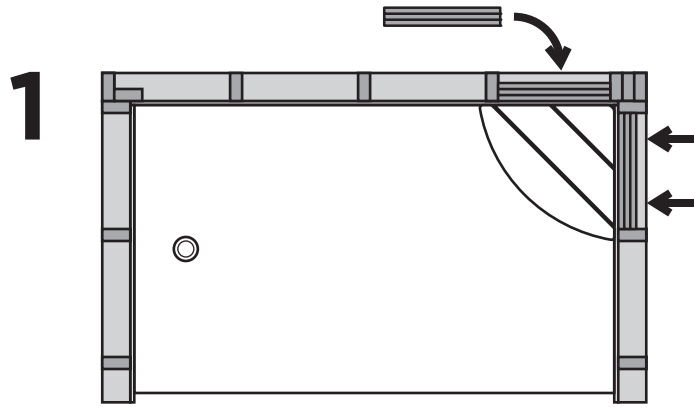[invisiacollection.com](http://invisiacollection.com)



B

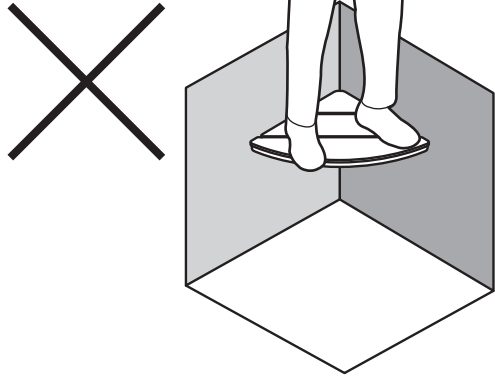5


6

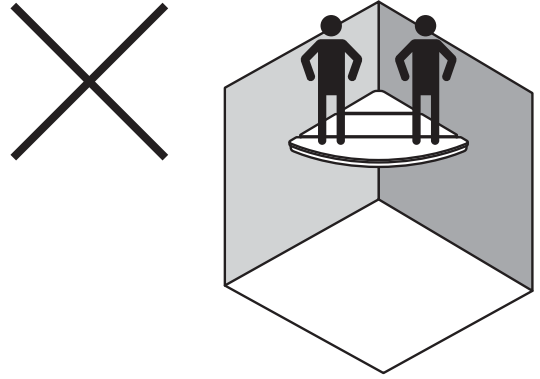
7

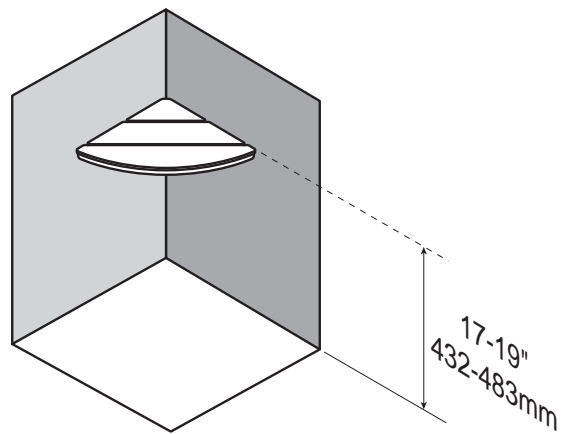
C


1

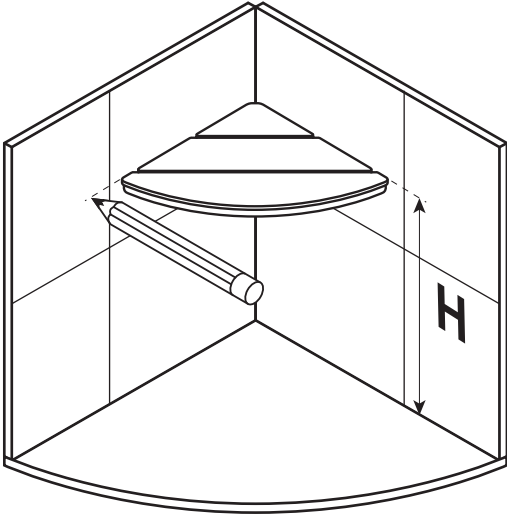
2

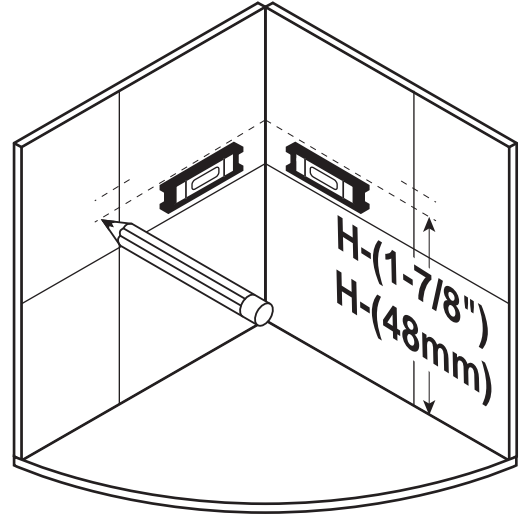
3

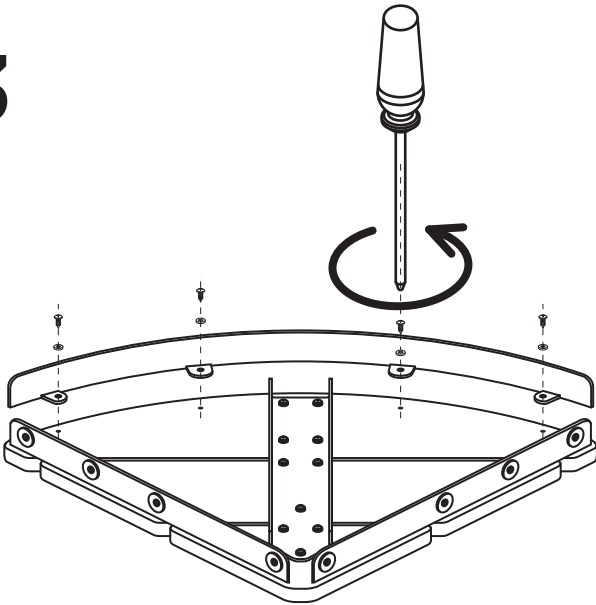
4

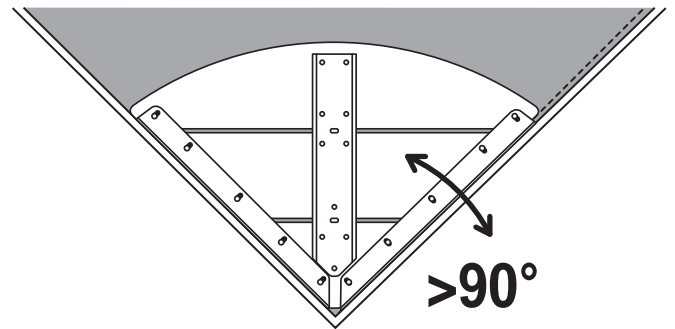
5

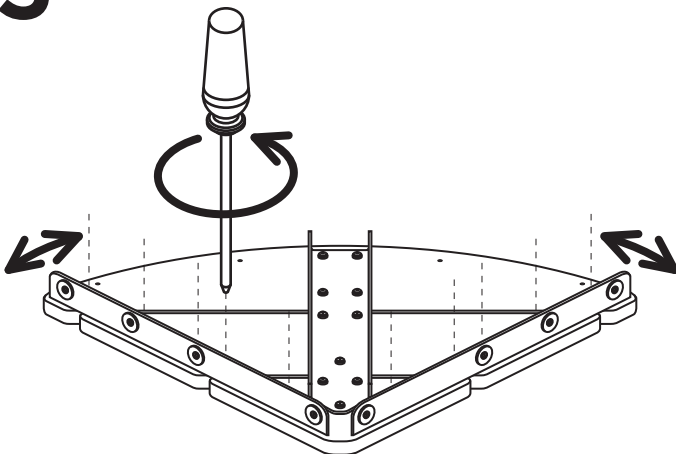
6

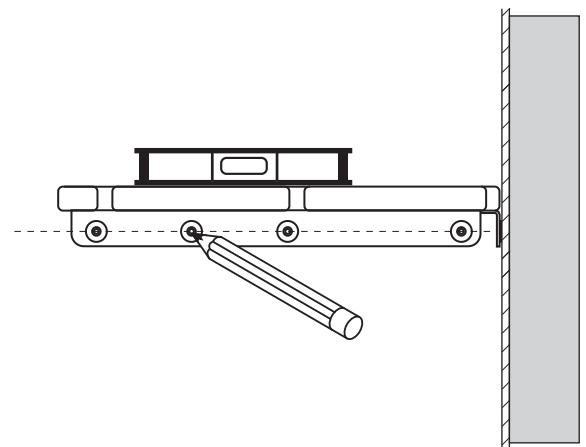
7

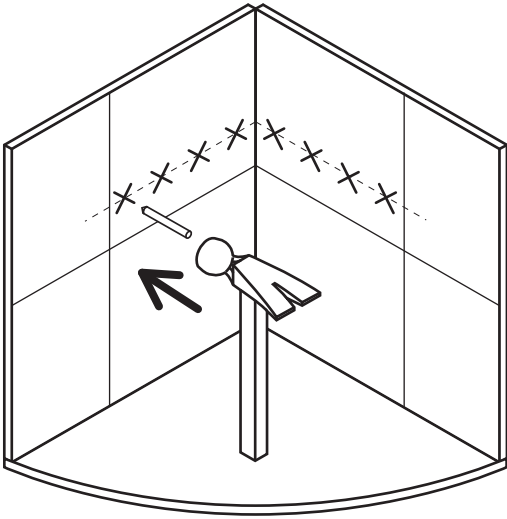
8

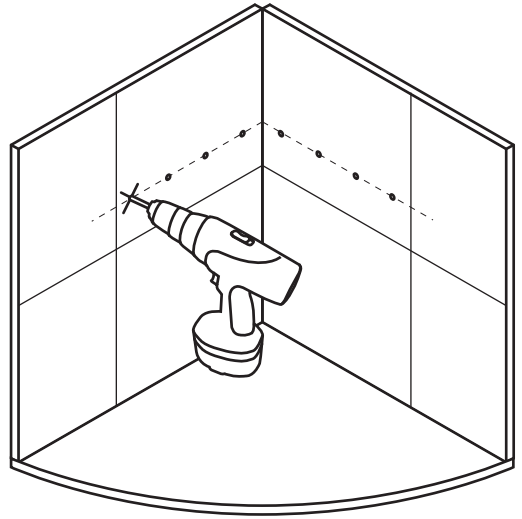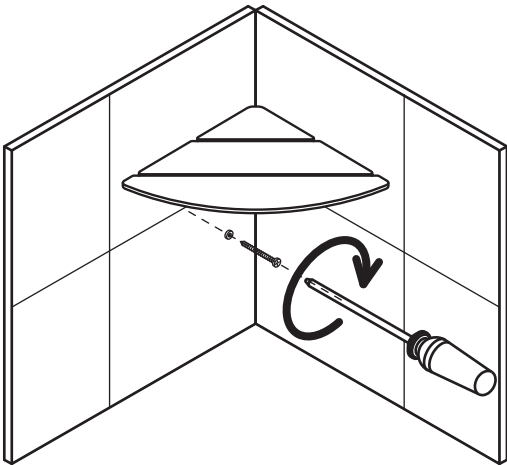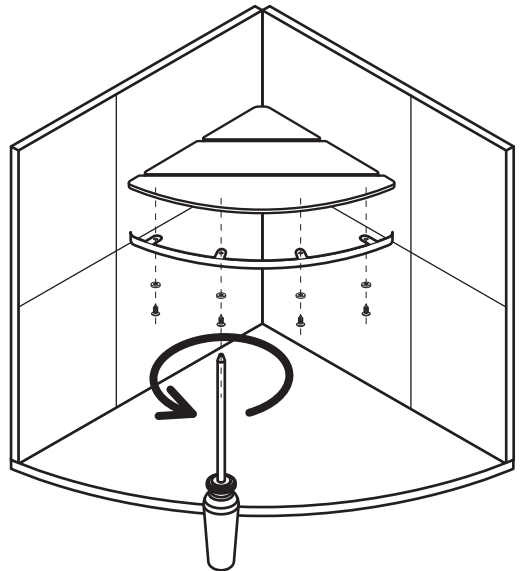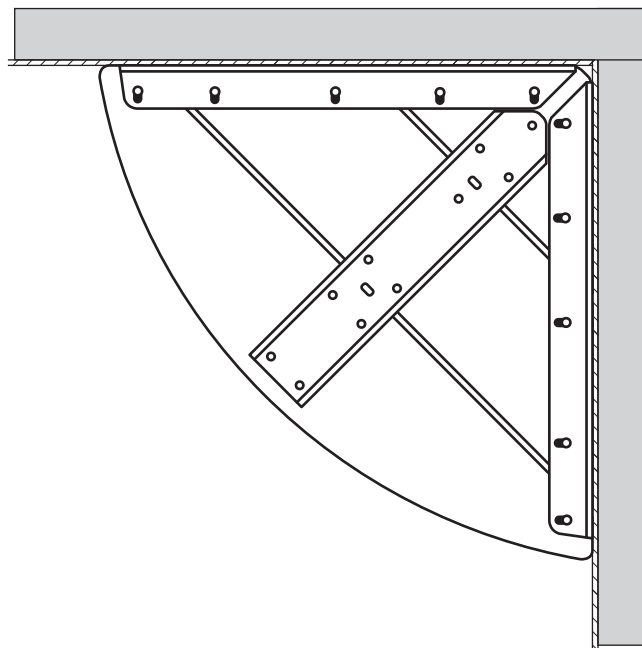
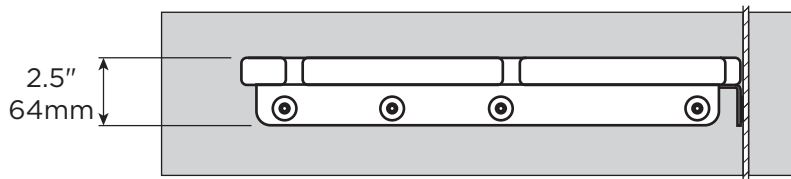
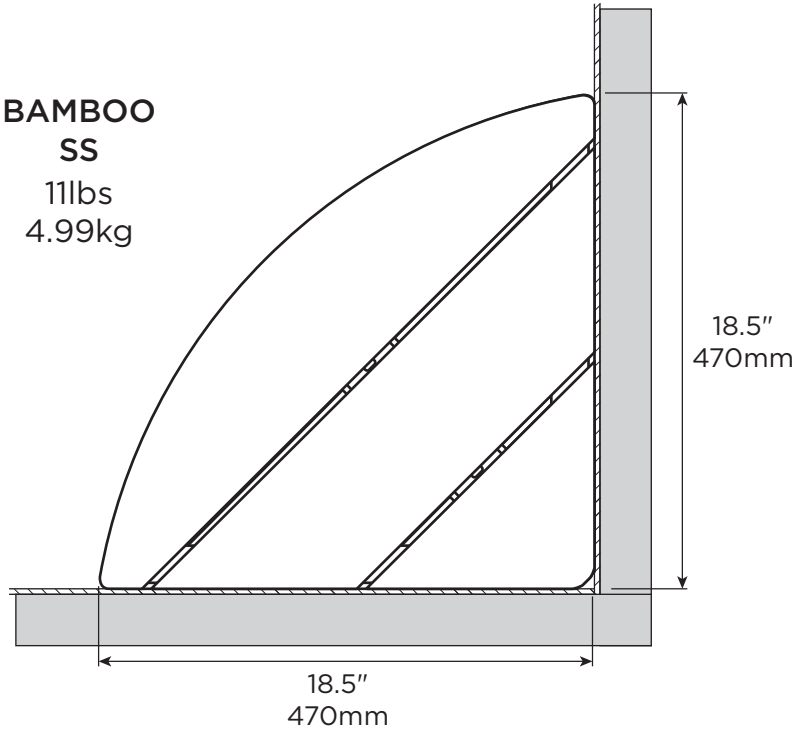
9



10

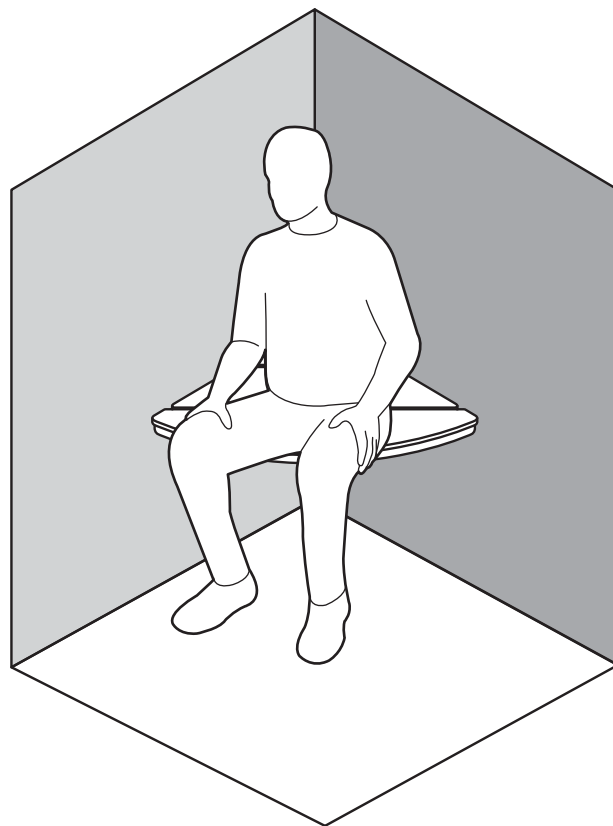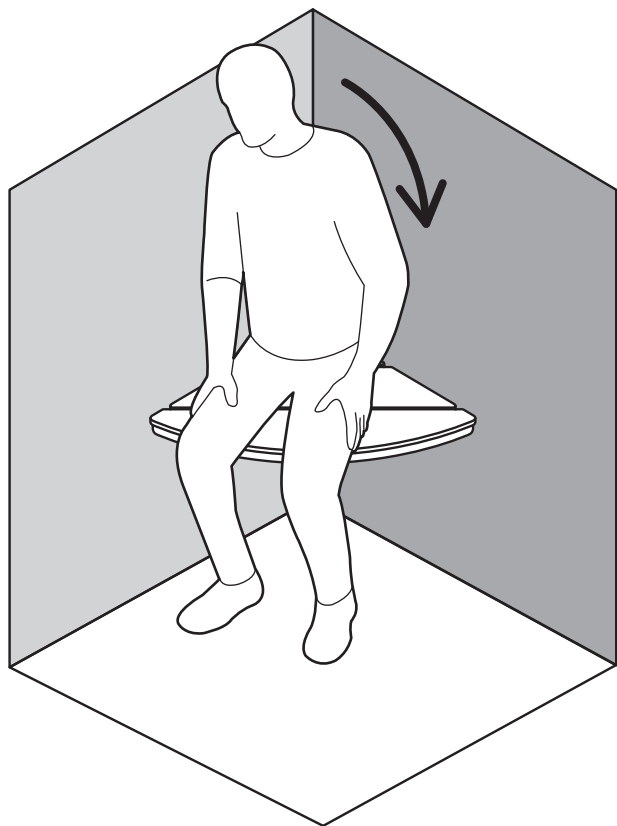 **BAMBOO**  
SS  
11lbs  
4.99kg

# English

## OVERVIEW

Thank you for choosing the Invisia Corner Seat. Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your Invisia Corner Seat is properly assembled, installed, and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your Invisia Corner Seat installed by a qualified contractor.

The Invisia Corner Seat is a high quality, commercial grade shower seat. The Invisia Corner Seat is intended to provide shower seating support for people with reduced mobility. The Invisia Corner Seat is installed with appropriate fasteners to a structural wall. The maximum user weight for this product is 500lbs / 227kgs. Note: Product is not intended to support full body weight (i.e. do not stand on the seat). The Invisia Corner Seat is not to be used in any other way than described above.

## LIMITED LIFETIME WARRANTY

Products are covered by a limited lifetime warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, or the use of corrosive or abrasive cleaning products.

Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described herein; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon. Specifications subject to change without notice.

HealthCraft Products Inc.  
2790 Fenton Road  
Ottawa, Canada  
K1T 3T7

Gate 88  
Kanalgatan 45B  
Skellefteå, Sweden  
93123

## DELIVERY CONTENTS / CORNER SEAT

FIGURE A.

1. Corner Seat
2. Wall Bracket (x2)
3. Metal Trim
4. #10 x 2" Wood Screw, S/S (x8)
5. Lock Washer, S/S (x8)

## INSTALLATION WARNINGS

FIGURE B.

1. Corner Seat must be installed to walls that are structurally sound. If desired installation location does not have adequate structure, add wood blocking.
2. CAUTION - Do not install if structure behind walls is unknown.
3. CAUTION - Do not install into metal studs.
4. CAUTION - Do not install in fiberglass / acrylic shower enclosures with insufficient stiffness and strength.
5. CAUTION - Do not use seat for purposes other than sitting.
6. CAUTION - Do not more seat more than 1 person at a time.
7. CAUTION - Seat and bathtub are slippery when wet.

## SUGGESTED CORNER SEAT LOCATIONS

FIGURE C.

To determine the optimal mounting location, we recommend consulting with an appropriate healthcare professional. ADA recommend seat height from finished floor 17-19" / 432-483mm.

## INSTALLATION

FIGURE D.

1. Mark off desired location for top surface of seat.
2. Wall bracket holes should be 1-7/8" / 48mm lower than the top surface of the seat. Use level to mark centerline.
3. Untighten screws to remove metal trim. This will allow better access to the mounting holes of the wall brackets.
4. The wall brackets are designed to be outwardly adjusted to accommodate corners that are slightly greater than 90°.
5. To ensure both wall brackets make full contact with the walls, untighten screws and shift wall brackets accordingly.
6. Position bracket holes over the centerline and mark locations.
7. Notch the tile with center punch.
8. Drill pilot holes for fasteners.
9. Fasten the Corner Seat to the wall using lock washers and appropriate fasteners and waterproofing. Inspect hardware regularly, and tighten if required.
10. Re-install metal trim onto the bottom of the seat.

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