


INVISIA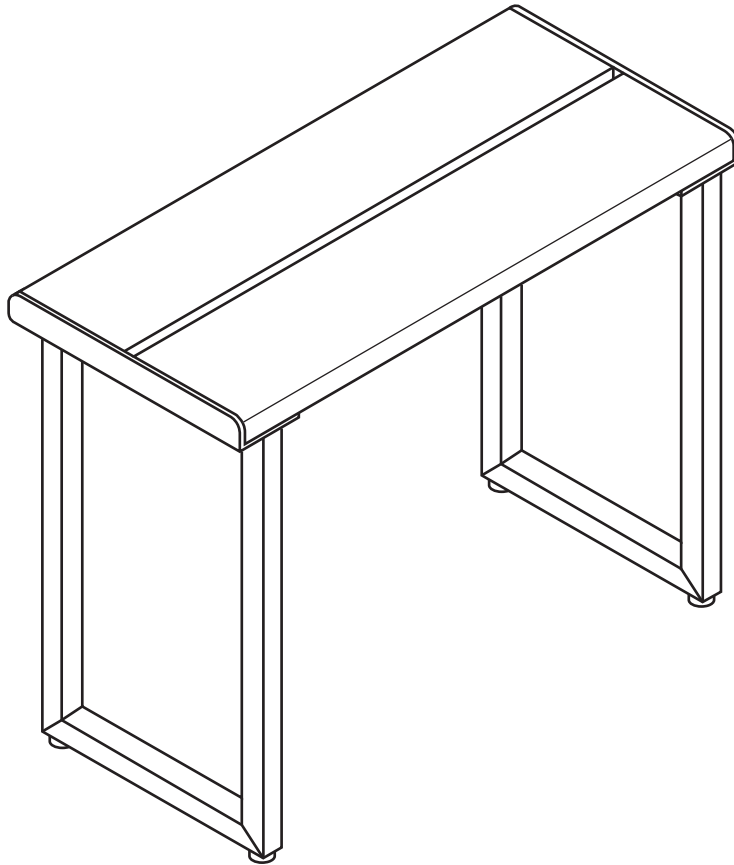


Shower Bench

INV-WSSB
(DC146)



invisiacollection.com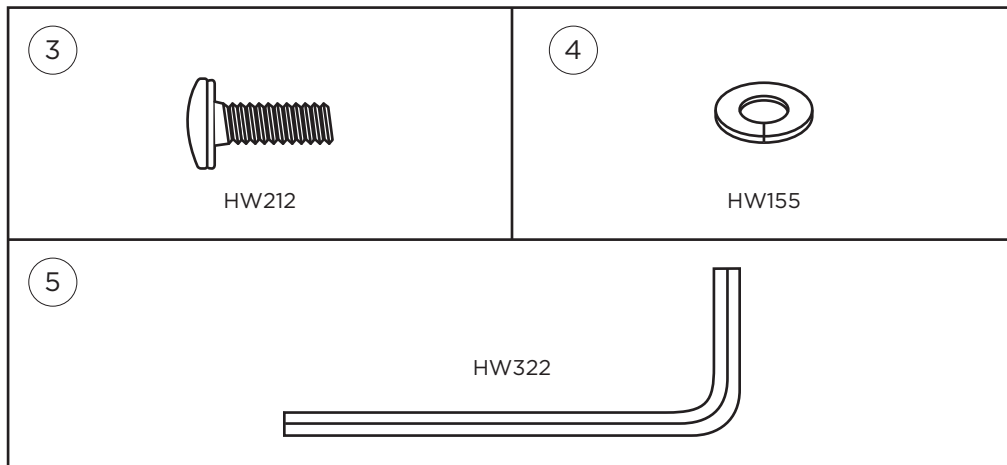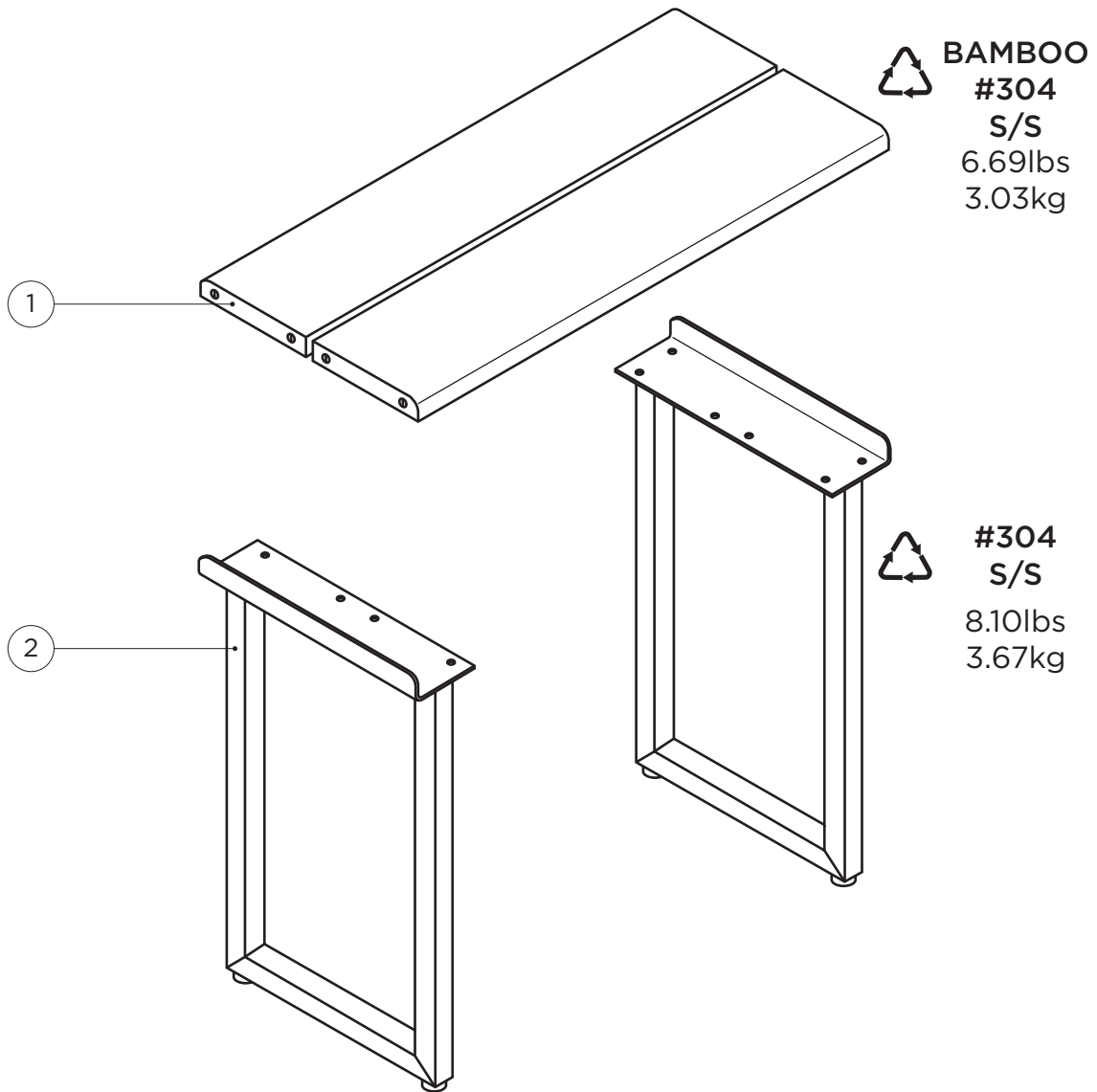


1

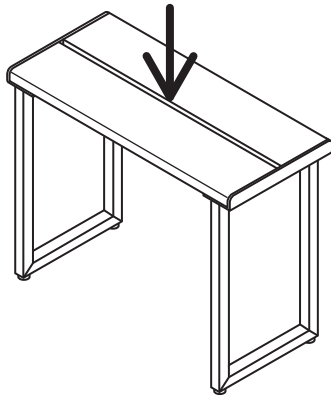
2

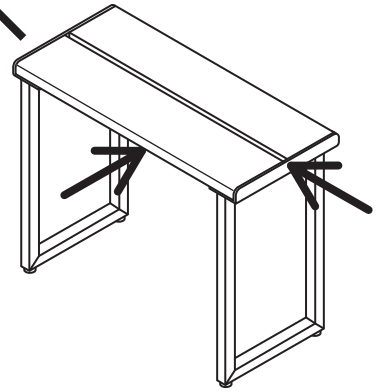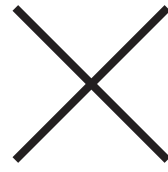
3

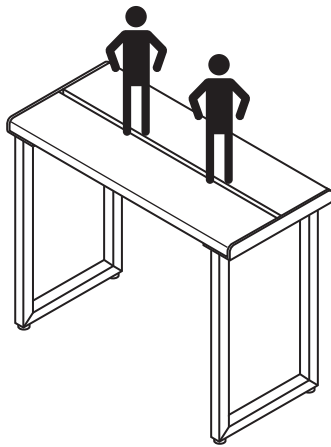4

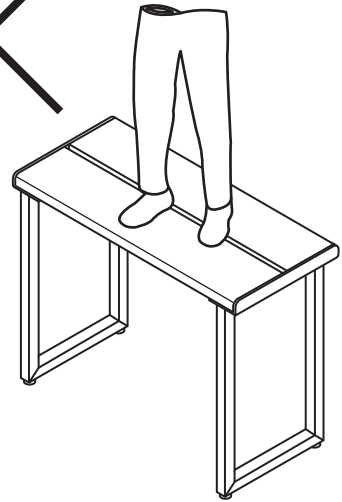
5

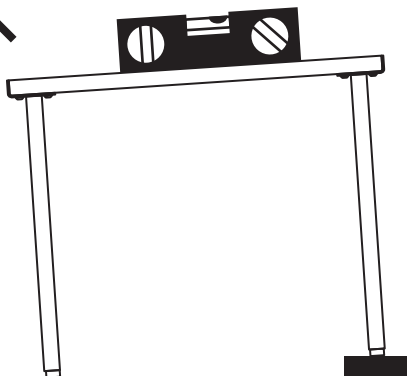
6


1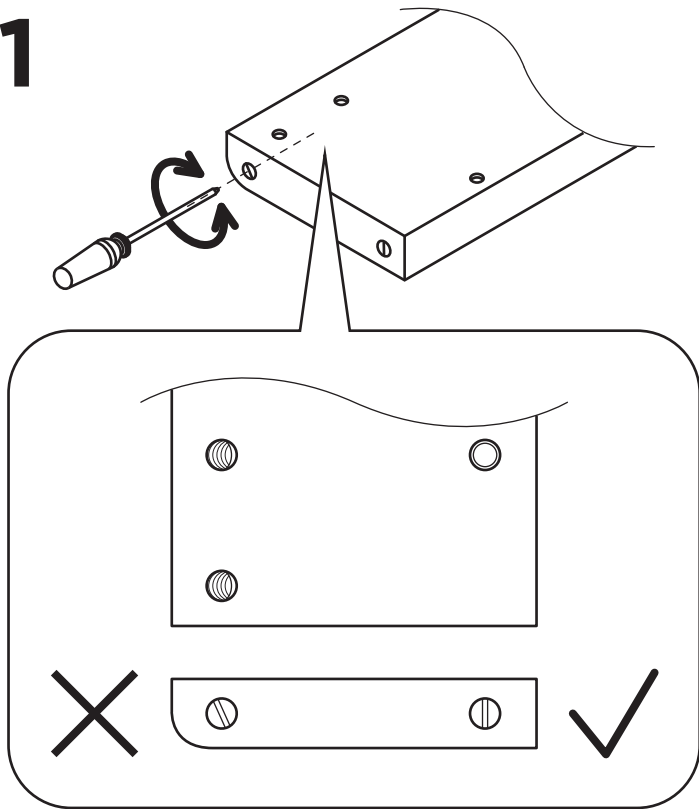


2

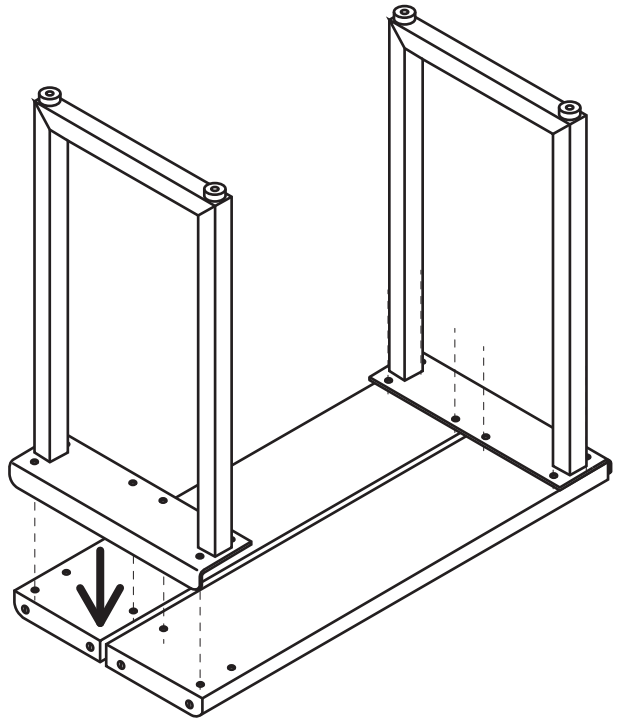
3


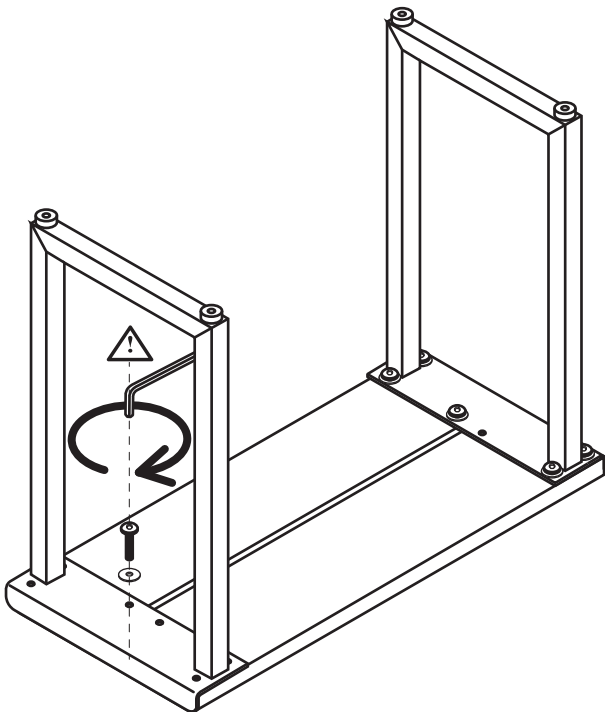
4


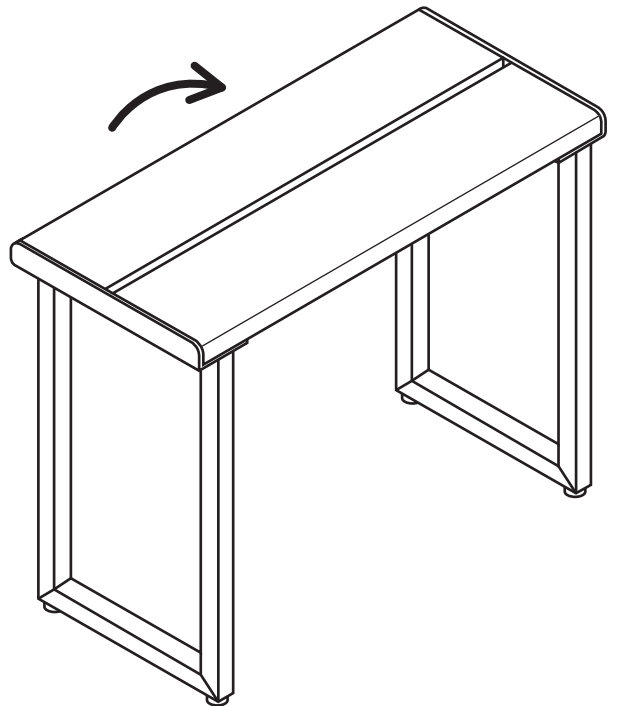
D

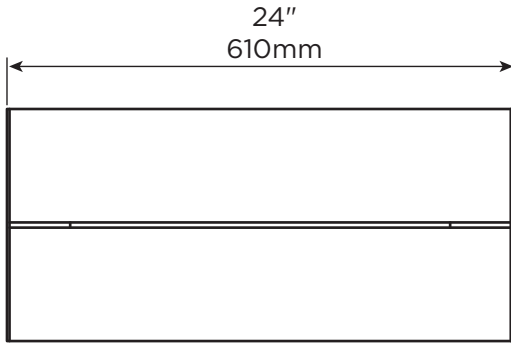MAX
500lbs
227kg

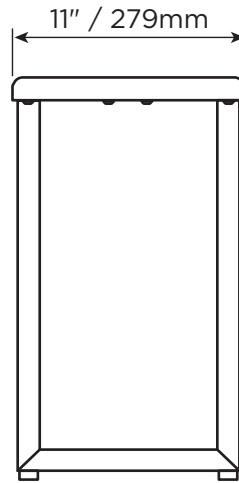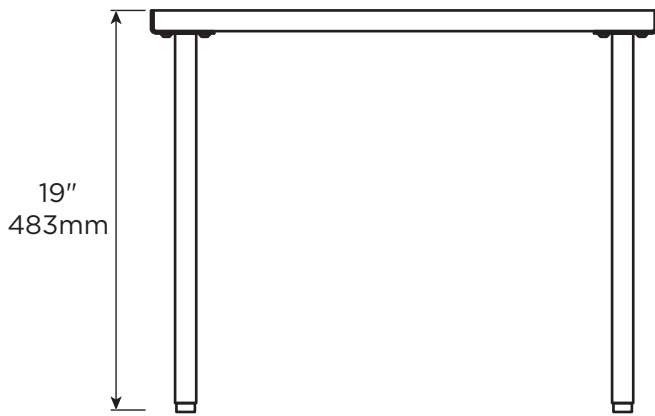E

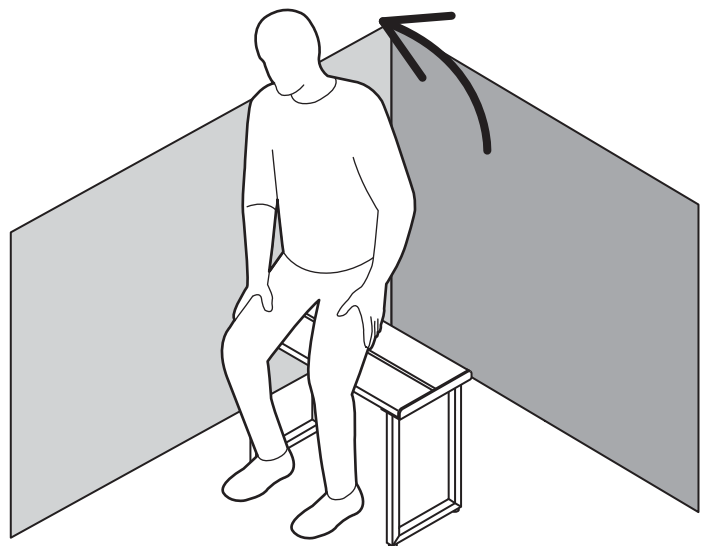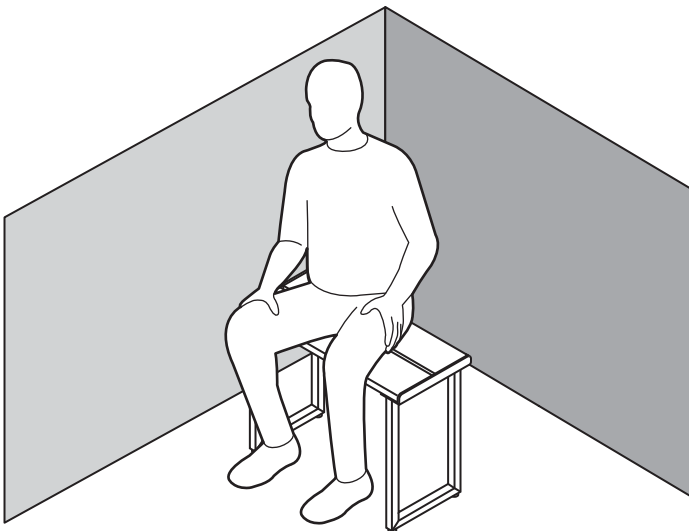English

OVERVIEW

Thank you for choosing the Invisia Shower Bench. Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your Invisia Shower Bench is properly assembled, installed, and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your Invisia Shower Bench installed by a qualified contractor.

The Invisia Shower Bench is a high quality, commercial grade shower seat. The Invisia Shower Bench is intended to provide shower seating support for people with reduced mobility. The maximum user weight for this product is 500lbs / 227kgs. Note: Product is not intended to support full body weight (i.e. do not stand on the bench). The Invisia Shower Bench is not to be used in any other way than described above.

LIMITED LIFETIME WARRANTY

Products are covered by a limited lifetime warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, or the use of corrosive or abrasive cleaning products.

Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described herein; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon. Specifications subject to change without notice.

HealthCraft Products Inc.
2790 Fenton Road
Ottawa, Canada
K1T 3T7

Gate 88
Kanalgatan 45B
Skellefteå, Sweden
93123

DELIVERY CONTENTS / SHOWER BENCH

FIGURE A.

1. Wood Panel with Barrel Nuts (x2)
2. Support Leg with Friction Feet (x2)
3. 1/4-20 x .75" Screw, S/S (x12)
4. #12 Lock Washer, S/S (x12)
5. 5/32" Hex Key

INSTALLATION WARNINGS

FIGURE B.

1. Shower Bench is intended to be used as a sitting surface only (i.e. downward vertical force only).
2. CAUTION - Do not exert sideward forces on the bench.
3. CAUTION - Do not use bench for more than 1 person at a time.
4. CAUTION - Do not use bench for purposes other than sitting (i.e. do not stand on the bench).
5. CAUTION - Do not use bench on uneven surfaces that may cause the seat to be unstable.
6. CAUTION - Seat and bathtub are slippery when wet.

ASSEMBLY

FIGURE C.

1. Ensure threaded holes in barrel nuts are aligned with holes in the wood. Use slot drive screwdriver to turn barrel nut if required.
2. Align the plate of the support legs with the holes in the wood.
3. Thread screw and washer to the barrel nut. Tighten to secure (x12). CAUTION: Do not overtighten screw - excessive force can crack wood.
4. Flip Shower Bench upright. Inspect hardware regularly, and tighten if required.

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