PLUS Corner Shelf

PLUS-CS
(DC192 - RevA)


healthcraftproducts.com

A

1


2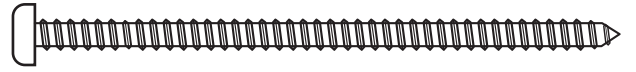


B


1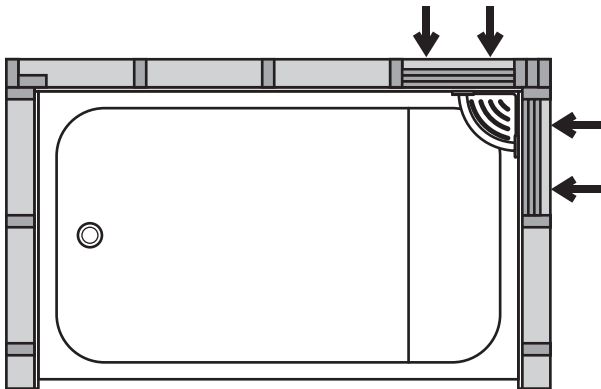


2

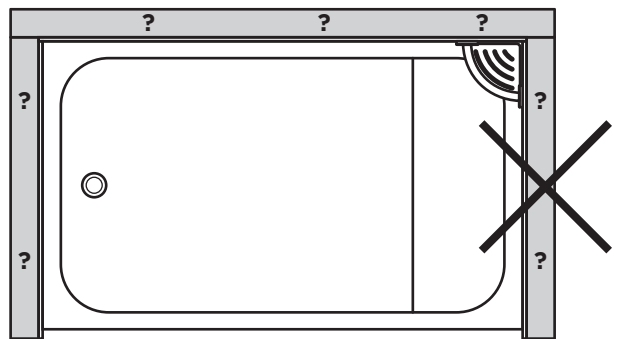
3

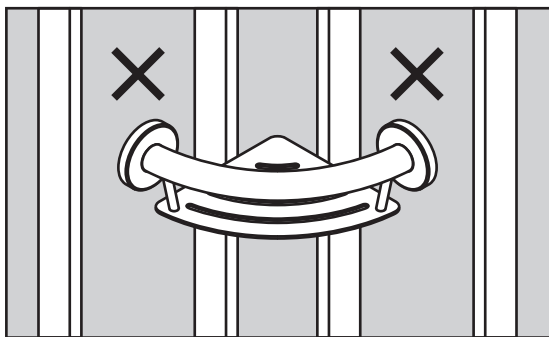
4

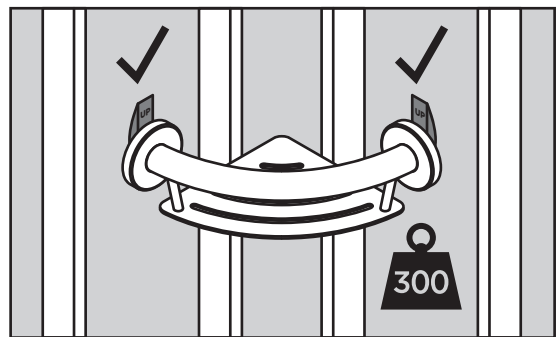
5

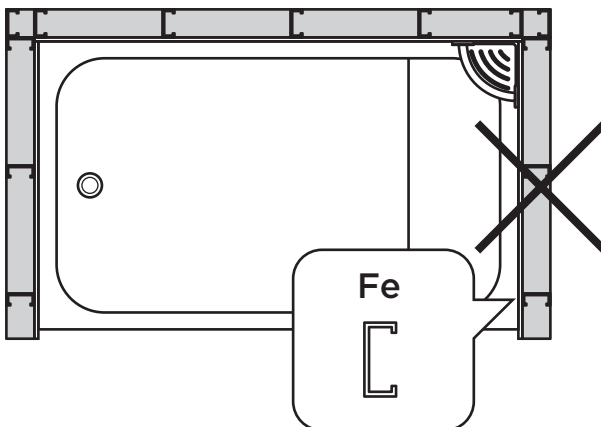
6

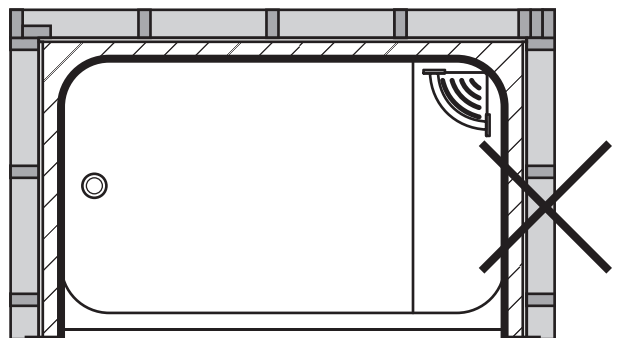
C

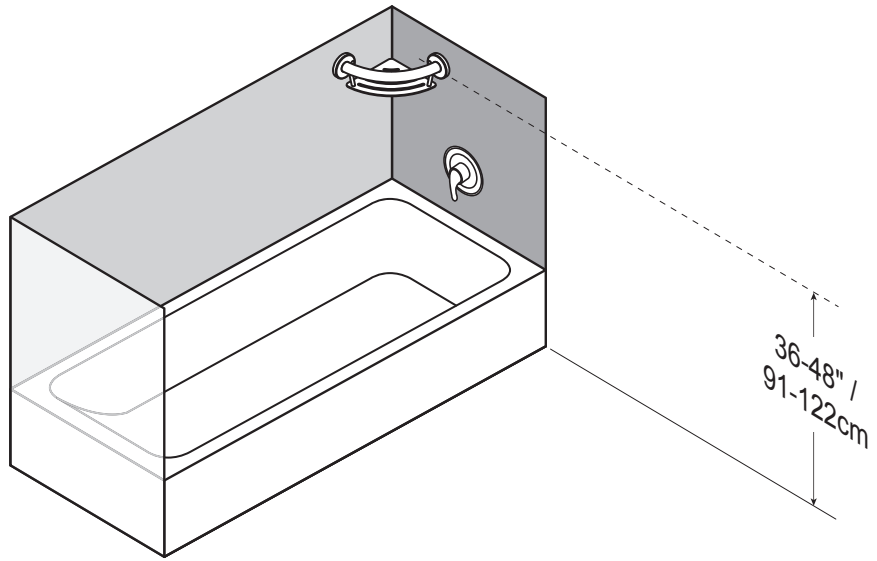
D

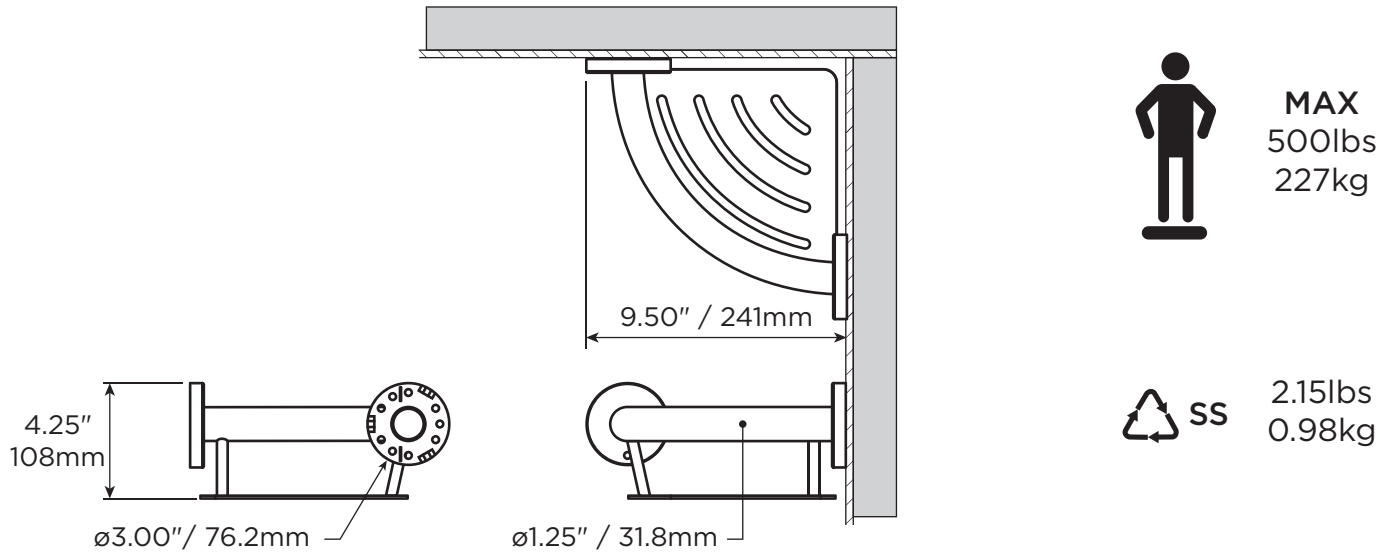
1

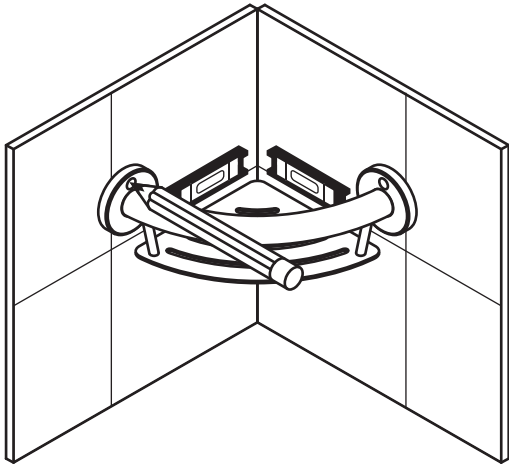
2

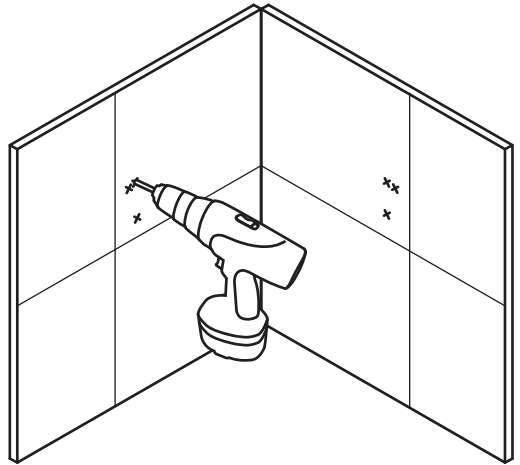
3

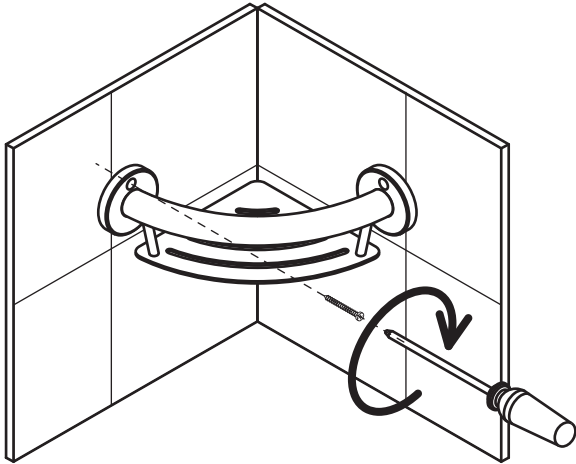
4




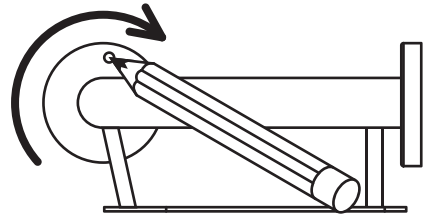
5

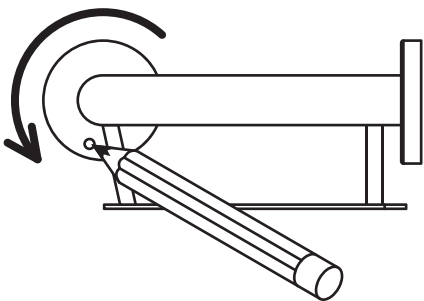6

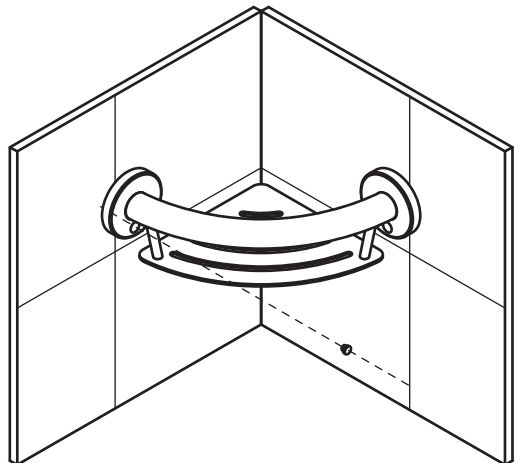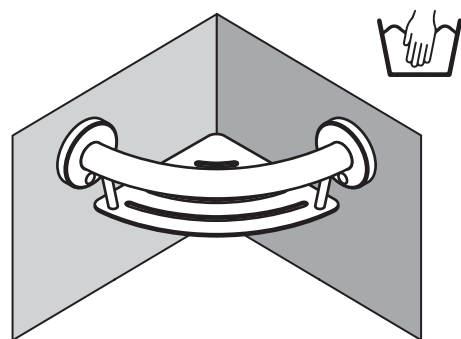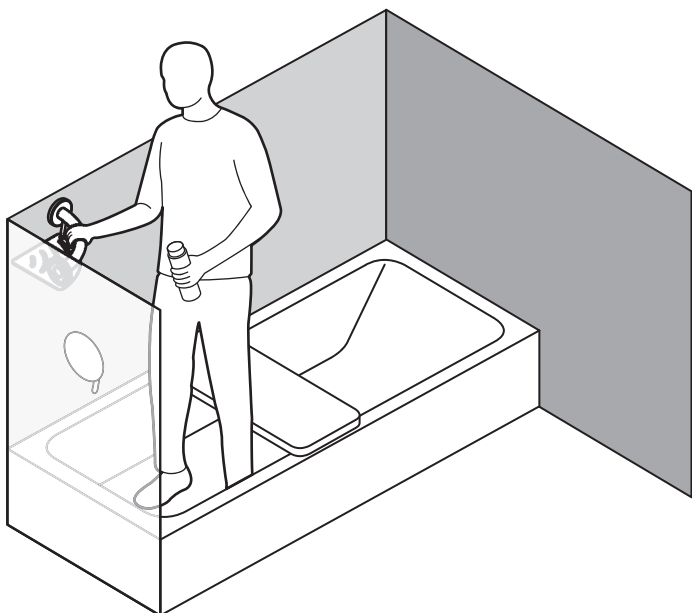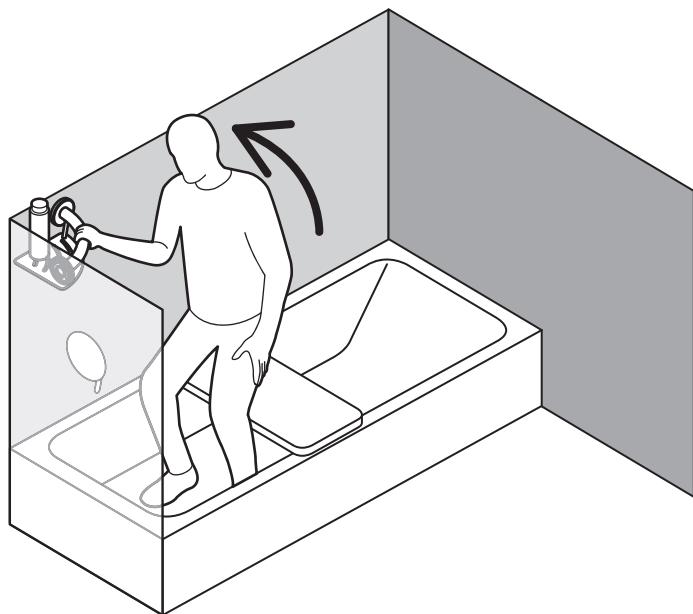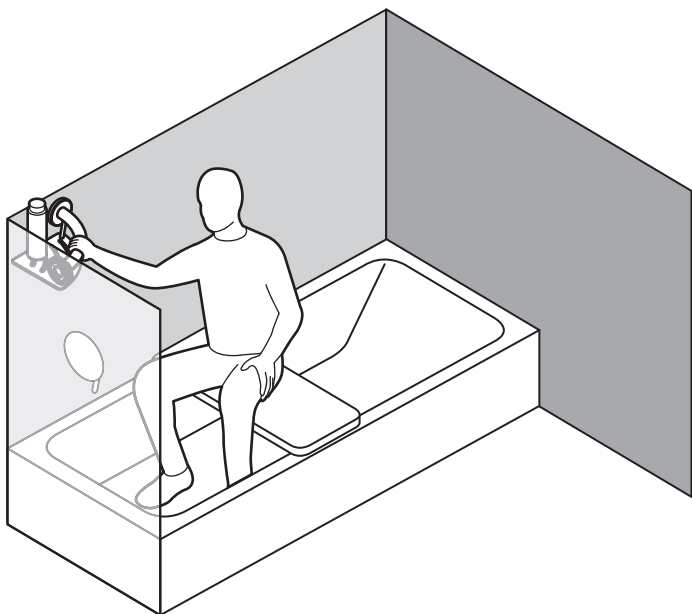English

OVERVIEW

Thank you for choosing the PLUS Corner Shelf. Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your PLUS Corner Shelf is properly assembled, installed, and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your PLUS Corner Shelf installed by a qualified contractor.

The PLUS Corner Shelf is a combination grab bar and corner shelf, designed to disguise safety in a bathroom accessory. The PLUS Corner Shelf is intended for moderate vertical and horizontal load bearing to provide sitting and standing support for people with reduced mobility. The PLUS Corner Shelf is installed with appropriate fasteners to a structural wall. The tray (which holds bottles) is NOT intended to be a support product. The maximum user weight for this product is 500lbs / 227kgs. Note: Product is not intended to support full body weight. The PLUS Corner Shelf is not to be used in any other way than described above.

LIMITED LIFETIME WARRANTY

Products are covered by a limited lifetime warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, or the use of corrosive or abrasive cleaning products.

Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described hereon; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon. Specifications subject to change without notice.

HealthCraft Products Inc.
2790 Fenton Road
Ottawa, Canada
K1T 3T7

Gate 88
Kanalgatan 45B
Skellefteå, Sweden
93123

DELIVERY CONTENTS / CORNER SHELF

FIGURE A.

1. Rail (Corner Shelf)
2. #10 x 1.75" Wood Screw, S/S (x6)
3. Plastic Plug (x2)

INSTALLATION WARNINGS

FIGURE B.

1. Rail must be installed to walls that are structurally sound. If desired installation location does not have adequate structure, add wood blocking.
2. CAUTION - Do not install if structure behind walls is unknown.
3. CAUTION - Do not install if there is no structure for one side. Do not install if there is no structure behind drywall.
4. To reinforce hollow walls, we recommend Easy-Mount Anchors (each sold separately). Weight capacity 300lbs.
5. CAUTION - Do not install into metal studs.
6. CAUTION - Do not install in fiberglass / acrylic shower enclosures with insufficient stiffness and strength.

SUGGESTED CORNER SHELF LOCATIONS

FIGURE C.

To determine the optimal mounting location, we recommend holding the rail in place and walking through the motions of a transfer (sitting, standing, reaching, etc.). Recommended rail height 36-48" / 91-122cm.

TECHNICAL DATA

See Figure D.

INSTALLATION

FIGURE E.

1. Hold the corner shelf handle against the wall and adjust with level. Using the flanges as a locating guide, mark the screw hole locations.
2. Drill pilot holes for fasteners.
3. Fasten the corner shelf handle to the wall using appropriate fasteners and waterproofing. Inspect hardware regularly, and tighten if required.
4. Rotate the flange cover to gain access to other mounting hole (beside top hole), and repeat steps 1-3.
5. Rotate the flange cover to gain access to other mounting hole (bottom hole), and repeat steps 1-3.
6. Install plastic caps to cover the hole in the flange cover.

PRODUCT USAGE

FIGURE F.

CAUTION - Rail and bathtub are slippery when wet.

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