

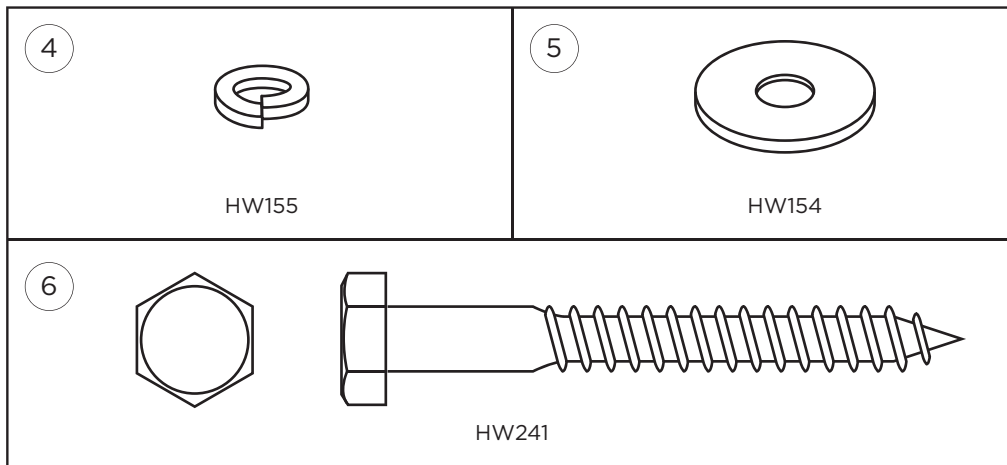
P.T.Rail™ Angled

PT-ANG32-PCW
(DC135 - RevC)


healthcraftproducts.com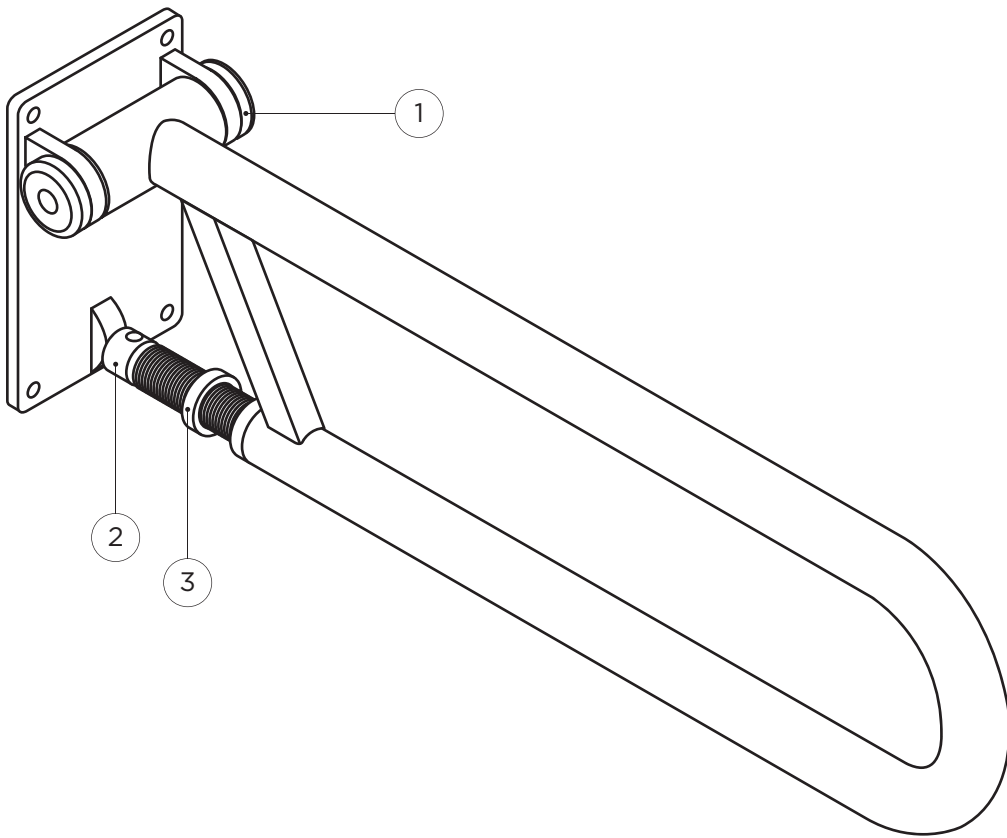


1

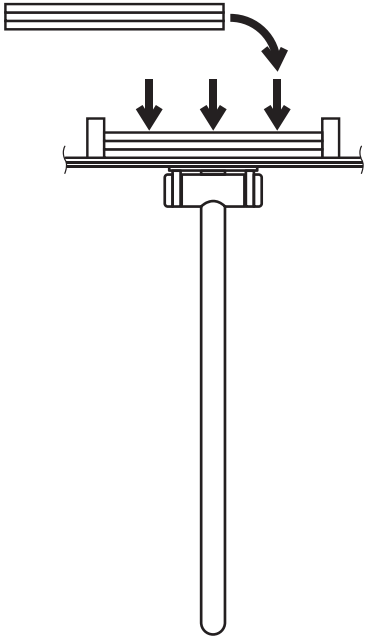
2


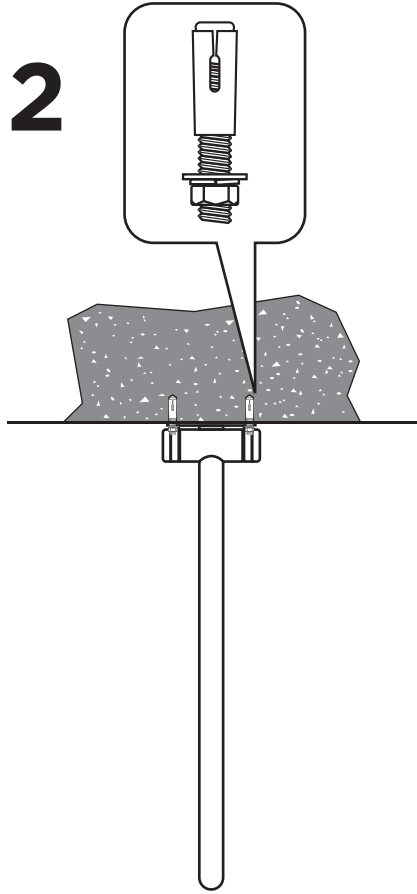
3

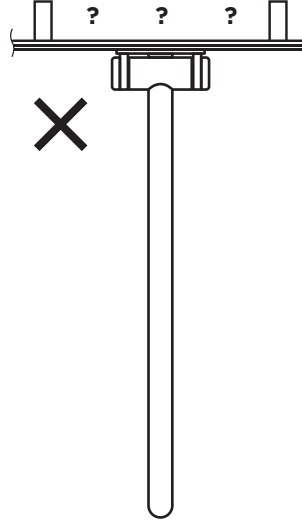4

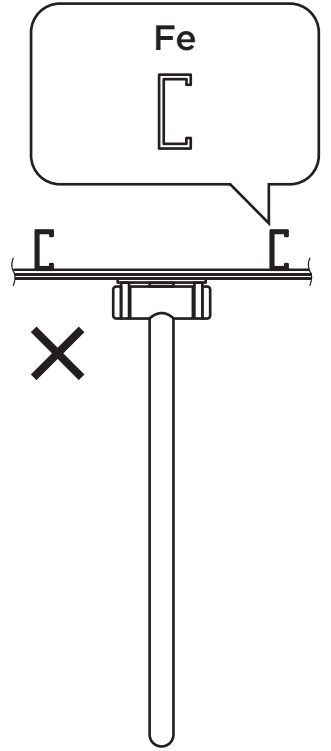5

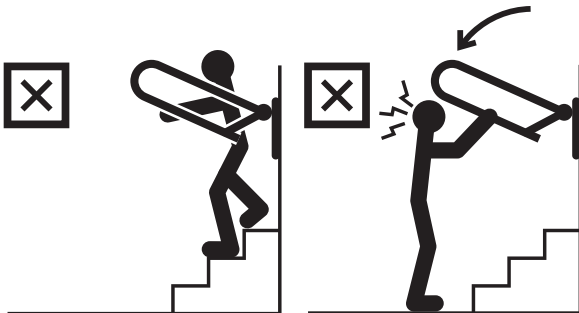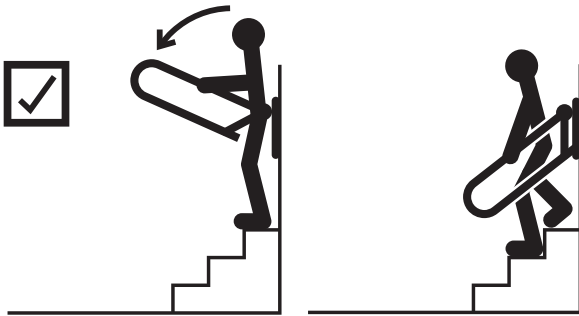
6


7


1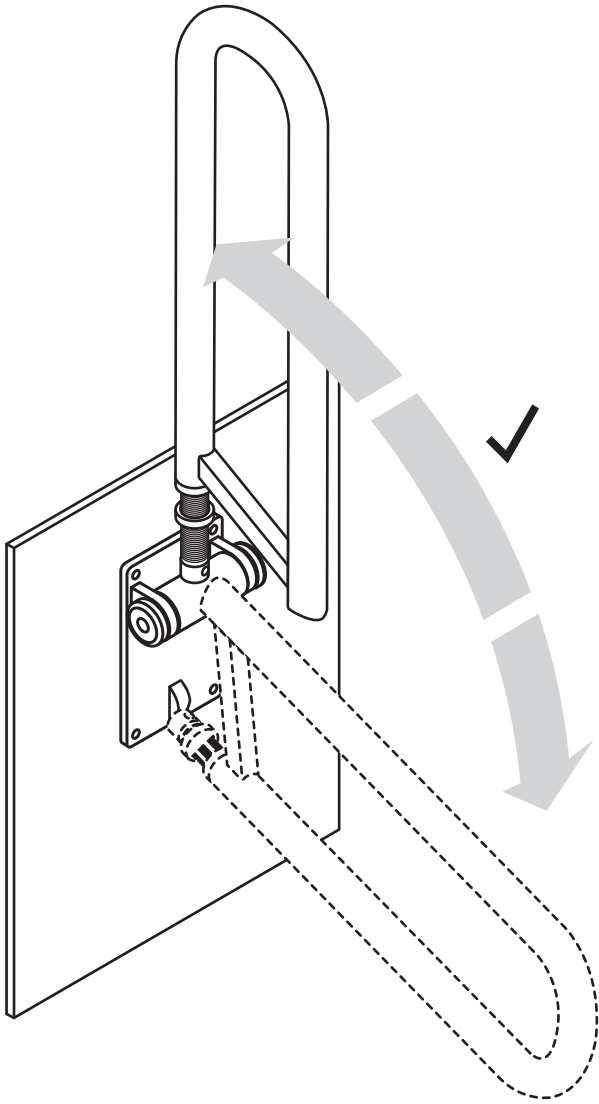


2

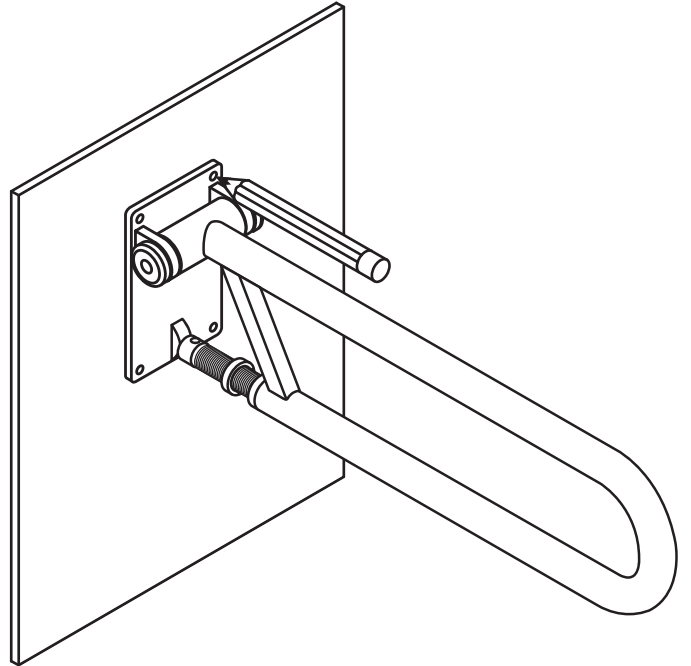3

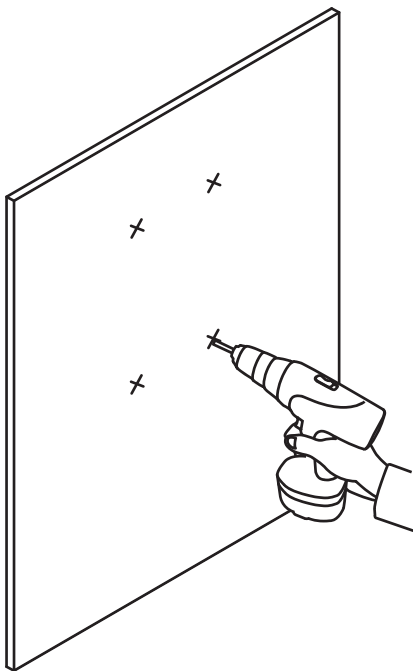
4

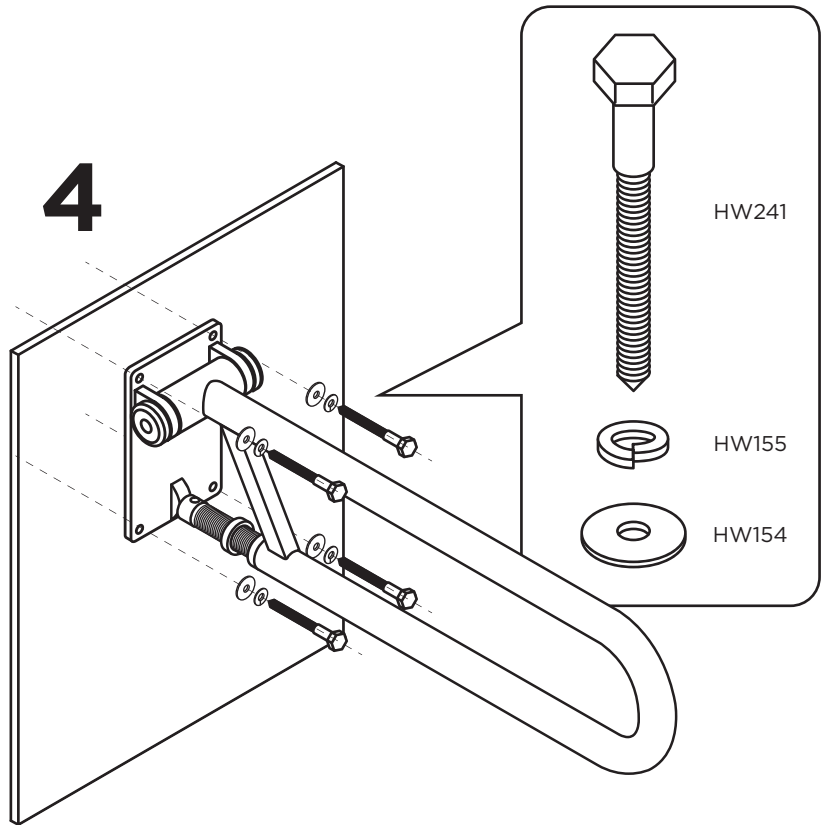
C

D




5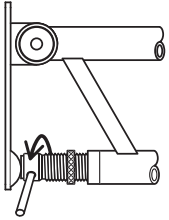


6

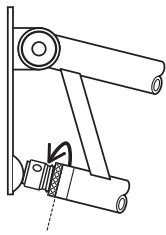7

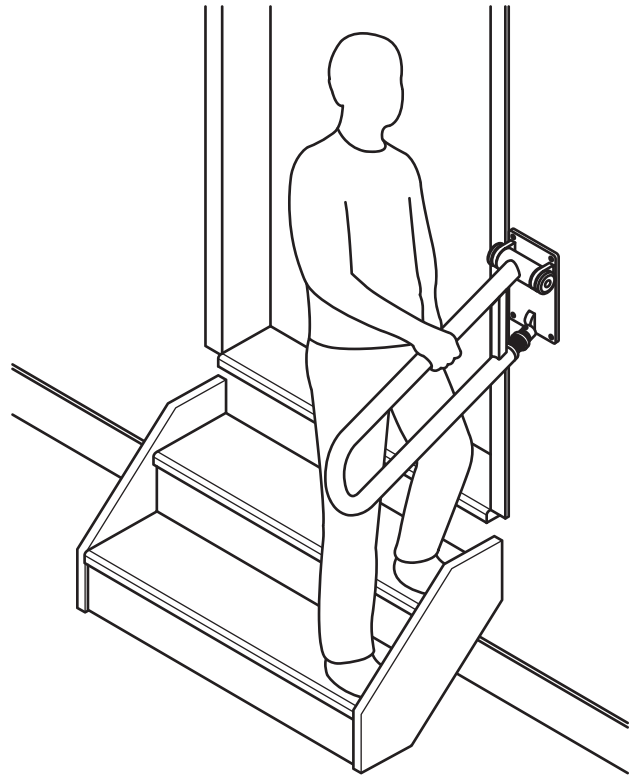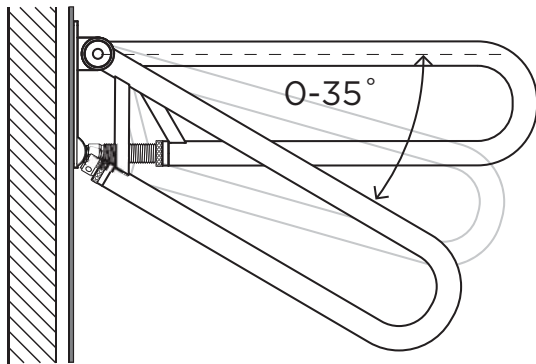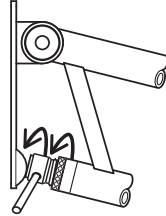E

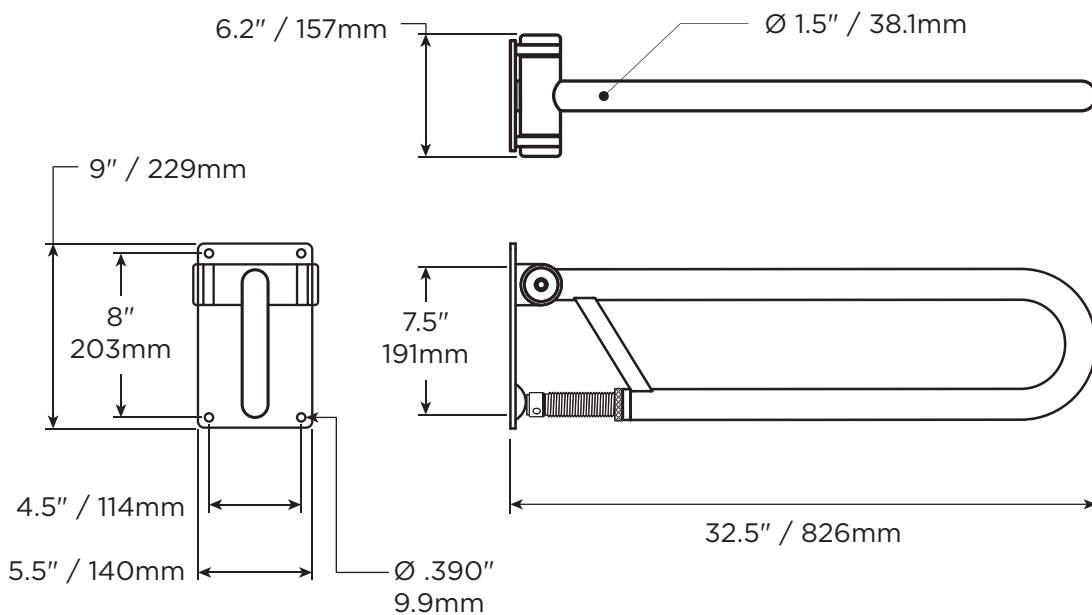MAX
400lbs
181kg

FE
15.0lbs
6.80kg

English

OVERVIEW

Thank you for choosing the P.T.Rail Angled from HealthCraft. Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your P.T.Rail Angled is properly assembled, installed, and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your P.T.Rail Angled installed by a qualified contractor.

The P.T.Rail Angled is a high quality, commercial grade, fold down rail for use on short staircases. The P.T.Rail Angled is intended for moderate vertical and horizontal load bearing to provide assistance and support for people with reduced mobility. The P.T.Rail Angled is intended to bear weight ONLY when the handle is in the down position; the handle is not intended to bear weight in any other position. The P.T.Rail Angled handle is intended to be folded up when not in use. A friction hinge ensures that the handle will not inadvertently lower when it is in the up position. The P.T.Rail Angled is installed with appropriate fasteners to a structural wall.

LIMITED LIFETIME WARRANTY

Products are covered by a limited lifetime warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, or the use of corrosive or abrasive cleaning products.

Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described herein; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon. Specifications subject to change without notice.

HealthCraft Products Inc.
2790 Fenton Road
Ottawa, Canada
K1T 3T7

Gate 88
Kanalgatan 45B
Skellefteå, Sweden
93123

DELIVERY CONTENTS - P.T.RAIL ANGLED

FIGURE A.

1. P.T.Rail Angled
2. Extension Screw
3. Jam Nut
4. Washer, 1/4", Lock (x4)
5. Washer, 1/4", Flat (x4)
6. 1/4" x 3" length Lag Bolt (x4)

P.T.RAIL ANGLED INSTALLATION WARNINGS

FIGURE B.

1. P.T.Rail must be installed to walls that are structurally sound. Add wood blocking if required.
 2. If installation into concrete wall is required, we recommend sleeve or wedge type anchors suitable for wall construction.
 3. CAUTION - Do not install if structure behind wall is unknown.
 4. CAUTION - Do not install into metal studs.
 5. CAUTION - Only use rail for assistance in down position. Use caution when moving rail.
 6. CAUTION - Pinch point between rail and wall plate.
 7. CAUTION - Be careful when on stairs.
- Supplied fasteners for wood structure only. If using other fasteners, ensure screw retention force rating at least 600lbs/272kgs.

INSTALLATION

FIGURE C.

1. Hold P.T.Rail in desired location and ensure unobstructed rail travel in all positions, and the space above rail is adequate to fold.
2. Using wall plate a template, mark off hole locations.
3. Drill pilot holes appropriate for fasteners.
4. Secure P.T.Rail to wall using appropriate fasteners.
5. To change angle of rail, insert shaft of screwdriver thru drive hole of extension screw and turn.
6. Turn jam nut until it is tight with end of rail.
7. Turn both jam nut and extension screw together to jam tight.
8. Remove screwdriver.

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