

INVISIA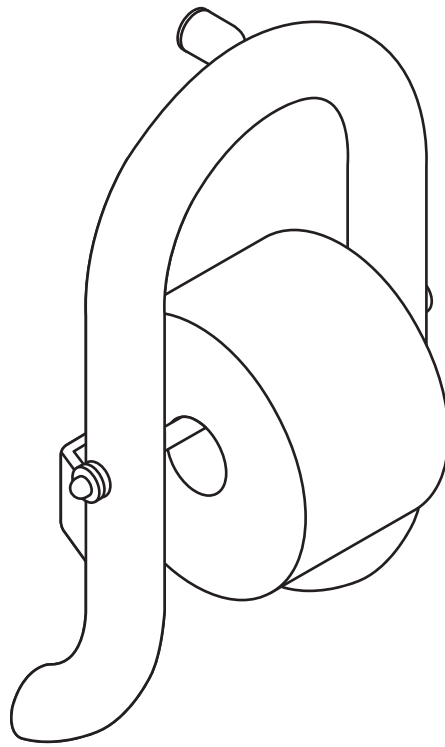

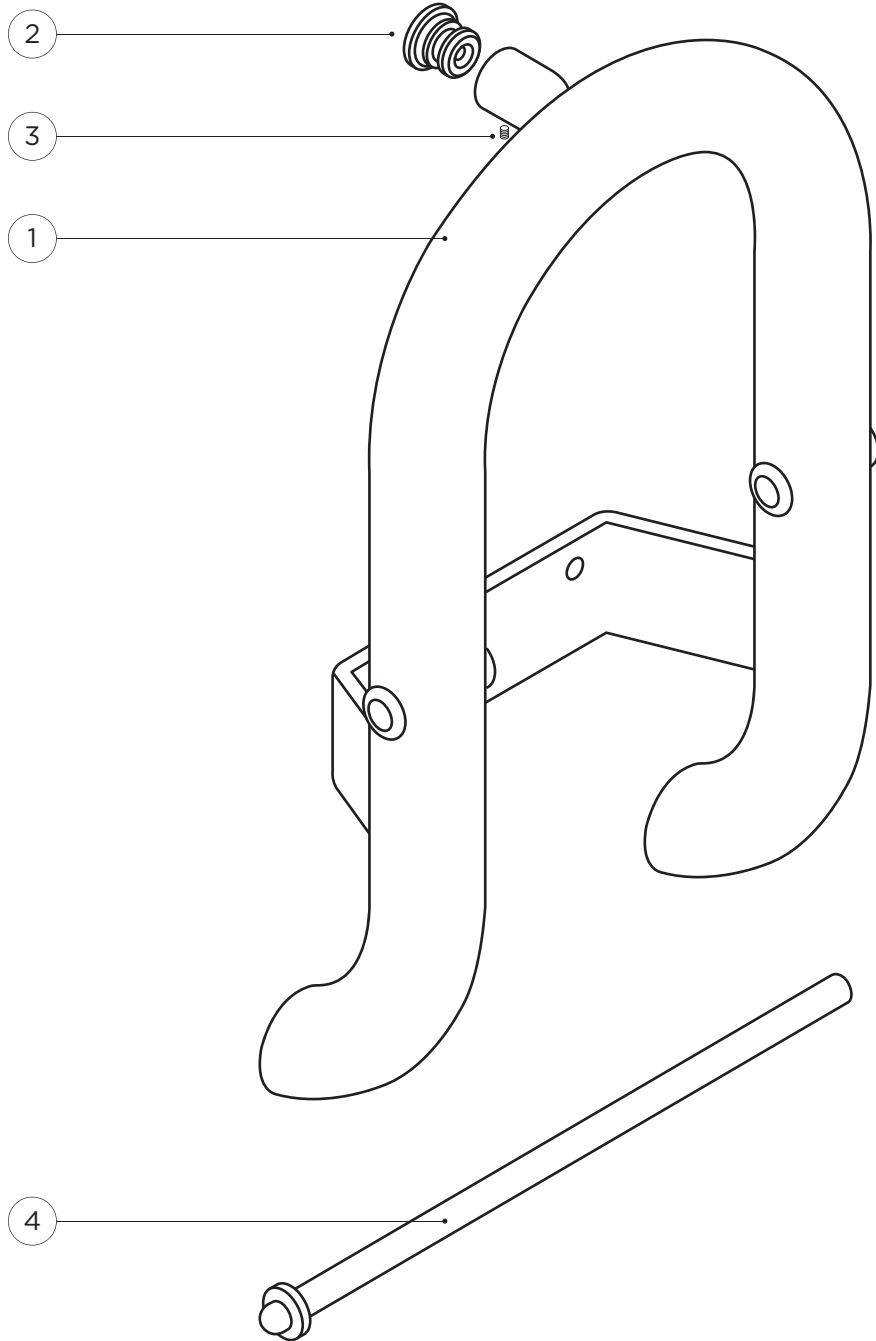Wall Toilet Roll Holder


INV-WTRH
(DC156 - RevE)

invisiacollection.com

<p>5</p>  <p>SD107</p>	
<p>6</p>  <p>HW300</p>	
<p>7</p>  <p>HW177</p>	<p>8</p>  <p>HW198</p>
<p>9</p>  <p>HW175</p>	
<p>10</p>  <p>HW199</p>	
<p>11</p>  <p>HW136</p>	
<p>12</p>  <p>HW225</p>	


1

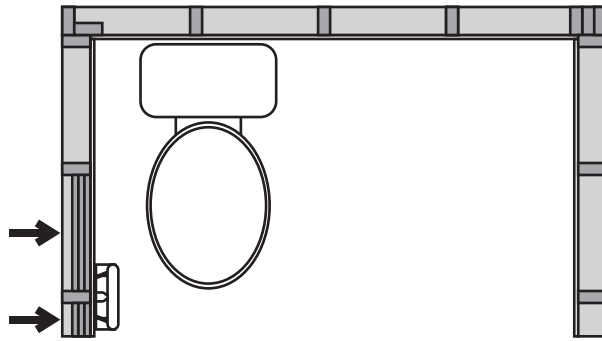2

MAX
500lbs / 227kg

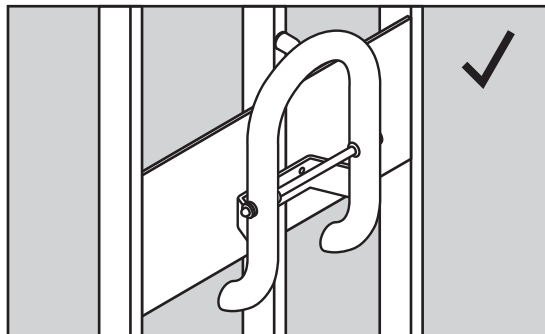
7 x1

9 x1

10 x3

MAX
350lbs / 159kg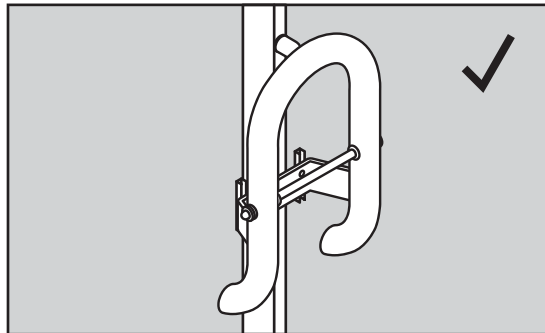


7 x1

9 x1


10 x1

11 x2


12 x2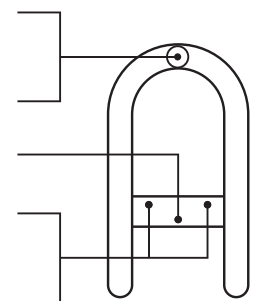


MAX
200lbs / 91kg

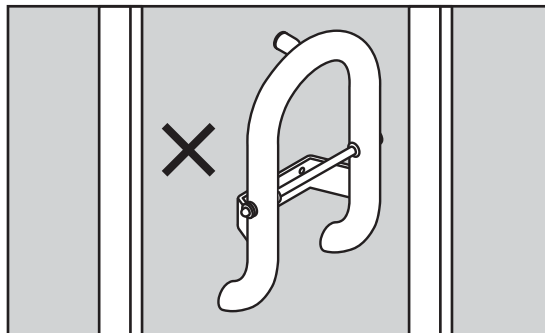
B

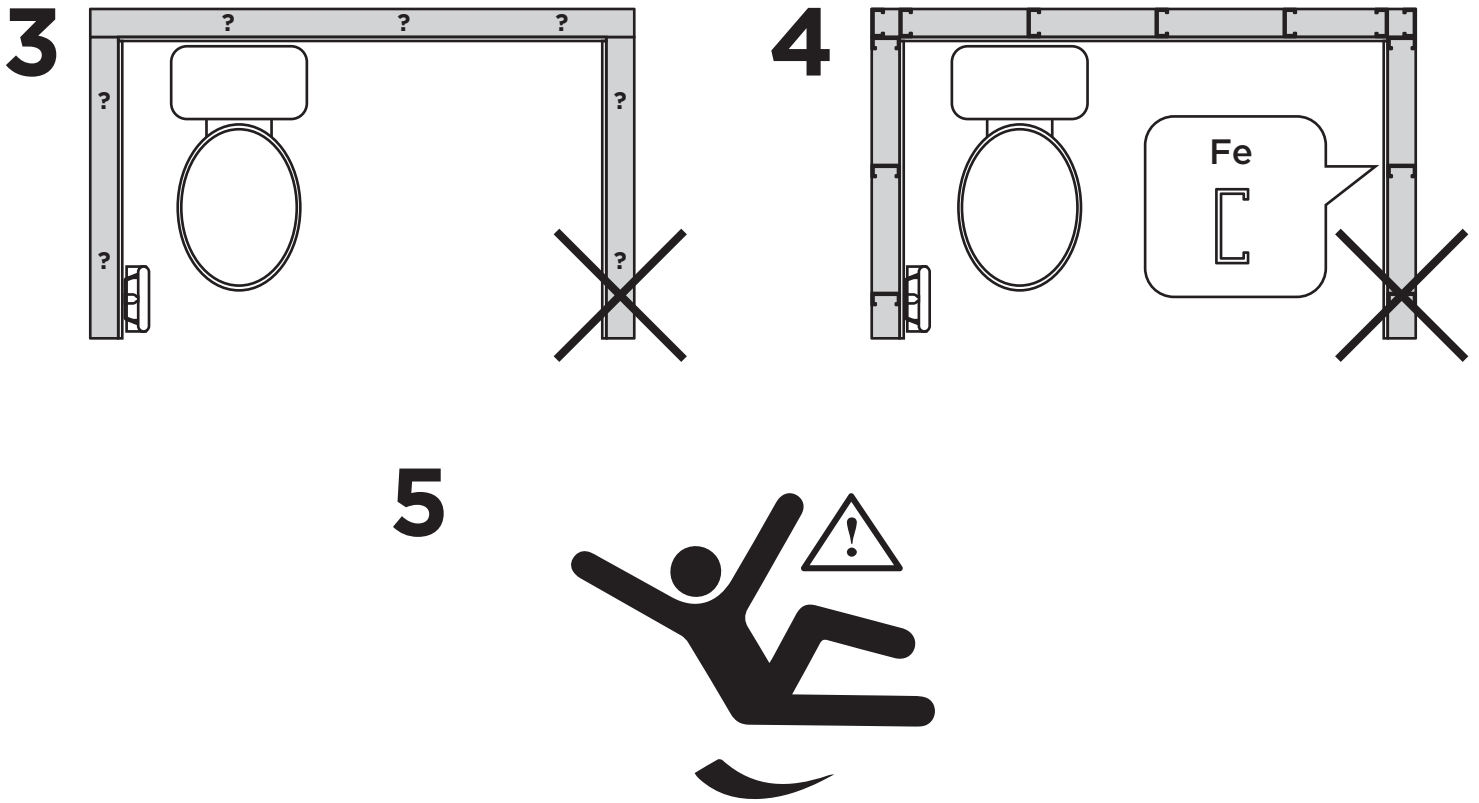
C

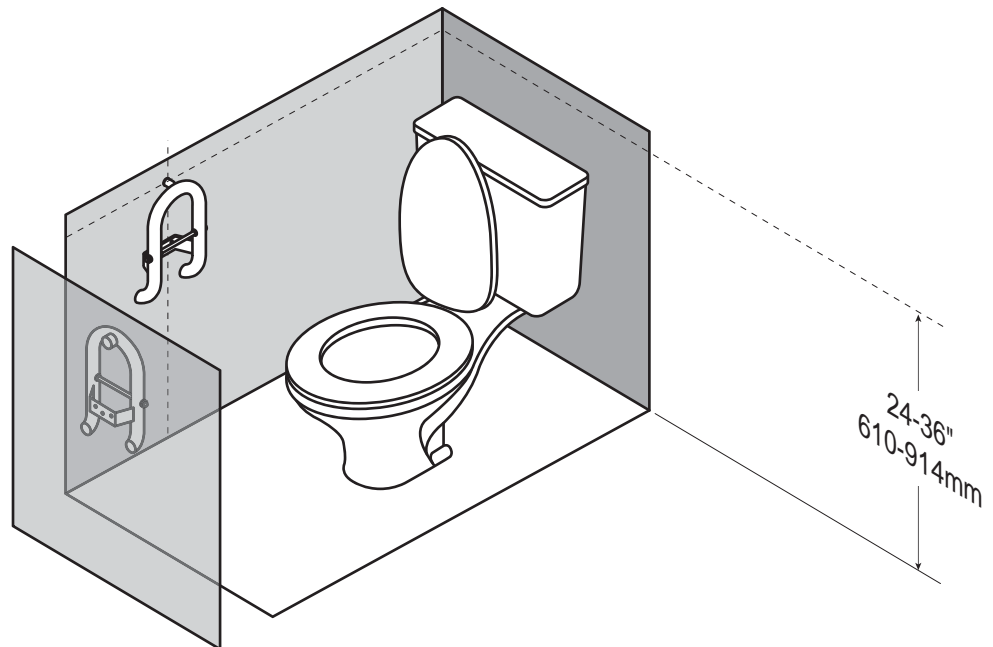
1

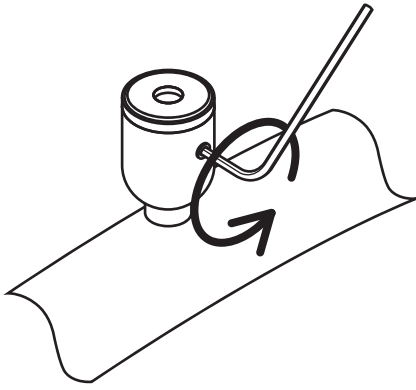
2

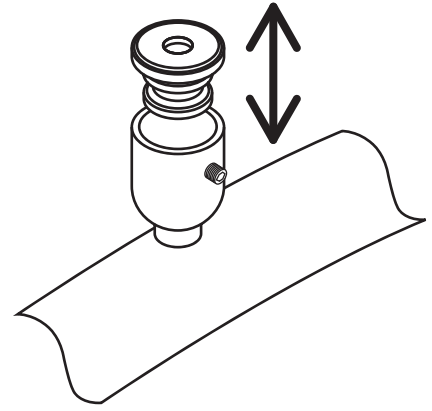
3

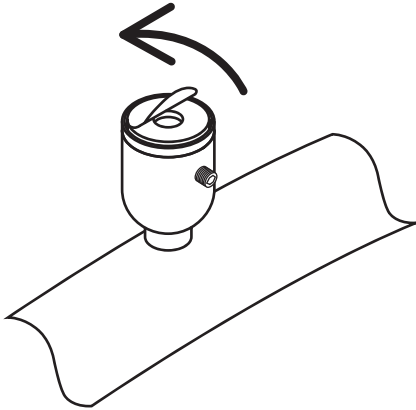
4

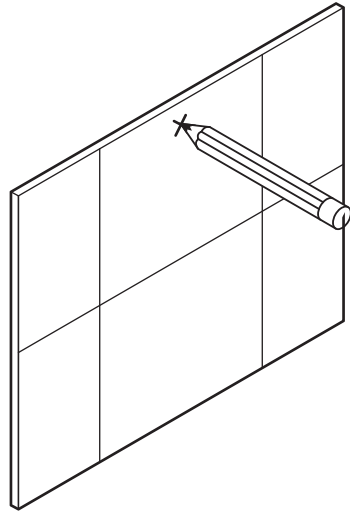
5

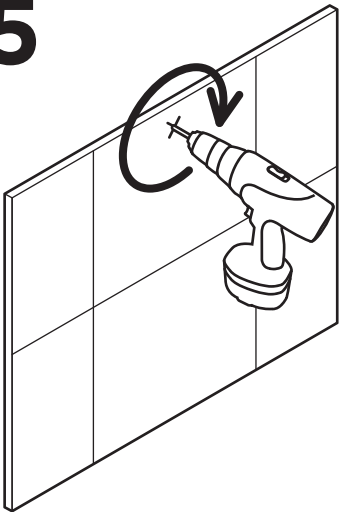
6

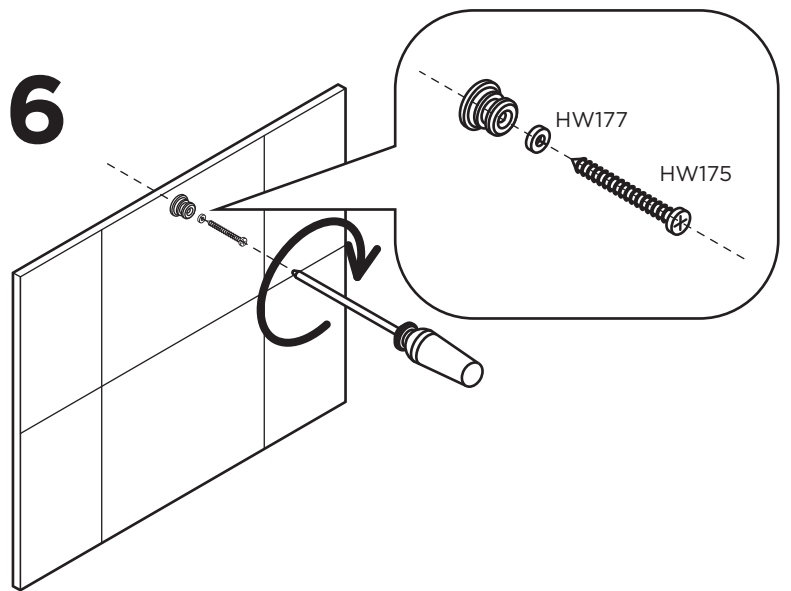
7

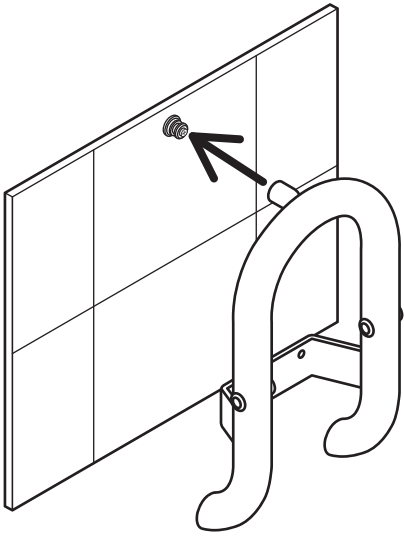
8

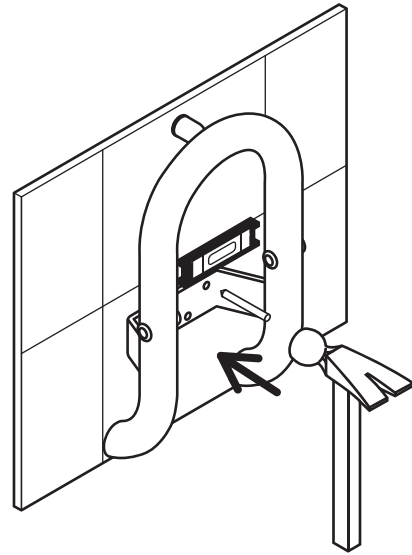
9

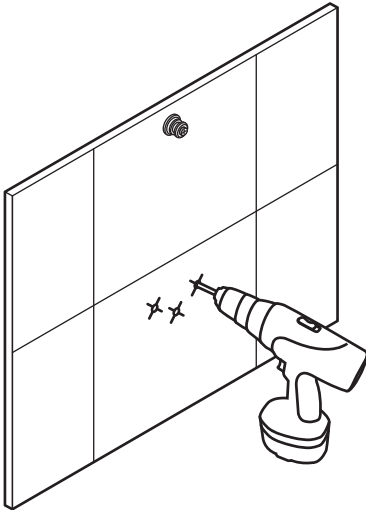
10





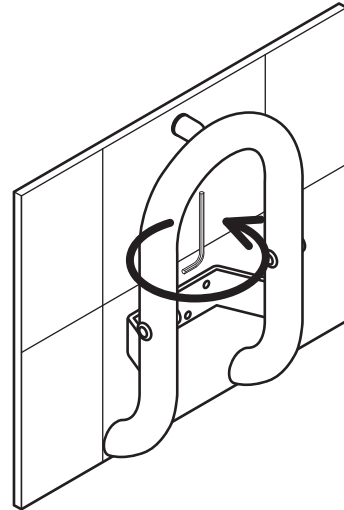
11

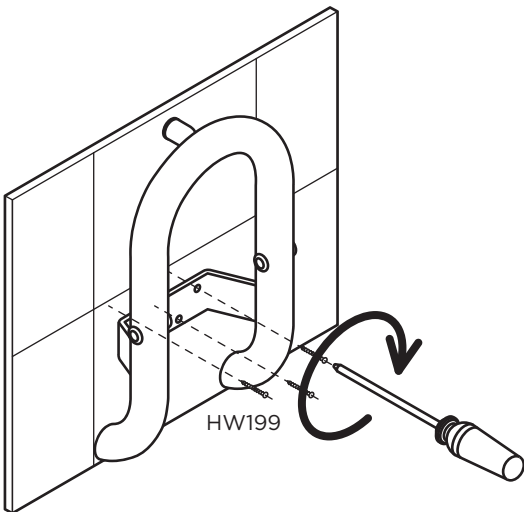12

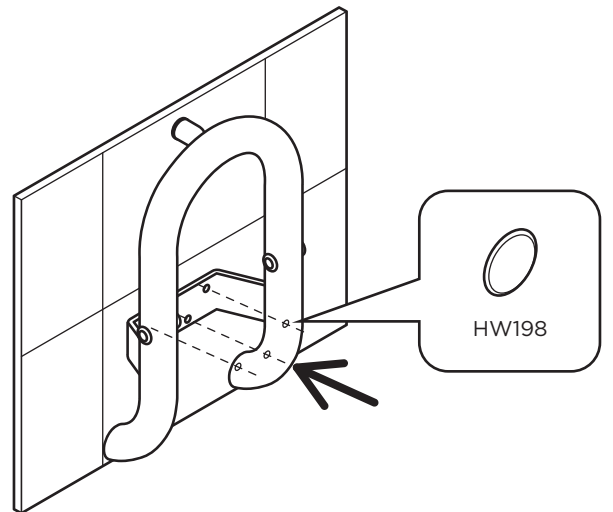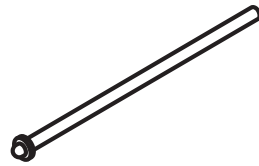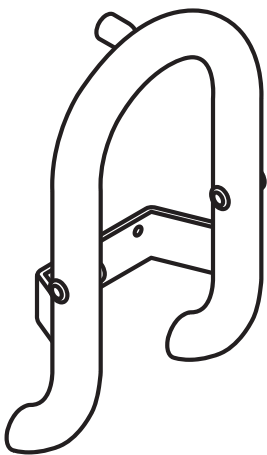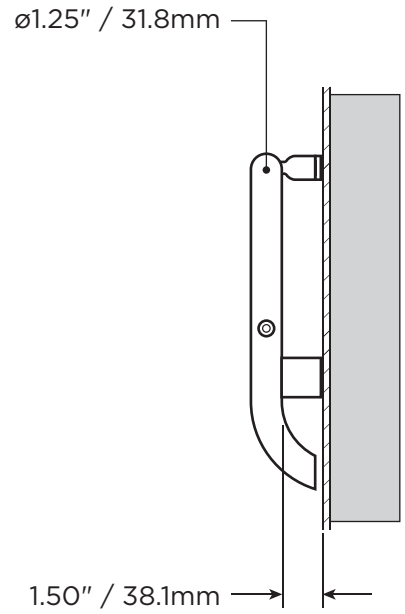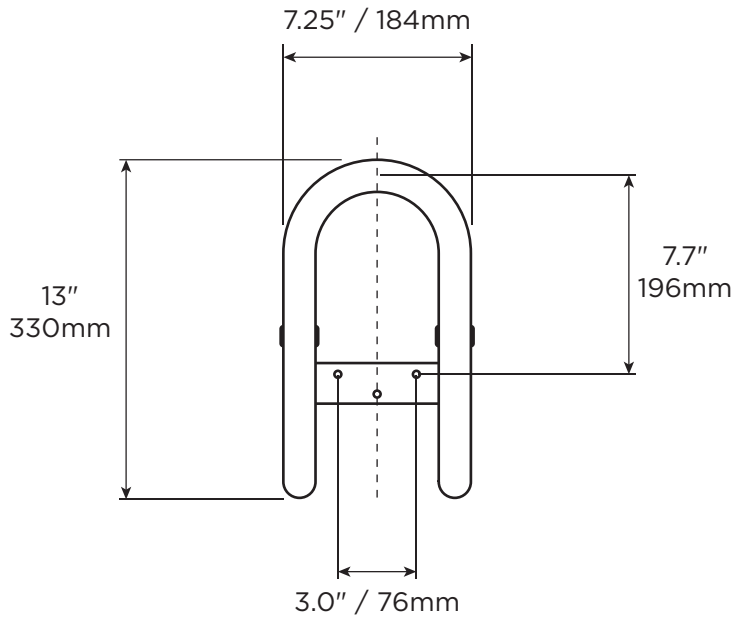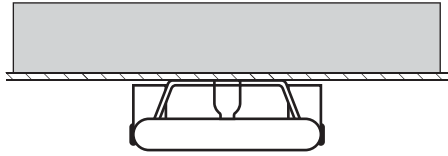2.05lbs
0.93kg



0.05lbs
0.02kg

0.20lbs
0.09kg

MAX

500lbs
227kg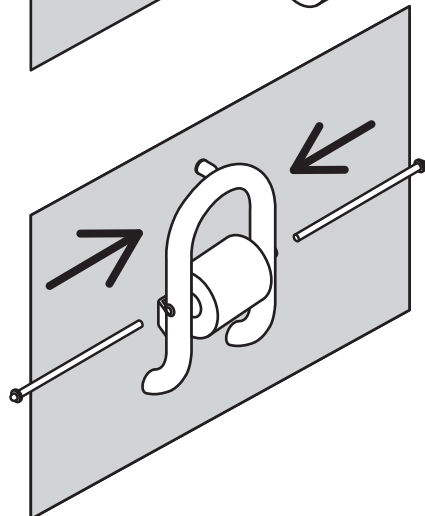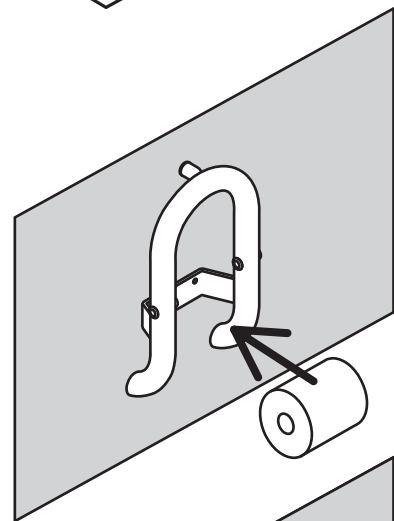

English

OVERVIEW

Thank you for choosing the Invisia Toilet Roll Holder. Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your Invisia Toilet Roll Holder is properly assembled, installed, and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your Invisia Toilet Roll Holder installed by a qualified contractor.

The Invisia Toilet Roll Holder is a combination grab bar and toilet roll holder, designed to disguise safety in a bathroom accessory. The Invisia Toilet Roll Holder is intended for moderate vertical and horizontal load bearing to provide sitting and standing support for people with reduced mobility. The Invisia Toilet Roll Holder is installed with appropriate fasteners to a structural wall. The paper axle (which holds a roll of toilet paper) is NOT intended to be a support product. The maximum user weight for this product is 500lbs / 227kgs. Note: Product is not intended to support full body weight. The Invisia Toilet Roll Holder is not to be used in any other way than described above.

LIMITED LIFETIME WARRANTY

Products are covered by a limited lifetime warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, or the use of corrosive or abrasive cleaning products.

Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described hereon; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon. Specifications subject to change without notice.

HealthCraft Products Inc.
2790 Fenton Road
Ottawa, Canada
K1T 3T7

Gate 88
Kanalgatan 45B
Skellefteå, Sweden
93123

DELIVERY CONTENTS / TOILET ROLL HOLDER

FIGURE A.

- | | |
|------------------------------|----------------------------------|
| 1. Rail (Toilet Roll Holder) | 7. Lock Washer, S/S (x1) |
| 2. Wall Stem Assembly | 8. Plastic Cap (x3) |
| 3. #10-32 Set Screw | 9. #10 Wood Screw, S/S (x1) |
| 4. Paper Axle | 10. #10 Wood Screw CSK, S/S (x3) |
| 5. Center Punch | 11. 3/16" Bolt, S/S (x2) |
| 6. 3/32" Hex Key | 12. Hollow Wall Anchor (x2) |

INSTALLATION WARNINGS

FIGURE B.

1. Rail must be installed to walls that are structurally sound. If desired installation location does not have adequate structure, add wood blocking.
2. For different wall constructions, use appropriate fastener combination (Note: reduced weight capacity when not full structure).
3. If using supplied hollow wall anchors, ensure center screw holes align with vertical wood stud.
4. CAUTION - Do not install if there is no structure behind drywall.
5. CAUTION - Do not install if structure behind walls is unknown.
6. CAUTION - Do not install into metal studs.
7. CAUTION - Rail and bathtub are slippery when wet.

SUGGESTED TOILET ROLL HOLDER LOCATIONS

FIGURE C.

To determine the optimal mounting location, we recommend holding the rail in place and walking through the motions of a transfer (sitting, standing, reaching ,etc.). Recommended rail height 24-36" / 610-914mm.

INSTALLATION

FIGURE D.

1. Back off set screw from rail socket.
2. Allow wall stem to freely move in and out of rail socket.
3. Peel off label on back of wall stem to expose sticker.
4. Mark location of top wall stem.
5. Drill pilot hole appropriate for fastener.
6. Fasten the wall stem to the wall using supplied lock washer and fastener.
7. Place rail back onto the fastened wall stem.
8. Using wall plate as guide, mark location of bottom holes. Ensure level.
9. Drill pilot holes appropriate for fasteners.
10. Place rail back onto the fastened wall stems, and secure rail to wall stem by tightening set screws.
11. Secure the wall plate to the wall using appropriate fasteners (x3).
12. Cover exposed screw heads with push on plastic caps (x3). Inspect hardware regularly, and tighten if required.

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