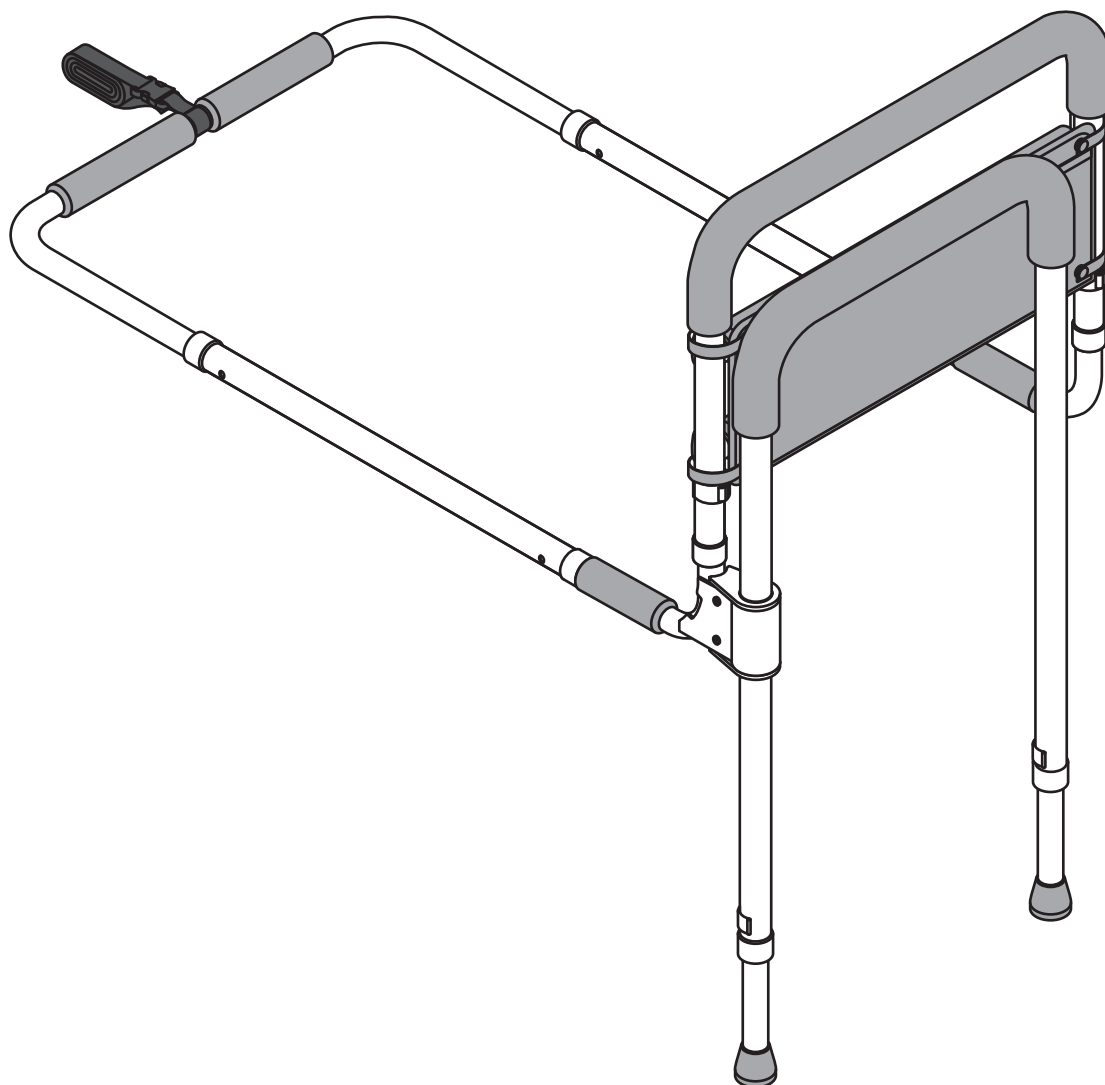
Smart-Rail™

SRS-S
(DC103 - RevF)

healthcraftproducts.com

WARNING FOR BED SAFETY

WARNING

ENTRAPMENT HAZARD

There is a risk of entrapment associated with all bedside mobility aids. Entrapment occurs when an individual becomes wedged between a bed rail and another object (example: mattress). Entrapment can result in serious injury and death. Read and understand the enclosed Entrapment Prevention Guidelines document. See the additional Entrapment Prevention Guidelines:

www.fda.gov/bedsafety

<https://www.cpsc.gov/safety-education/safety-guides/furniture-furnishings-and-decorations/adult-bed-rails>

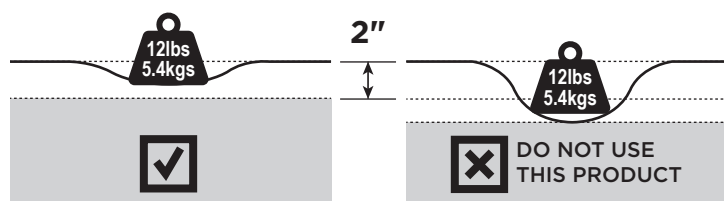
PRIOR TO INSTALLATION, review all of the product specifications and instructions to ensure that it is safe to use with your bed and mattress.

(a) This product is equipped with a permanently attached safety strap to securely install the product to a metal bed frame. NEVER remove this safety strap.

(b) This product is intended to be installed to a metal bed frame. Never use this product without a bedframe. Never use this product on adjustable beds.

(c) PRIOR TO EVERY USE, ensure there is no gap between the product and the side of the mattress. If there is a gap, the product was improperly installed. **Stop using immediately if product shifts from its original position, if product is damaged, broken or missing parts.**


(d) Increase risk of entrapment when using bed rails with a soft mattress. Test mattress by applying 12lbs (5.4kg) of weight on the edge of the mattress. If the mattress can compress by 2" (51cm), do not use this product.

(e) If product is used in a care facility, follow the installations policies of the institutions. If this product does not comply with those policies, do not install this product.

(f) When changing bed linens, always confirm that product is installed correctly as per installation instructions. Re-tighten the safety strap to eliminate a gap between product and side of mattress.

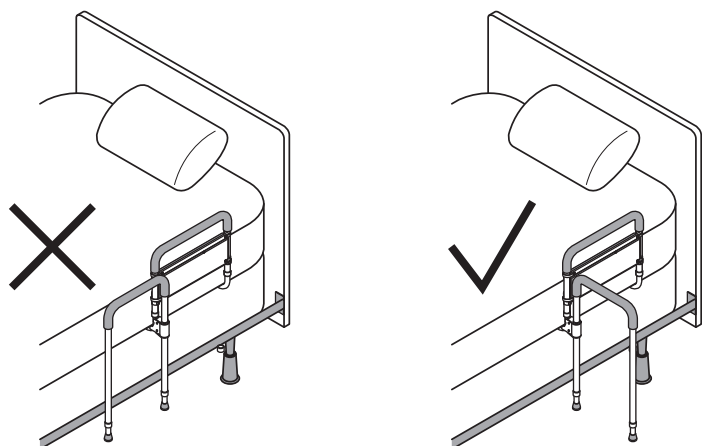(g) This product is not intended as a physical constraint or barrier to exiting the bed. NEVER leave the pivot rail against the bed.

WARNING

ENTRAPMENT HAZARD

NEVER use product without properly securing it to bed. Incorrect installation can allow product to move away from mattress, bed frame and/or head or foot boards, which can lead to entrapment and death.

WARNING

SUFFOCATION, STRANGULATION, ENTRAPMENT HAZARD

If product is installed incorrectly or moves from its initial position gaps can occur which can entrap and cause fatality. People with Alzheimer's disease, dementia or other neurological conditions, or those who are sedated, confused, or frail, are at increased risk of entrapment, suffocation and strangulation.

- NEVER use unless product is tight against mattress, without gaps, and at least 12-1/2 in. from headboard and footboard.
 - NEVER use with children.
 - NEVER use on toddler, bunk, water, electrical or inflatable beds, or on beds with mattress toppers or soft compressible pads.
 - DO NOT USE if bedside organizer is removed from rail.
- This product is under the jurisdiction of the FDA; to report problems or injuries, call 1-800-332-1088.

WARNING

ENTRAPMENT, STRANGULATION, SUFFOCATION AND FALL HAZARDS

Gaps in and around this product can entrap and kill. People with Alzheimer's disease or dementia, or those who are sedated, confused, or frail, and are at increased risk of entrapment and strangulation. People attempting to climb over this product are at increased risk of injury or death from falls. Always make sure this product is properly secured to bed. If product can move away from bed or mattress, it can lead to entrapment and death.

WARNING

ENTRAPMENT, STRANGULATION, SUFFOCATION AND FALL HAZARDS

If the strap provided is not properly secured the product may move into an unsafe position which increases the danger of entrapment. See instructions for proper use of the straps.

For questions about product safety or installation, or in case of a serious event with this product, contact HealthCraft at 1-888-619-9992 immediately. In addition to contacting the manufacturer directly, consumers should report problems to the CPSC at this website: <https://www.saferproducts.gov/> or call 1-800-638-2772, or to the FDA at 1-800-332-1088.

OVERVIEW

Thank you for choosing the Smart-Rail from HealthCraft. Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your Smart-Rail is properly assembled, installed, and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your Smart-Rail installed by a qualified contractor. The Smart-Rail is a bed rail with 2 rails. The fixed rail is to assist with in-bed positioning. The pivot rail swings out to assist with sitting and standing. The Smart-Rail is intended ONLY for moderate vertical load bearing to provide bed positioning, sitting and standing support for people with reduced mobility. The Smart-Rail is installed between the mattress and box spring, and is secured in place with a safety strap. The fixed rail is height adjustable to accommodate the thickness of the mattress. The pivot rail is height adjustable to accommodate the height of the user. The maximum user weight for this product is 300lbs / 136kgs. This product is not intended to support full body weight. The Smart-Rail is not to be used in any other way than described above.

LIMITED LIFETIME WARRANTY

Products are covered by a limited lifetime warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, or the use of corrosive or abrasive cleaning products. Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described hereon; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon. Specifications subject to change without notice. **Manufacturer:** HealthCraft Products Inc. 2790 Fenton Road, Ottawa, Canada, K1T 3T7. © 2024 HealthCraft Products Inc.

DELIVERY CONTENTS / PARTS

FIGURE A.

- | | |
|--------------------------------|---------------------------------|
| 1. Handle w/ Bag Assembly (x1) | 8. Extension Leg (x2) |
| 2. Upright Tube (x1) | 9. Plastic Bushing (x2) |
| 3. Upright Tube w/ Collar (x1) | 10. Locking Pin for Handle (x4) |
| 4. Side Tube (x2) | 11. Locking Pin for Legs (x2) |
| 5. Back Tube (x1) | |
| 6. Safety Strap (x1) | |
| 7. Pivot Rail (x1) | |

SAFETY CONSIDERATIONS

FIGURE B.

1. The Smart-Rail is intended to be used when the bed casters are locked.
2. The Smart-Rail is intended for moderate vertical load bearing ONLY.
3. The Smart-Rail is not intended for sideward (lateral) support.
4. The Smart-Rail is not intended to support the full weight of an individual.
5. The Smart-Rail is not intended for children to play or hang onto.
6. The Smart-Rail is not to be used on electric or hospital style beds.
7. The Smart-Rail is not intended as a physical constraint or barrier to exiting the bed. NEVER leave the pivot rail against the bed.
8. The Smart-Rail is intended to be placed as per following: (A) Side of bed rail must be at least 12.5" / 320mm from end of bed (B) Top of bed rail must be at least 4" / 102.5mm above top of mattress (C) Base of crossbar must be at least 3" / 76mm below top of mattress.
9. The fixed rail is intended to be height adjustable to the mattress thickness. Select the appropriate position (P) for the mattress thickness. The pivot rail is intended to be height adjustable to assist with standing.

ASSEMBLY

FIGURE C.

1. Slide side tubes over back tube and engage spring locks.
2. Select side of bed the Smart-Rail will be installed. Slide upright tubes into sides tubes and engage spring locks; ensure the pivot tube is installed closer to the foot of the bed.
3. Slide handle + bag assembly over upright tubes.
4. Set desired height, align holes and install locking pin.
5. P3 ONLY - cover exposed holes with additional locking pins (x2).

INSTALLATION

FIGURE D.

1. While laying in bed, note position of waistline.
2. Slide rail between box spring and mattress, with the pivot tube in line with waistline.
3. Loop the strap between the rail and the angle iron of the bed frame.
4. Tighten strap to pull rail tight against the side of the bed. Ensure no gap between rail and mattress.
5. Install pivot rail into pivot tube.
6. Slide plastic bushings onto ends of pivot rail.
7. Slide extension legs into pivot rail.
8. To set height of pivot rail in the opened position, align holes, puncture through seal, and install locking pins.

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