


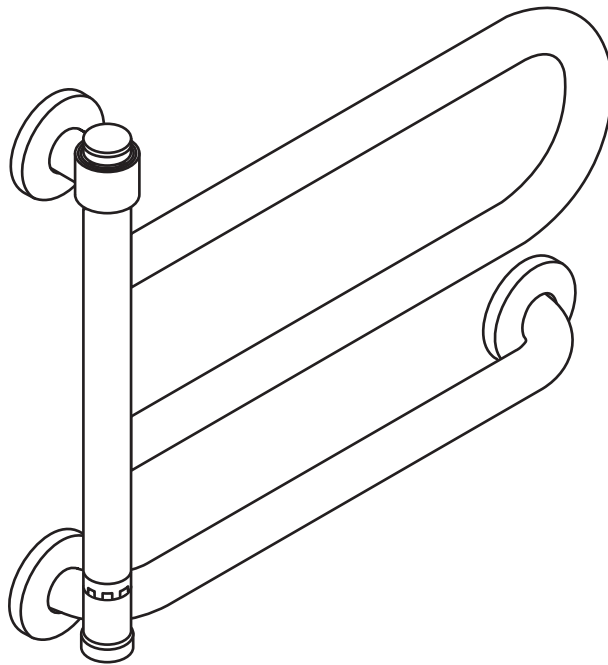
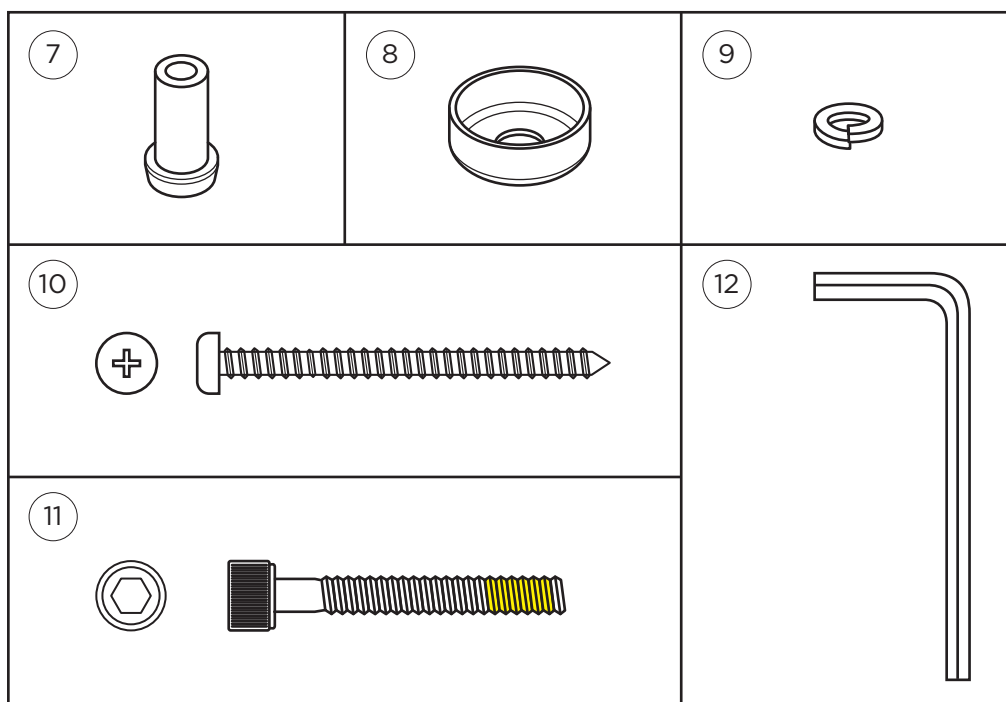
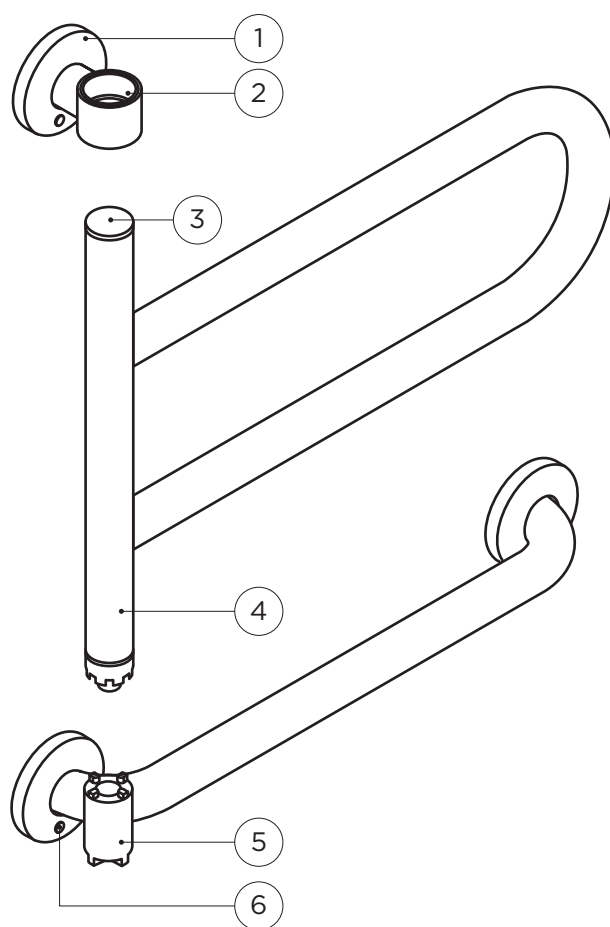
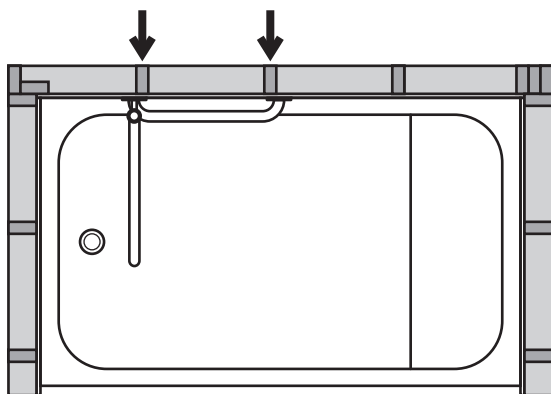
**DependaBar™**  
DBT-xxx  
(DC201 - RevA)


1

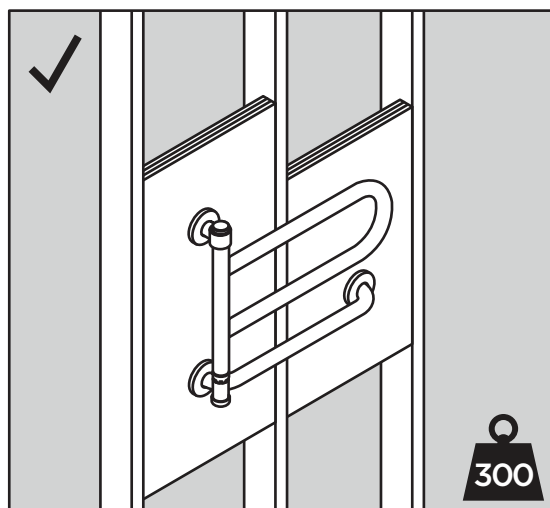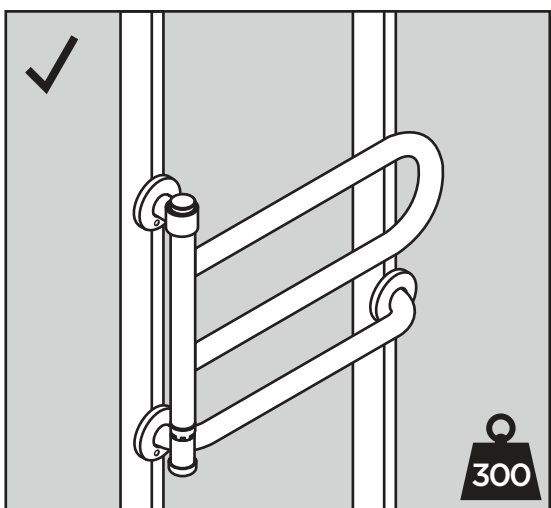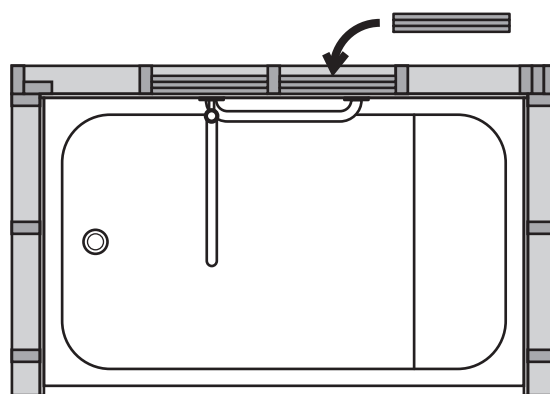
2


3

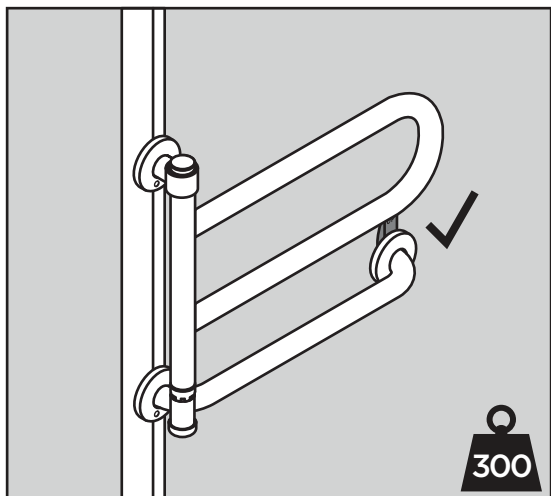
4

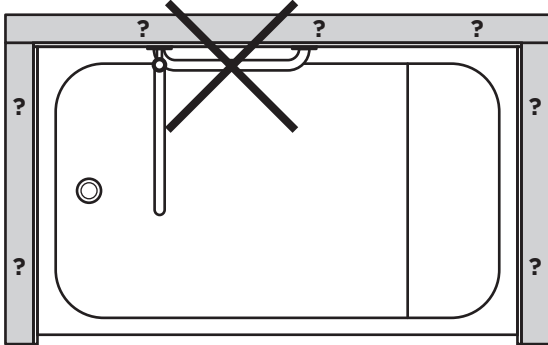
5

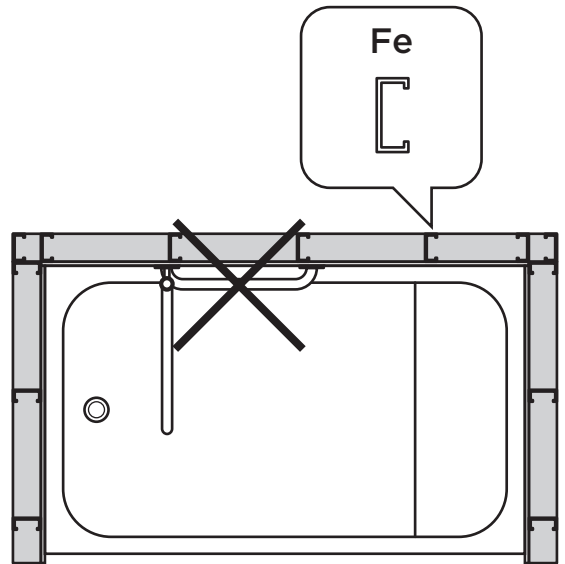6

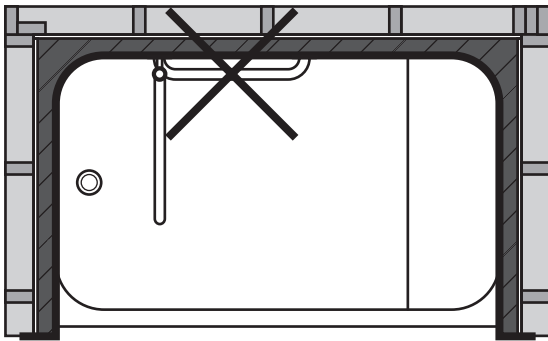
7




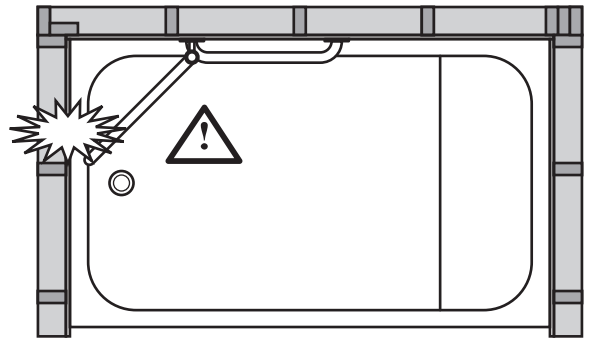
8

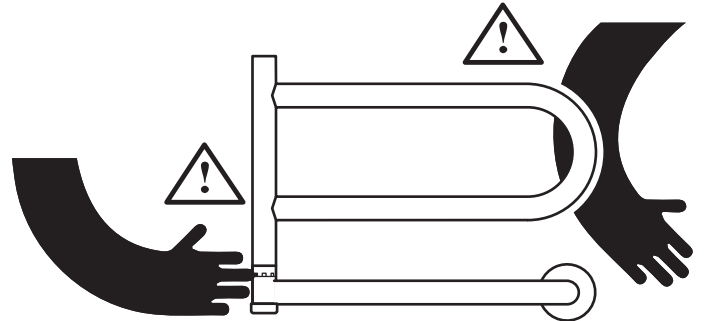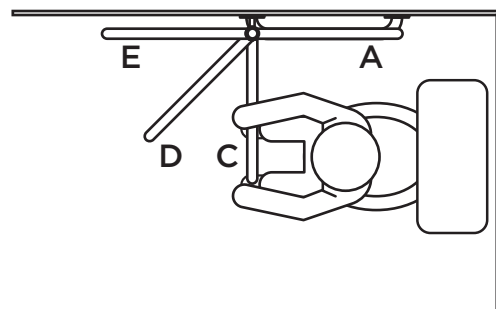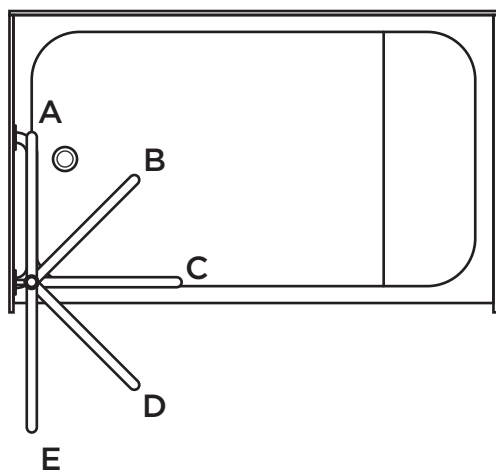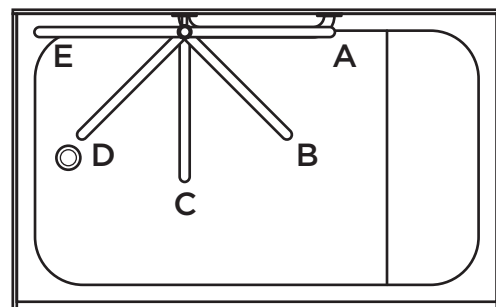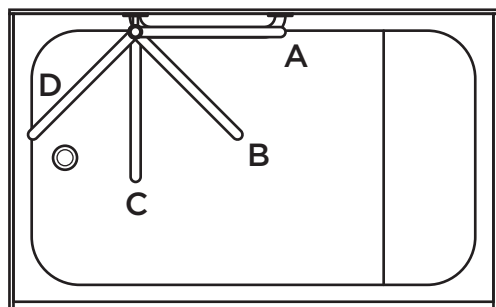
9



1

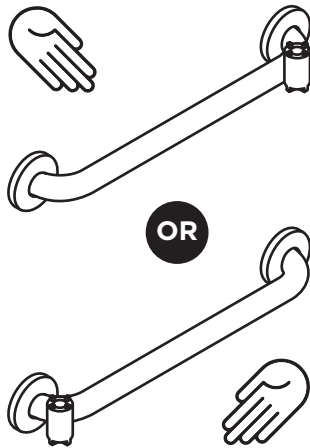
2

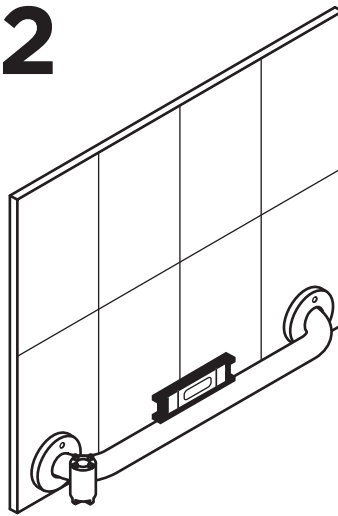
3


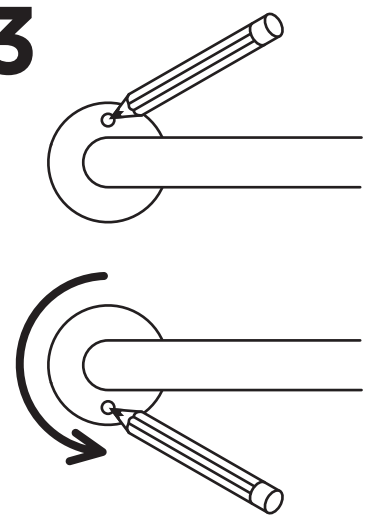
4

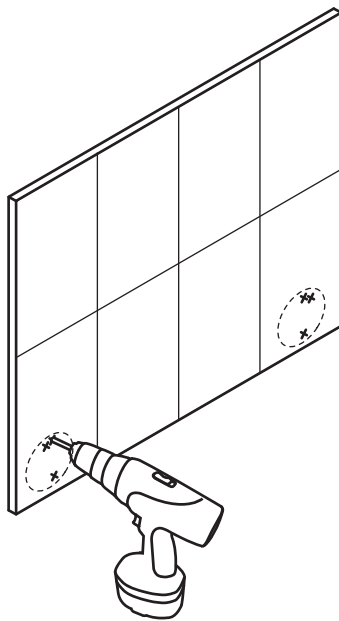
5

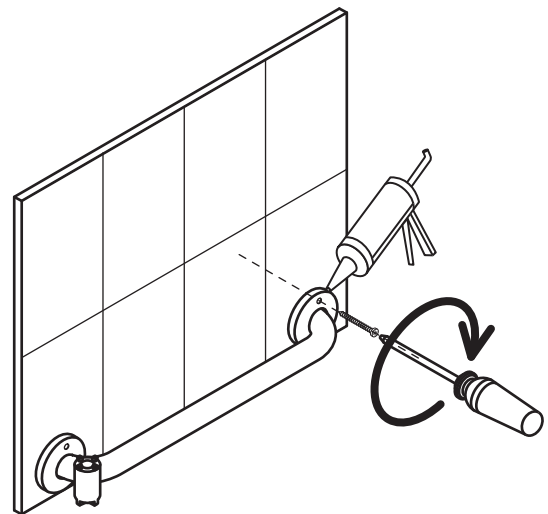6

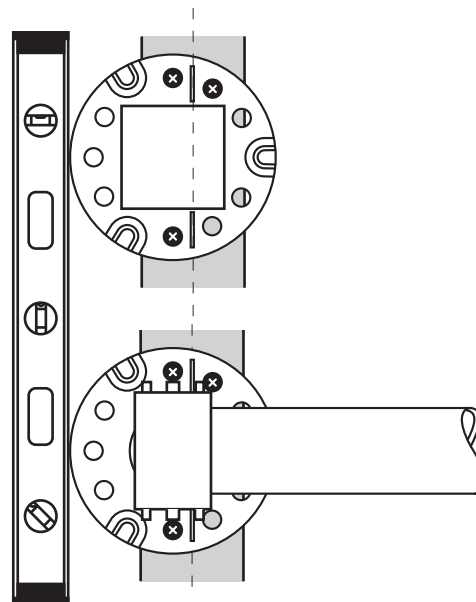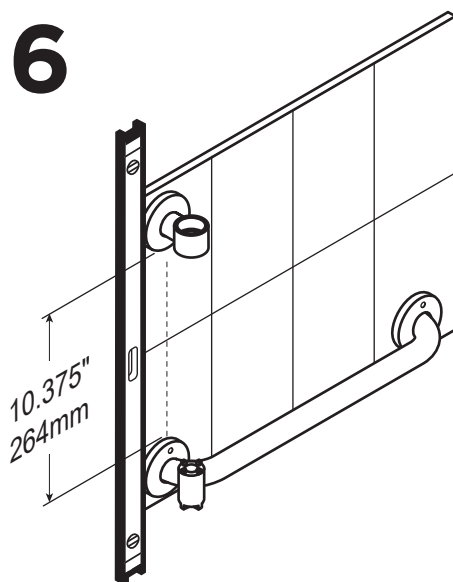
7

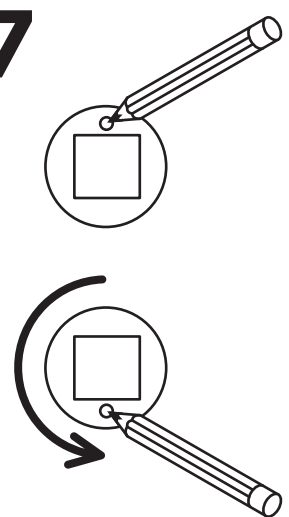
8

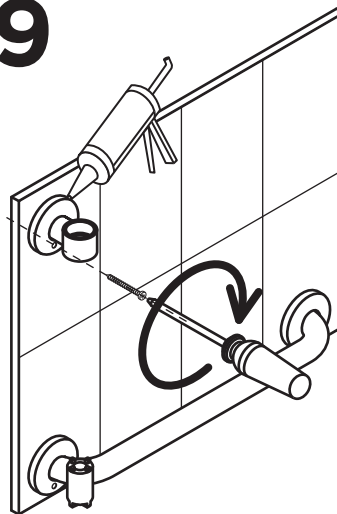
9


10

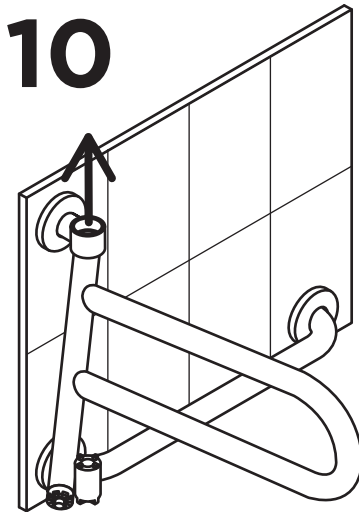
11

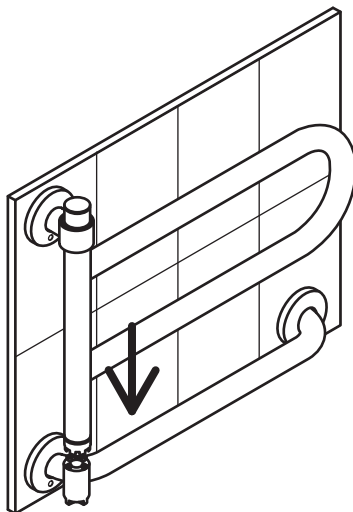
12

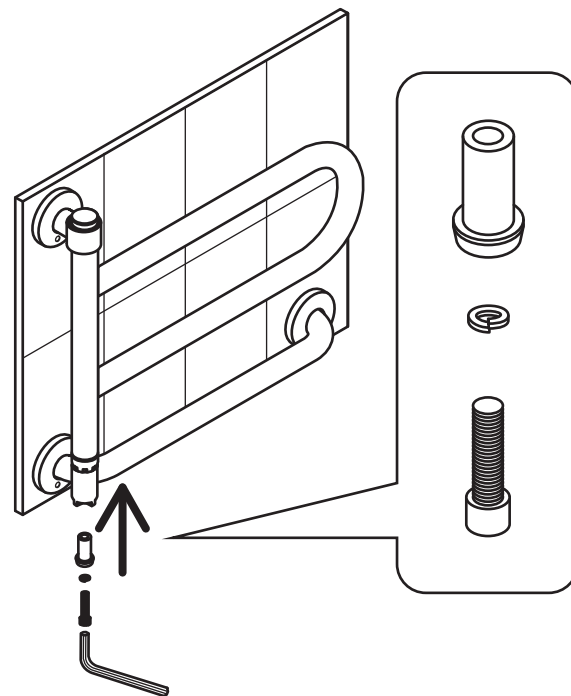
13


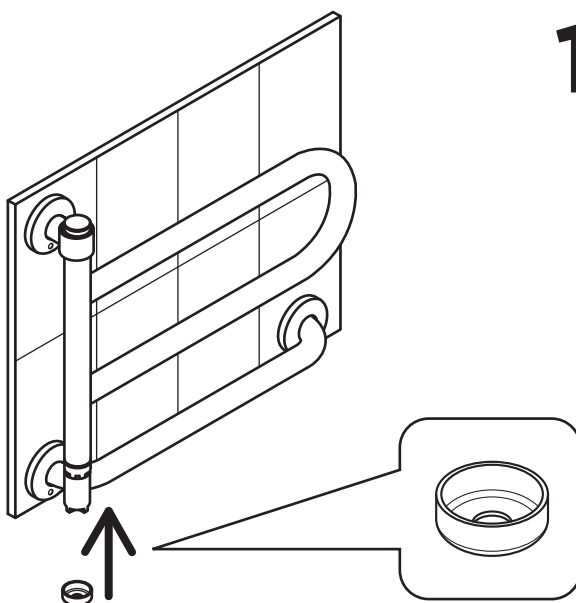
14

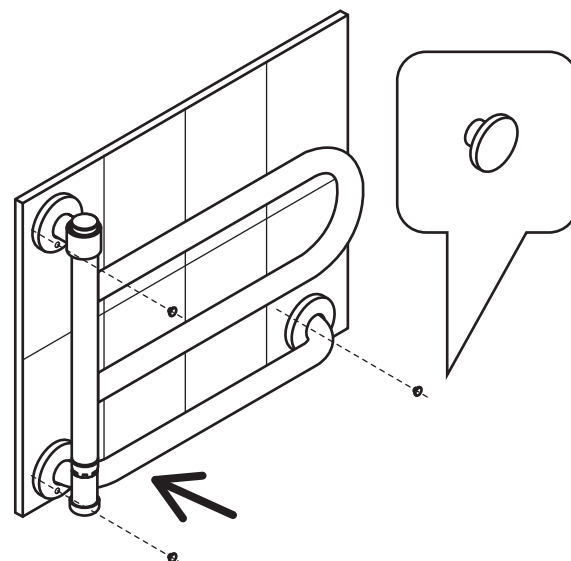
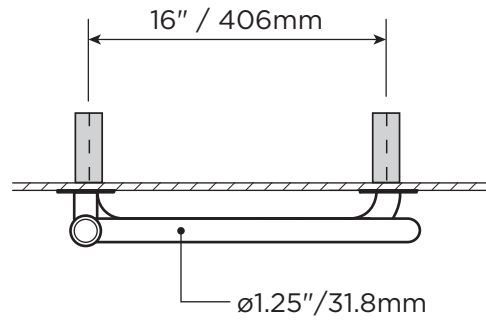
**MAX**  
300lbs / 136kg

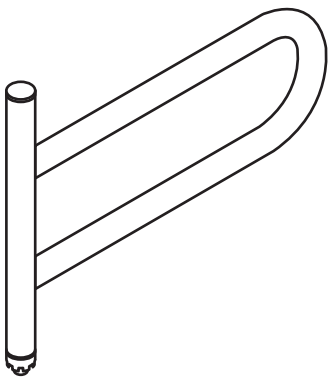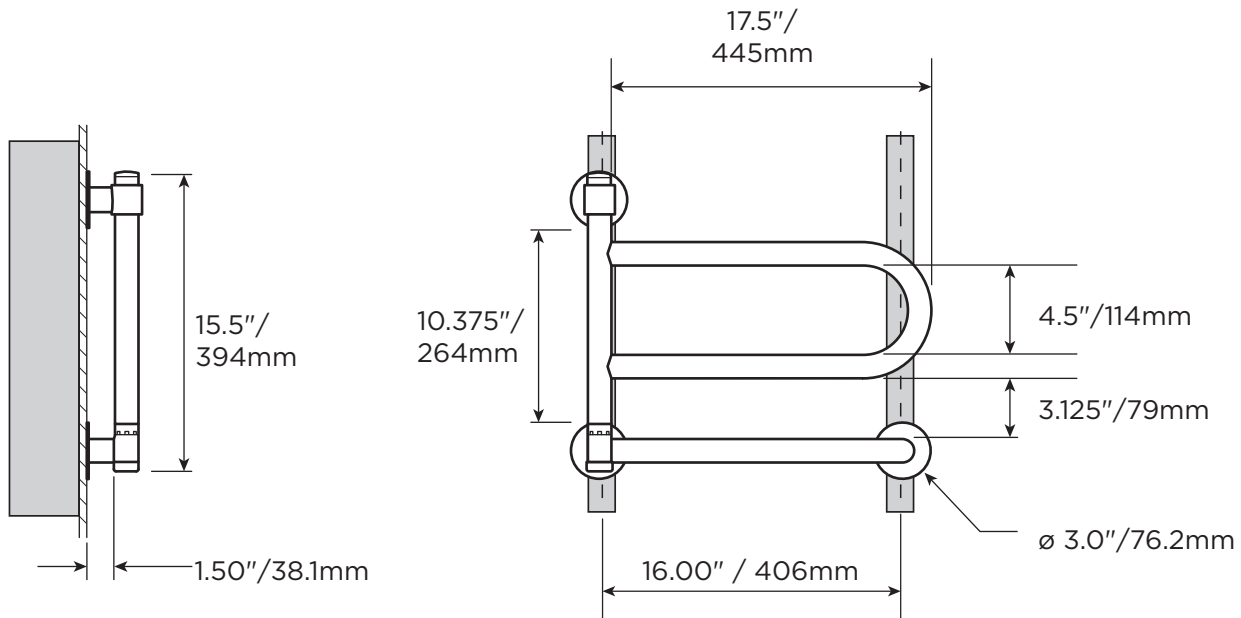
DBT200  
3.81lbs / 1.73kg









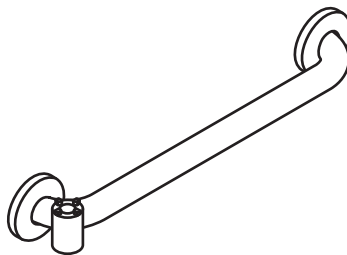
DBT201  
2.25lbs / 1.02kg

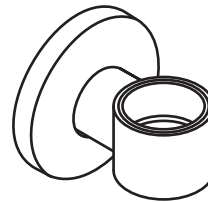DBT202  
0.45lbs / 0.20kg

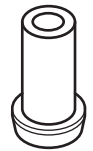DBT106  
0.05lbs / .02kg
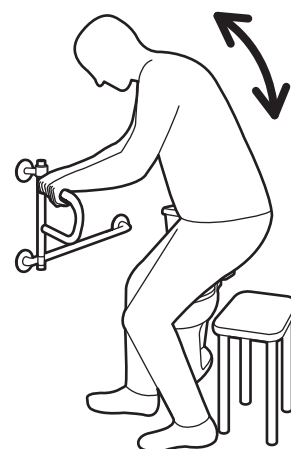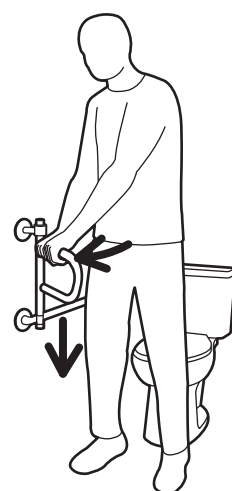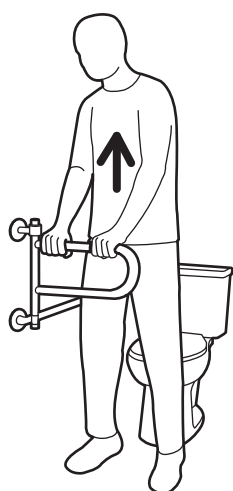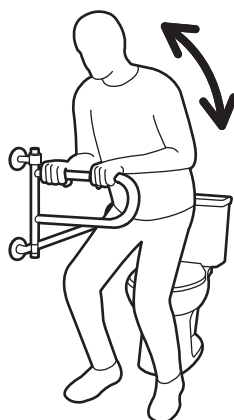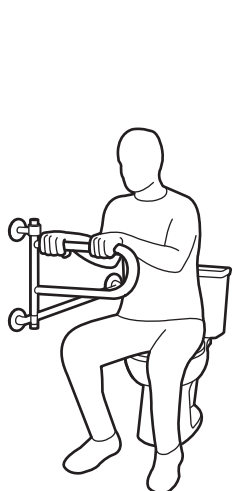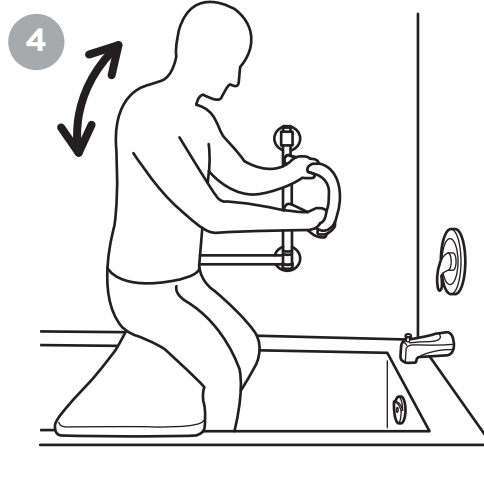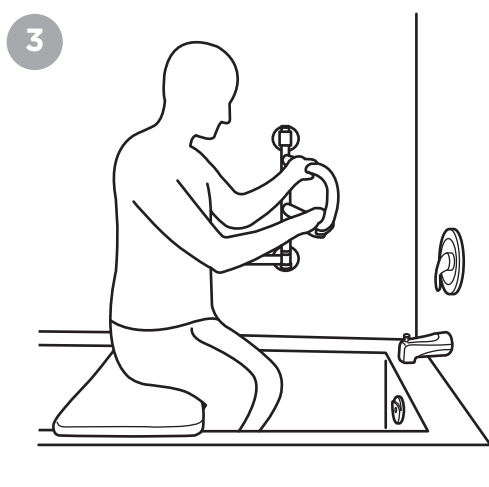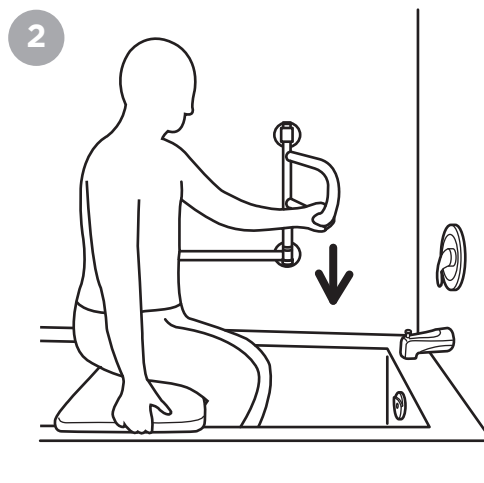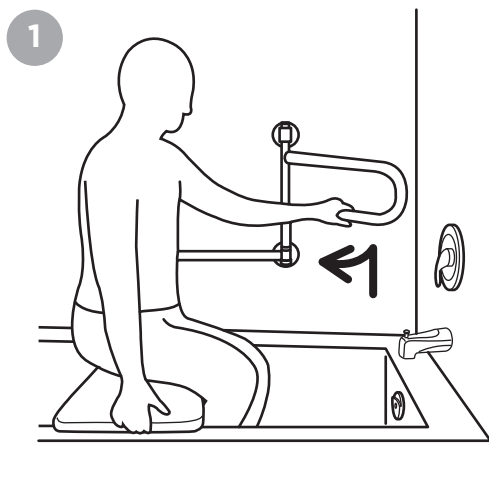
## OVERVIEW

Thank you for choosing the Dependa-Bar from HealthCraft. Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your Dependa-Bar is properly assembled, installed, and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your Dependa-Bar installed by a qualified contractor.

The DependaBar is a high quality, commercial grade pivoting and locking horizontal bar that connects to a wall mounted grab bar. The horizontal bar can be lifted, pivoted and lowered to the next locking position. The Dependa-Bar is intended for moderate vertical and horizontal load bearing to provide sitting and standing support for people with reduced mobility.

The Dependa-Bar is installed to a structural wall. The locking mechanism for the horizontal bar is welded to a horizontal grab bar: a castellated collar with 8 grooves (every 45 degrees) to provide a mechanism strong enough to support the user. It is important to set proper height and orientation of the Dependa-Bar to optimize the user's range of motion. The maximum user weight for this product is 300lbs / 136kgs. Note: Product is not intended to support full body weight. The DependaBar is not to be used in any other way than described above.

## WARRANTY

All HealthCraft Products are covered by a one year limited warranty with the exception of hand grips which are subject to normal wear. Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described hereon; buyers omission or neglect.

HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc.

HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon. Specifications subject to change without notice. **Manufacturer:** HealthCraft Products Inc. 2790 Fenton Road, Ottawa, Canada, K1T 3T7. Patent Pending. © 2024 HealthCraft Products Inc.

## DELIVERY CONTENTS / DEPENDA-BAR

## FIGURE A.

- |                                 |                               |
|---------------------------------|-------------------------------|
| 1. Top Flange Weldment + Cover  | 7. Lift Spacer                |
| 2. Plastic Sleeve               | 8. Vinyl Cap                  |
| 3. Plastic Plug                 | 9. Lock Washer, S/S           |
| 4. Pivot Rail Weldment          | 10. Wood Screw, #10, S/S (x9) |
| 5. Lower Rail Weldment + Covers | 11. Bolt, 1/4-20 x 1.50", S/S |
| 6. Plastic Plug (x3)            | 12. 3/16" Hex Key             |

## INSTALLATION WARNINGS

## FIGURE B.

1. Rail must be installed to walls that are structurally sound.
2. If desired installation location does not have adequate structure, add wood blocking.
3. Vertically aligned flanges must be installed into structural wood. To reinforce hollow wall behind the third flange, we recommend Easy-Mount Anchors (sold separately).
4. CAUTION - Do not install if structure behind walls is unknown.
5. CAUTION - Do not install into metal studs.
6. CAUTION - Do not install in fiberglass / acrylic shower enclosures with insufficient stiffness and strength.
7. CAUTION - Pinch and entrapment points in locking positions.
8. CAUTION - Rail and bathtub are slippery when wet.
9. CAUTION - Pinch point.

## SUGGESTED DEPENDA-BAR POSITIONS

## FIGURE C.

- A = stored out of the way  
 B = standing and transfer support  
 C = standing and transfer support  
 D = standing and transfer support  
 E = standing and transfer support

## INSTALLATION

## FIGURE D.

1. Position lower rail pivot feature on desired side (left or right).
2. Hold the lower grab bar against the wall and adjust with level.
3. Rotate the flange cover to gain access to other mounting holes, and mark the screw hole locations.
4. Drill pilot holes for fasteners.
5. Fasten the lower grab bar to the wall using appropriate fasteners and waterproofing. Inspect hardware regularly, and tighten if required.
6. Align the top flange 10-3/8" / 264mm above lower rail. Hold the top flange against the wall and adjust with level.
7. Rotate the flange cover to gain access to other mounting holes, and mark the screw hole locations.
8. Drill pilot holes for fasteners.
9. Fasten the top flange to the wall using appropriate fasteners and waterproofing. Inspect hardware regularly, and tighten if required.
10. Slide pivot rail up thru bottom of top flange.
11. Lower pivot rail on castellated collar.
12. Ensure pivot rail is engaged to castellated collar. Insert lift spacer, lock washer and bolt into the underside of the castellated collar, and secure to bottom of pivot rail with supplied hex key.
13. Cover exposed grooves of the castellated collar with vinyl cap.
14. Install plastic caps to cover the hole in the flange cover (x3).

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**Manufacturer:** HealthCraft Products Inc. 2790 Fenton Road, Ottawa, Canada, K1T 3T7. Brevet en instance. © 2024 HealthCraft Products Inc.

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